DENTSPLY
DETREY

Fix Adhesive – Liquid

Fix Adhesive – Spray

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Dansk

Fix Adhesive – Liquid

Tray Adhesive for Alginate Impression Materials

Fix Adhesive – Spray

Tray Adhesive for Alginate Impression Materials

Fix Solvent

Cleanser for Removal of Fix Adhesives

For more than three decades, the Fix products have been successfully used as valuable accessories when taking alginate impressions.

Fix Adhesive – Liquid and **Fix Adhesive – Spray** are solvent-based adhesives which increase the adhesion between alginate impression materials and the impression tray surface.

Fix Solvent is a spray to remove any remaining adhesive from the impression tray after impression taking and casting.

Caution: To be applied by or under the control of a dentist.

COMPOSITION

Fix Adhesive – Liquid:

Diethylenetriamine polymer
Propan-2-ol
Xylene
Erythrosine

Fix Adhesive – Spray:

Diethylenetriamine polymer
Propan-2-ol
Xylene
Erythrosine
Propellent: Butane

Fix Solvent:

Propan-2-ol
Xylene
Propellent: Butane

INDICATIONS

Fix Adhesive – Liquid and Fix Adhesive – Spray are tray adhesives for alginate impression materials.

Fix Solvent is used to remove Fix Adhesives from the impression tray after impression taking and casting.

CONTRAINDICATIONS

Use in patients or by persons with a known allergy to diethylenetriamine or any other components of the products.

WARNINGS

Fix Adhesive – Liquid:

The product is:

- highly flammable
- harmful by inhalation and in contact with skin
- irritating to the eyes, mucosa and skin.

Vapours may cause drowsiness and dizziness.

In isolated cases, allergic reactions and reversible inflammatory changes of the oral and nasal mucosa have been reported after exposure to the product.

Diethylenetriamine may cause sensitisation and allergic reactions in susceptible persons.

Fix Adhesive – Spray:

The product is:

- extremely flammable
- harmful by inhalation and in contact with skin
- irritating to the eyes, mucosa and skin.

Vapours may cause drowsiness and dizziness.

In isolated cases, allergic reactions and reversible inflammatory changes of the oral and nasal mucosa have been reported after exposure to the product.

Diethylenetriamine may cause sensitisation and allergic reactions in susceptible persons.

Fix Solvent:

The product is:

- extremely flammable
- harmful by inhalation and in contact with skin
- irritating to the eyes, mucosa and skin.

Vapours may cause drowsiness and dizziness.

In isolated cases, allergic reactions and reversible inflammatory changes of the oral and nasal mucosa have been reported after exposure to the product.

PRECAUTIONS

Use in a well ventilated place.

Keep away from sources of ignition. Do not smoke.

Avoid contact with skin, mucosa and eyes.

In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

Avoid inhalation of vapours and spray.

Keep container tightly closed.

INTERACTIONS WITH DENTAL MATERIALS

Fix Adhesive – Liquid and Fix Adhesive – Spray forms an adhesive bond between unset alginate impression material and the surface of the impression tray.

STEP-BY-STEP INSTRUCTIONS

Application of Fix Adhesive

1. Select suitable impression tray.
2. Prior to mixing the alginate impression material, apply a thin coat of Fix Adhesive – Liquid or Fix Adhesive – Spray onto the inner surface of the tray. Do not extend over the border of the tray onto surfaces which will not be covered by the impression material.
3. Allow the Fix Adhesive layer to dry for at least 3 minutes to become tacky.
4. Load the tray with alginate impression material, e.g. Blueprint® cremix, in the usual way.

Cleaning of Tray with Fix Solvent

Removal of the impression tray from the cast may be facilitated by warming the tray over a flame.

1. To remove any remaining adhesive, spray Fix Solvent on the inner surface of the impression tray. Leave for some seconds to dissolve the remaining adhesive.
2. Wipe off residual solvent with paper towel.

STORAGE

Keep container tightly closed.

Keep out of sunlight. Not to be stored at temperatures exceeding 50 °C.

Do not pierce or burn even after use.

BATCH NUMBER AND EXPIRY DATE

The batch number should be quoted in all correspondence which requires identification of the product.

Do not use after expiry date.

If you have any questions, please contact:

Manufacturer:
DENTSPLY DeTrey GmbH
De-Trey-Str. 1
78467 Konstanz
GERMANY
Phone +49 (0) 75 31 5 83-0

Distributor:
DENTSPLY Limited
Hamm Moor Lane
Addlestone, Weybridge
Surrey KT15 2SE
Phone (0 19 32) 85 34 22

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© DENTSPLY DeTrey 2001-07-10