DENTSPLY
DETREY

IRM®

Directions for Use _____	2
Gebrauchsanweisung _____	4
Mode d'emploi _____	6
Istruzioni per l'uso _____	8
Instrucciones de uso _____	10
Bruksanvisning _____	12

English

Deutsch

Français

Italiano

Español

Svenska

IRM®

IRM® is a reinforced zinc-oxide-eugenol composition for intermediate restorations lasting up to one year. It can also be used as a base.

INDICATIONS

1. For temporary restorations
2. Base material

DOSAGE AND HANDLING

Dosage

1 scoop of powder : 1 drop of liquid

The recommended powder/liquid ratio is 6 to 1 by weight. If a thinner mix is used, some physical properties of the set material may be sacrificed.

Before dispensing, fluff powder to ensure uniform bulk density of contents. Fill measuring scoop to excess without packing, then level scoop with spatula. Place powder on mixing pad. Dispense one drop of liquid for each level scoop of powder. After dispensing liquid, replace cap immediately to prevent evaporation and subsequent contamination. Keep powder and liquid separated until ready for spatulation.

Mixing

Mix at normal room temperature (23 °C) and 50% relative humidity. Use a suitable parchment pad and a spatulation technique which quickly and thoroughly combines 50% of the powder with the liquid. Bring the remaining powder into the mix in 2 or 3 increments and spatulate thoroughly. The mix will appear quite stiff and should be stropped (wiped vigorously) for 5 to 10 seconds. This type of mix results in a working consistency that is smooth and adaptable.

Mixing Time

The mix should be completed in approximately 1 minute.

Placement

Cavity preparation must provide for the mechanical retention of the material. Isolate field of operation in usual manner. Dry cavities with cotton wool; avoid prolonged air drying. Cavity varnishes are not necessary. A thin layer of a calcium hydroxide composition (Dycal®) may be used as a protective liner. Place IRM directly into cavity preparation. Use conventional method of matrix application, where indicated. Check occlusion and trim excess material if necessary.

Initial Set

Initial set is about 5 minutes from start of mix. When carving or adjustment is necessary, use a carbide bur (not a diamond). Increased temperature and/or humidity reduce available working time.

COMPOSITION

1 g powder contains:	Zinc oxide	0.8 g
	Polymethylmethacrylate	0.2 g
1 g liquid contains:	Eugenol	0.99 g

CONTRAINDICATIONS, SIDE EFFECTS

None reported.

NEGATIVE INTERACTIONS WITH DENTAL MATERIALS

Do not use zinc oxide/eugenol cements as a base for composite restorations as this may cause softening of the overlaying composite material.

SPECIAL NOTES

This product is formulated for dental use only.

Avoid contact with eyes. In case of contact, rinse eyes immediately with plenty of water and seek medical attention.

Replace cap tightly after use.

STORAGE

Store IRM separately from composite materials.

Keep away from direct sunlight.

SUPPLIED AS

606.61.500	Introductory Package:	Powder 40 g, Liquid 15 ml
606.61.520	Refill Package (powder):	Powder 40 g
606.61.510	Refill Package (liquid):	Liquid 15 ml

Differences in the various language versions of the Directions for use arise due to specific national regulations

If you have any questions, please contact:

Manufacturer:

DENTSPLY DeTrey GmbH
De-Trey-Str. 1
78467 Konstanz
GERMANY
Phone +49 (0) 75 31 5 83-0

Distributor:

DENTSPLY Limited
Hamm Moor Lane
Addlestone, Weybridge
Surrey KT15 2SE
Phone (0 19 32) 85 34 22

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© DENTSPLY DeTrey