DENTSPLY

X-flow™

Universal Flowable Restorative

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Magyar

Universal Flowable Restorative

X-flow™ is a smooth flowing, light-curing, radiopaque composite material which makes it ideal for restoration of small cavities, for lining and for luting of ceramic or composite inlays and onlays.

X-flow is delivered in Compulas®¹ for direct intra-oral application and is available in the Vita®² shades A2, A3, A4, B1, C2, and in TL (translucent) and O-A3 (an opaque shade).

X-flow adapts to the cavity walls without the use of hand instruments.

X-flow is used following application of Xeno®III, Single Step Self-Etching Adhesive or acid conditioning with e.g. DeTrey® Conditioner 36 followed by application of Prime&Bond®NT, a universal self-priming dental adhesive designed to bond the restorative to enamel and dentine.

Caution: For dental use only.

COMPOSITION

Strontium aluminosodium fluoro phosphor silicate glass
 Di- and multifunctional acrylate and methacrylate resins
 Diethylene glycol dimethacrylate (DGDMA)
 Highly dispersed silicon dioxide
 UV stabiliser
 Ethyl-4-dimethylaminobenzoate
 Camphor quinone
 Butylated hydroxy toluene (BHT)
 Iron pigments
 Titanium dioxide

INDICATIONS

- Minimal invasive restorations – this comprises:
 - Small, non occlusal stress-bearing restorations in anterior and posterior teeth
 - Extended pit & fissure sealings
 - Shallow cervical lesions.
- Margin repair of direct and indirect restorations.
- Cavity lining.
- Adhesive luting of porcelain or composite inlays and onlays, if they allow sufficient light transmission to the tooth-restoration interface.
- Blocking out undercuts in connection with preparation of indirect restorations.

CONTRAINDICATIONS

- Use on patients with a known allergy to dimethacrylate resins or any other of the components of X-flow.
- Permanent restorations of occlusal stress-bearing cavities in permanent posterior teeth.

¹ The Compula combines a **Compules®** Tip with a steel cannula for precise dosage and placement.

² Vita is a registered trademark of Vita Zahnfabrik H. Rauter GmbH & Co.

- Core build-up for full ceramic crowns.
- Luting of crowns or veneers.
- Luting of inlays and onlays that are impenetrable by dental curing lights or the walls of which exceed a thickness of 5 mm.
- When moisture control cannot be guaranteed during application (deep subgingival cavity margins).

WARNINGS

1. X-flow contains methacrylates that may be irritating to the eyes. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.
2. X-flow may cause sensitisation by contact with skin or mucous membranes in susceptible persons. After accidental contact, wash immediately with plenty of soap and water or rinse with plenty of water.
If sensitisation has occurred, discontinue use.
3. Use a gentle, even motion when exerting pressure on the applicator gun. Use of excessive force or sudden movement could create a potential hazard with extrusion.

The following adverse reaction has been associated with the use of acrylate monomers:

- Reversible inflammatory changes of the oral mucosa after accidental contact.

INTERACTIONS WITH DENTAL MATERIALS

Eugenol containing dental materials should not be used in conjunction with this product because they may interfere with hardening and cause softening of the polymeric components of the material.

If H₂O₂ has been used to clean the cavity, proper rinsing is essential. Higher concentration H₂O₂ may interfere with the setting of polymerisable material and should not be used prior to the application of Prime&Bond NT.

STEP-BY-STEP INSTRUCTIONS

I. DIRECT RESTORATIONS

1. Shade Selection and Cleaning

Shade selection should be made prior to the restorative procedure whilst the teeth are hydrated. Remove any extraneous plaque or surface stain with a rubber cup and pumice or a prophyl-paste like Nupro®. A Vita Lumin® Vacuum shade guide may be used. The X-flow shade corresponds to the central part of the respective Vita tooth.

2. Cavity Preparation

In all classes of cavity this may be kept to the minimum required for caries removal.

In cases where no cavity preparation has been made refresh the surface with a finishing bur. Wash surface thoroughly with air/water spray.

Remove rinsing water by blowing gently with an air syringe or blot-dry with a cotton pellet. Do not desiccate the dentine structure.

3. Pulp Protection

For direct or indirect pulp-capping cover the dentine close to the pulp with a hard-setting calcium hydroxide liner (e.g. Dycal®) leaving the rest of the cavity surface free for bonding with Prime&Bond NT.

4. Acid Conditioning and Bonding

Prior to application of X-flow, the cavity surfaces have to be treated with Xeno III, or acid-conditioned with e.g. DeTrey Conditioner followed by application of Prime&Bond NT. Please see the respective directions for use for these products.

5. Placement of X-flow

- 5.1 Insert Compula into the notched opening of the applicator gun barrel.
Be certain that the collar on the Compula is inserted first.
- 5.2 Remove the coloured cap from the Compula.
- 5.3 The Compula may be rotated to gain the proper angle of entrance into the cavity.
- 5.4 To dispense the material, use a slow, steady pressure. Excessive force is not necessary.
If more than gentle pressure is required, remove from patient field and check for obstruction.
- 5.5 Dispense X-flow directly into the cavity preparation. In deep cavities, incremental placement and curing (in 2 mm layers or less) is recommended to minimise polymerisation shrinkage.

6. Curing

Cure each increment separately with a VLC dental polymerisation unit for at least 20 seconds³ (shades A2, A3, B1, C2, TL) and for at least 40 seconds³ (shades A4 and O-A3), respectively. The tip of the light guide should be held as close as possible to the restoration during curing.

Important: Be sure to expose each area of the entire restoration to the curing light. Additionally, the restoration should be cured through lingual or buccal enamel walls.

7. Finishing

Finish immediately after curing. Gross excess material may be removed with fluted finishing burs or diamonds. Finishing is best achieved by using Enhance™ Finishing and Polishing Discs and interproximal finishing and polishing strips. A high final lustre can be obtained by applying PoGo™ One-Step Diamond Micro-Polisher or Prisma® Gloss™ and Prisma Gloss Extrafine polishing pastes.

II. INDIRECT RESTORATIONS

1. Pre-Treatment of Restoration

The internal surfaces should be treated according to the manufacturers' recommendations.

2. Moisture Control

Surface cleanliness is paramount for the development of adhesion.

3. Enamel and Dentine Pre-Treatment

Clean enamel and dentine prepared during a previous appointment with a rubber cup and pumice or a cleaning paste such as Nupro Prophylaxis Pastes. Wash thoroughly with water spray and then air-dry. Clean freshly cut enamel and dentine with water spray and then air-dry.

4. Conditioning and Application of Prime&Bond NT

Prior to application of X-flow, the cavity surfaces have to be acid-conditioned with e.g. DeTrey Conditioner followed by application of Prime&Bond NT. Please see the respective directions for use for these products.

5. Dispensing of X-flow

Dispense X-flow onto the restoration and/or onto the tooth preparation.

6. Placement of Restoration

Place restoration. Apply pressure and then release to allow the hydraulic pressure to dissipate. Repeat this procedure at least one more time and ensure that the restoration is completely seated.

⁵ Check curing light for minimum curing output of at least 500 mW/cm².

Remove as much excess as possible before the composite is light-cured.
If any excess composite remains after curing, remove with curette, scalers or finishing burs/discs, or strips.

7. Light-Curing

Expose all surfaces of the restorations to the dental curing light for 40 seconds each. Depending on the number of surfaces of the restoration, the diameter of the curing light outlet, and the size of the tooth, this results in up to 5 curing cycles per restoration.

8. Occlusal Adjustments and Finishing

Make occlusal adjustments and finish the margins with finishing burs or Enhance Finishing and Polishing Discs, Cups or Points.

Remove excess from proximal surfaces with diamond strips or with the diamond tips for the EVA contra-angle handpiece.

For polishing, apply PoGo One-Step Diamond Micro-Polisher or Prisma Gloss and Prisma Gloss Extrafine pastes. Work interdentially with aluminium oxide strips.

STORAGE

X-flow Compulas should be tightly closed immediately after use.

Keep out of sunlight.

To be stored at temperatures between 10 °C and 24 °C.

Humidity can adversely affect the properties of unsealed Compulas. Therefore keep Compulas tips sealed in their blister pack until use. Under normal ambient conditions, unsealed Compulas tips stay usable for about 4 weeks.

Inadequate storage conditions will shorten the shelf life and may lead to a malfunction of the product.

BATCH NUMBER AND EXPIRY DATE

The batch number should be quoted in all correspondence, which requires identification of the product.

Do not use after expiry date.

If you have any questions, please contact:

Manufacturer:
DENTSPLY DeTrey GmbH
De-Trey-Str. 1
78467 Konstanz
GERMANY
Phone +49 (0) 75 31 5 83-0

Distributor:
DENTSPLY Limited
Hamm Moor Lane
Addlestone, Weybridge
Surrey KT15 2SE
Phone (0 19 32) 85 34 22

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© DENTSPLY DeTrey 2002-12-13