---

**DENTSPLY**

**X**  **n** **O**<sup>®</sup> 

**Single Step Self-Etching Dental Adhesive**

## Standard

<b>Directions for Use</b> _____	<b>2</b>
<b>Gebrauchsanweisung</b> _____	<b>10</b>
<b>Mode d'emploi</b> _____	<b>20</b>
<b>Istruzioni per l'uso</b> _____	<b>28</b>
<b>Instrucciones de uso</b> _____	<b>36</b>
<b>Инструкция по применению</b> _____	<b>44</b>

## Special

<b>Instruções de utilização</b> _____	<b>54</b>
<b>Bruksanvisning</b> _____	<b>62</b>
<b>Gebruiksaanwijzing</b> _____	<b>70</b>
<b>Instrukcja użytkowania</b> _____	<b>78</b>
<b>Brugsanvisning</b> _____	<b>88</b>
<b>Οδηγίες χρήσης</b> _____	<b>96</b>
<b>Návod k použití</b> _____	<b>104</b>
<b>Használati utasítás</b> _____	<b>112</b>

**DENTSPLY**

**X e n o<sup>®</sup> III**

**Single Step Self-Etching Dental Adhesive**

**Xeno® III<sup>1</sup>** is a single step self-etching dental adhesive designed to bond light-cured restoratives to enamel and dentine.

**Xeno III** is available in bottles or as Unit Dose Delivery.

**Xeno III** has the following features:

- This single step bonding system acid-etches, primes and bonds both enamel and dentine simultaneously.
- No water rinsing after application.

## **COMPOSITION**

### **Liquid A**

2-hydroxyethyl methacrylate (HEMA)

Purified water

Ethanol

Butylated hydroxy toluene (BHT)

Highly dispersed silicon dioxide

### **Liquid B**

Phosphoric acid modified methacrylate (Pyro-EMA)

Mono fluoro phosphazene modified methacrylate (PEM-F)

Urethane dimethacrylate

Butylated hydroxy toluene (BHT)

Camphorquinone

Ethyl-4-dimethylaminobenzoate

---

<sup>1</sup> Xeno III is the third generation of a self-etching dental adhesive developed by DENTSPLY Sankin and marketed in Japan under the brand Xeno CF II Bond.

## INDICATIONS

Single step self-etching adhesive for **direct, light-cured** restorative materials.

## CONTRAINDICATIONS

- Do not use on patients who have a history of severe allergic reactions to methacrylate resins or any other of the components.
- Do not use for direct or indirect pulpcapping.
- Do not use with chemical or dual-cure materials.

## WARNINGS

**1. Eye contact:** Xeno III contains methacrylates which may be irritating to eyes. Before using this product wear protective glasses as well as covering the patient's eyes to protect from splashing material. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

- 2. Skin contact:** Xeno III contains polymerisable monomers which can cause skin sensitisation (allergic contact dermatitis) in susceptible individuals. If contact with skin, immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water after contact. If skin rash and sensitisation or other allergic reaction occurs, discontinue use and consult a physician.
- 3. Oral mucosa contact:** Avoid contact with oral soft tissues. If accidental contact occurs, remove adhesive with cotton pellet and rinse as soon as the clinical situation allows this. After prolonged contact with the adhesive the alveolar mucosa may turn white and/or numb due to the contact of the adhesive. This is a temporary phenomenon and should disappear within a day. If it lasts longer, the patient should contact his dental professional.
- 4. Ingestion:** Xeno III is acidic. Use caution to prevent the patient accidentally swallowing the product. If the patient accidentally swallows this product, have him drink lots of water.

## PRECAUTIONS

1. Xeno III is a light-cured, volatile material. If not protected from ambient light (closed CliXdish™), proceed immediately once the liquids have been placed into the mixing well or in the case of Unit Dose Delivery the foil has been pierced.
2. Xeno III is less effective on unground, unprepared, or uncut enamel. Overfilling of restorative onto unetched, unground, or uncut enamel could cause marginal discolouration.
3. Avoid excessive contact of the adhesive with gingival tissues, as this may cause a transient whitening and/or numbness of the alveolar mucosa. (See WARNINGS section).
4. Interactions with dental materials
  - 4.1 Insufficient data exist to support the use of Xeno III with indirect restorations.
  - 4.2 Dental materials containing eugenol should not be used in conjunction with this product because they may impede curing and cause softening of the polymeric components of the material.
  - 4.3 If H<sub>2</sub>O<sub>2</sub> has been used to clean the cavity, proper rinsing is essential. Higher concentration H<sub>2</sub>O<sub>2</sub> may interfere with the setting of polymerisable material and should not be used prior to the application of Xeno III.
5. Xeno III bottles should be tightly closed immediately after use. Do not interchange caps.
6. This product is intended to be used only as specifically outlined in the Directions for Use. Any use of this product inconsistent with the Directions for Use is at the discretion and sole responsibility of the practitioner.

## **ADVERSE REACTIONS**

1. Corneal damage may result with prolonged eye contact and/or exposure to Xeno III.
2. Allergic contact dermatitis and other allergic reactions may occur in susceptible individuals to Xeno III. (See CONTRA-INDICATIONS, WARNINGS, and PRECAUTIONS sections).
3. Transient whitening and/or numbness of the alveolar mucosa may occur after contact with Xeno III.

## **STEP-BY-STEP INSTRUCTIONS**

### **1. Cleaning and Preparation**

Cavity cleanliness is paramount for the development of adhesion.

In cases where no cavity has been prepared, clean the tooth surface with a rubber cup and pumice or a prophy-paste (e.g. Nupro®). Preparing a fresh surface with a finishing bur will significantly increase bond strength to enamel. In all situations where maximum retention is required instrument enamel (e.g. with a finishing diamond bur) to achieve optimal bonding results.

Wash surface thoroughly with air/water spray.

Remove rinsing water by blowing gently with an air syringe or blot-dry with a cotton pellet.

Do not desiccate the dentine structure.

## 2. Pulp Protection

For direct or indirect pulp capping cover the dentine close to the pulp (< 1 mm) with a hard-setting calcium hydroxide liner (e.g. Dycal®), leaving the rest of the cavity surface free for bonding with Xeno III.

## 3. Application

3.1 When using bottles:

3.1.1 Dispense an equal amount of Liquid A and Liquid B into a clean mixing well.

For optimum dosage, proceed as follows:

### Liquid A

Shake bottle two to three times prior to use.

Incline bottle.

If liquid does not drop, squeeze.

After use, close bottle with green cap.

### Liquid B

Incline bottle; wait for approx. two seconds.

Squeeze until liquid drops.

After use, close bottle with black cap.

3.1.2 Mix liquid thoroughly for approx. five seconds with the applicator tip supplied. If protected from ambient light (closed CliXdish), the mixed material stays functional for up to 60 minutes!

3.2 When using Unit Dose Delivery:

3.2.1 It is not necessary to open the larger round cap. Open the triangular tab. Rotate the container so that the open tab is at the bottom; incline and shake container until one applicator emerges.

3.2.2 Detach a Unit Dose by folding along the perforation and pull the printed side upwards. Hold the Unit Dose blister between thumb and forefinger. Firmly press green area to force liquid into orange mixing and dispensing well.

- 3.2.3 Perforate foil on top of mixing and dispensing well by placing the applicator at right-angles to the centre of the orange circle. Enlarge the opening with circular movements to maximum width. Create a homogeneous mixture by stirring for at least 5 seconds. To ease removal of the adhesive the Unit Dose may be inclined.
- 3.3 With the applicator provided, apply generous amounts of Xeno III to thoroughly wet all cavity surfaces.
- 3.4 Leave undisturbed for at least 20 seconds.
- 3.5 Uniformly spread the adhesive using a gentle stream of oil-free air for at least 2 seconds until the adhesive no longer flows. This ensures proper removal of solvent. Avoid thinning the adhesive layer by excessive air-drying.
- Do NOT water rinse.**

- 3.6 Cure<sup>2</sup> the adhesive with a light-curing unit for at least 10 seconds.
- 3.7 Place restorative immediately following manufacturer's instructions.

### **STORAGE**

Keep out of direct sunlight and protect from moisture. Store at temperatures between 2 °C and 24 °C. Inadequate storage conditions will shorten the shelf life and may lead to a malfunction of the product. Close caps tightly after use.

---

<sup>2</sup> Check curing light for minimum curing output of at least 500 mW/cm<sup>2</sup>.

**BATCH NUMBER**  **AND EXPIRY DATE** 

The batch code should be quoted in all correspondence, which requires identification of the product.

Do not use after expiration date.

If you have any questions, please contact:

**Manufacturer:**  
**DENTSPLY DeTrey GmbH**  
**De-Trey-Str. 1**  
**78467 Konstanz**  
**GERMANY**  
**Phone +49 (0) 75 31 5 83-0**

**Distributor:**  
**DENTSPLY Limited**  
**Hamm Moor Lane**  
**Addlestone, Weybridge**  
**Surrey KT15 2SE**  
**Phone (0 19 32) 85 34 22**

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**© DENTSPLY DeTrey 2006-06-01**
**DENTSPLY DeTREY GmbH**