DENTSPLY

DeTrey[®]Zinc

Zinc Phosphate Cement

Zinkphosphatzement

Ciment oxyphosphate de zinc

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Cemento de fosfato de zinc

Zinkfosfatcement

Directions for Use	3
Gebrauchsanweisung	5
Mode d'emploi	7
Istruzioni per l'uso	9
Instrucciones de uso	11
Brugsanvisning	13

DeTrey®Zinc

DeTrey®Zinc is a fine grain zinc phosphate cement which complies with the requirements of ISO 9917.

The product is a powder-liquid system for manual mixing.

1 Product Information

1.1 Composition

Powder: Zinc oxide
Magnesium oxide

Liquid: Phosphoric acid

1.2 Indications for use

DeTrey Zinc is indicated for:

- Cementation of inlays, crowns, bridges, orthodontic appliances and intra-oral splints
- Cavity lining / bases under amalgam or composite materials
- Temporary restorations
- Cement build-up

1.3 Contraindications

DeTrey Zinc is contraindicated for direct or indirect pulp-capping.

2 Safety Notes

2.1 Warnings

DeTrey Zinc contains phosphoric acid which causes burns.

Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.

Avoid contact with oral tissues and skin. If accidental contact occurs, flush affected area with generous amounts of water.

2.2 Precautions

This product is intended to be used as specifically outlined in these directions for use. Any use of this product inconsistent with the directions for use is at the discretion and sole responsibility of the dental practitioner.

2.3 Adverse Events

Pulpal irritation may occur in rare cases with zinc phosphate cements.

2.4 Interactions

No negative interactions with other dental materials are reported.

3 Step by Step instructions

3.1 Dentine pre-treatment and pulp-protection

Dentine in close proximity to the pulp should be covered with a calcium hydroxide liner (Dycal®). The preparation is washed and dried, but not dehydrated. Avoid contamination with water/saliva during application and setting of cement.

3.2 Mixing

Place an appropriate amount of powder and liquid on a clean and dry glass slab. Divide powder into small portions using a stainless steel spatula. Add these portions one by one to the liquid, spatulating thoroughly after each addition. Do not add liquid once mixing has been commenced.

For cementation purposes and for use as a liner, a smooth and creamy mix should be obtained.^{1,2}

For use as a base, cement build-up or temporary filling, more powder may be incorporated to obtain a putty-like consistency.³

3.2.1 Mixing time

The mixing time should be approximately 1½ minutes. Rapid incorporation of powder accelerates the setting time, slow mixing retards it.

3.2.2 Working time

The working time from start of mix is approximately 2½ minutes for the cementation consistency.

3.2.3 Setting time⁴

The setting time from end of mix for:

- **Cementation** is 5-6 minutes
- **Base consistency** is 2-3 minutes

3.3 Finishing

Excess material may be removed at once or after completion of setting time (see above). When used as a temporary restoration, finishing burs are recommended. For removal of surplus material after crown cementation, appropriate hand instruments, like a dental probe, and dental floss are used.

3.4 Clean-up

Zinc phosphate cements adhere to instruments and glass slabs. Immediately after use, either clean instruments with water or drop in disinfectant or water to facilitate subsequent cleaning.

4 Hygiene

In order to avoid contamination of the powder and liquid bottles, prevent exposure to spatter or spray of body fluids and touch only with disinfected hands/gloves.

5 Storage and Disposal

Store at temperatures between +10 °C and +24 °C. Inadequate storage may shorten shelf life. Keep out of sunlight. Replace cap of powder and liquid bottle tightly after use.

Dispose the product in accordance with the local regulations.

6 Batch number () and expiration date ()

1. Do not use after expiration date.
2. The following numbers should be quoted in all correspondence.
 - Reorder Number
 - Batch number
 - Expiration date

7 Contact

If you have any questions, please contact:

Manufacturer ()
DENTSPLY DeTREY GmbH
De-Trey-Str. 1
78467 Konstanz
GERMANY
Phone +49 (0) 75 31 5 83-0

Distributor
DENTSPLY Limited
Hamm Moor Lane
Addlestone, Weybridge
Surrey KT15 2SE
Phone (0 19 32) 85 34 22

© DENTSPLY DeTREY 2006-10-19

¹ A powder : liquid ratio of 2.8 g : 1 g is proposed for cementation and testing purposes according to ISO 9917.

² Mixing and application of the material should be done at normal room temperature. Higher temperatures accelerate the setting time of the cement. Under warm conditions or if a long working time is required, it is therefore recommended that the glass slab be cooled. For testing purposes according to ISO 9917 the material is tested at 23 °C ± 1 °C and with a relative humidity of 50 ± 10 %.

³ A powder : liquid ratio of 5.6 g : 1 g is proposed for use as a base and for testing purpose according to ISO 9917.

⁴ Net setting time determined according to ISO 9917.

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DENTSPLY DeTrey GmbH