Addendum

To Cavitron® Directions for Use

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Warnings**For all Units (including OEM Scalers):**

- It is the responsibility of the Dental Healthcare Professional to determine the appropriate uses of this product and to understand:
 - the health of each patient,
 - the dental procedures being undertaken,
 - and applicable industry and governmental agency recommendations for infection control in dental healthcare settings, requirements, and regulations for safe practice of dentistry; and
 - these Directions for Use in their entirety, including Precautions, Infection Control, and System Care.
- Where asepsis is required or deemed appropriate in the best professional judgment of the DHCP, this product should not be used.
- During boil-water advisories, this product should not be operated as an open water system (e.g. connected to a public water system). DHCP should discontinue use on patients and contact the local water authority to determine when it is safe to continue use of this product. When the advisory is cancelled, the local water authority should provide guidance for flushing of waterlines. All incoming waterlines from the public water system inside the dental office (e.g., faucets, waterlines, and dental equipment) should be flushed in accordance with manufacturer’s instructions for a period of 2 to 5 minutes.
- It is suggested that prior to beginning treatment, patients should rinse with a known antimicrobial such as Chlorhexidine Gluconate 0.12%. Rinsing with an antimicrobial reduces the chance of infection and reduces the number of microorganisms the patient might release in the form of aerosols during treatment.
- The use of High Volume Saliva Evacuation to reduce the quantity of aerosols released during treatment is highly recommended.
- Failure to follow the recommendations for environmental operating conditions, including input water temperature, could result in injury to patients or users.

System Precautions:

- Always ensure that the electrical connections on the handpiece cable and the Steri-Mate® Handpiece or JET-Mate™ Handpiece are clean and dry before assembling them for use.

Water Supply Recommendations**For OEM (scalars connected via the Dental Unit) Scalers:**

- This product is designed to assist in eliminating susceptibility to retraction of oral fluids. To ensure adequate protection from cross-contamination from other devices connected to the Dental Unit, it is highly recommended that the Dental Unit be installed with anti-retraction features. Additionally, the Dental Unit, including the anti-retraction features, must be properly maintained and periodically tested. For more information, please contact your Dental Unit manufacturer.

For all Units (including OEM Scalers):

- As a medical device, this product must to be installed in accordance with applicable local, regional, and national regulations, including guidelines for water quality (e.g. drinking water). As an open water system, such regulation may require this device to be connected to a centralized water control device. The Cavitron® DualSelect™ Dispensing System may be installed to allow this unit to operate as a closed water system. Cavitron® Select™ SPS™ and Cavitron® Select™ Scalers may also be installed using the Cavitron Select Dispenser for operation as a closed water system.

Water Line Requirements**For all Units (including OEM Scalers):**

- Incoming water temperature to the Cavitron system should not exceed 25°C (77°F). If needed, a water control device should be installed to maintain a temperature within this specification, or a Cavitron DualSelect™ Dispensing System attached to allow this system to be operated as a closed water system.

Cavitron Ultrasonic Inserts**Insert Marking**

Date (YDDD = Single digit Year and triple digit day of the year. Example: 6040 = fortieth day of 2006 or February 9, 2006).

PERIOD OF WARRANTY**Full Two Year Warranty**

Cavitron® SPS™ Ultrasonic Scaler

Cavitron® Select™ SPS™ Ultrasonic Scaler

Cavitron® G98A SPS OEM Ultrasonic Scaler

Cavitron® G90 SPS OEM Ultrasonic Scaler

Cavitron® JET with SPS™ Combination System - Ultrasonic Scaler and Air Polishing Prophylaxis System

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DENTSPLY DeTrey GmbH