Dyract[®] XP

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Dyract® XP

The Caries Preventing Restorative

CAUTION: For dental use only.

USA: Rx only.

Content	Page
1 Product description	2
2 Safety notes	3
3 Step-by-step instructions	5
4 Hygiene	7
5 Lot number and expiration date	7

1 Product description

Dyract® XP restorative is a light curing restorative material for all cavity classes in anterior and posterior teeth.

Dyract® XP restorative combines the fluoride release of glass ionomer materials with the strength and esthetics of a light curing composite.

eXtra Care

Dyract® XP restorative restorations continuously release fluoride ions, act on the tooth restoration interface as an acid buffer and effectively support the prevention of approximal caries.

The choice of **Dyract® XP** restorative for patients prone to risk of caries is therefore advisable as an additional care against the onset of new caries.

1.1 Delivery forms

- Predosed Compules® Tips

1.2 Composition

- Urethane dimethacrylate (UDMA)
- Carboxylic acid modified dimethacrylate (TCB resin)
- Triethyleneglycol dimethacrylate (TEGDMA)
- Trimethacrylate resin (TMPTMA)
- Dimethacrylate resins
- Camphorquinone
- Ethyl-4(dimethylamino)benzoate
- Butylated hydroxy toluene (BHT)
- UV stabilizer
- Strontium-alumino-sodium-fluoro-phosphor-silicate glass
- Highly dispersed silicon dioxide
- Strontium fluoride
- Iron oxide pigments and titanium oxide pigments

1.3 Indications

- Direct restorations of all cavity classes in anterior and posterior teeth. Cavity width must be less than 2/3 of the intercuspal distance.

1.4 Contraindications

- Use with patients who have a history of severe allergic reaction to methacrylate resins or any of the other components.
- Class I and II cavities the width of which exceeds 2/3 of the intercuspal distance.
- Core build-up.

1.5 Compatible adhesives

The material is to be used following application of a compatible methacrylate-based dentin/enamel adhesive such as Dentsply Sirona adhesives designed for use with visible light cured composite restoratives.

Xeno[®], self etching adhesives and Prime&Bond[®] NT, etch and rinse adhesive are especially recommended.

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

The material contains methacrylates and polymerizable monomers which may be irritating to skin, eyes and oral mucosa and may cause sensitization by skin contact and allergic contact dermatitis in susceptible persons.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with water and soap. In case of skin sensitisation or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.

- The Compules® Tips cannot be reprocessed. To prevent Compules® Tips when used for indirect application from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the devices are handled offside the dental unit with clean/disinfected gloves. Discard devices if contaminated.
- Compules® Tips may be used for direct intraoral application of restorative material into a cavity or for indirect application by first placing the restorative material on a pad. After intraoral use, discard Compules® Tips and do not reuse in other patients in order to prevent cross-contamination.
- Use of Compules® Tips with the Compules® Tips Gun is recommended.
- The Compules® Tips Gun is only intended to be used with Dentsply Sirona Compules® or Compula® Tips.
- For further information please refer to the Compules® Tips Gun Instructions for Use.
- The material should extrude easily. DO NOT USE EXCESSIVE FORCE. Excessive pressure may cause the Compules® Tip to rupture or to eject from the Compules® Tips Gun.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.
- Interactions:
 - Do not use eugenol- and hydrogen peroxide-containing materials in conjunction with this product since they may interfere with hardening of the product.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store in sealed packs at temperatures between 10 °C and 24 °C (50 °F and 75 °F).
- Keep out of direct sunlight and protect from moisture.
- Do not freeze.
- Do not use after expiration date.

Humidity can adversely affect the properties of the material. For unsealed Compules® Tips, the following is recommended:

- Store in a dry environment (< 80% relative humidity).
- Use within 4 weeks.

3 Step-by-step instructions

3.1 Shade selection

Shade selection is made whilst the teeth are hydrated and prior to the restorative procedure.

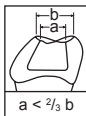
1. Remove any extraneous plaque or surface stain.
2. Use the VITAPAN^{® 1} classical shade guide. Use the central part of the respective VITA^{® 1} tooth for shade selection.
3. Select the most suitable Dyract[®] XP.

3.2 Cavity preparation

Cavity width.

Cavity width (a) must be less than 2/3 of the intercuspal distance (b).
The graphic on the right graph also appears on all outer packages.

1. Clean the tooth surface with a fluoride free prophylaxis paste (e.g. Nupro[®] prophylaxis paste).
2. Prepare cavity (unless not necessary, e.g. cervical lesion).
3. Use adequate isolation such as rubber dam.
4. Rinse surface with water spray and carefully remove rinsing water. Do not desiccate the tooth structure.

3.2.1 Matrix placement and wedging (Class II restorations)

1. Place a matrix (e.g. AutoMatrix[®] matrix system or Palodent[®] matrix system) and wedge. Burnishing of the matrix band will improve contact and contour. Pre-wedging or BiTine[®] ring placement is recommended.

3.3 Pulp protection, tooth conditioning/dentin pre-treatment, adhesive application

Refer to adhesive manufacturer's Instructions for Use for pulp protection, tooth conditioning and/or adhesive application. The Dentsply Sirona Xeno[®] adhesives are applied in the self etch technique. XP Bond[®] is applied in the etch and rinse technique. Prime&Bond[®] NT may in combination with the Dyract[®] restoratives be applied in both techniques. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately to placement of the material.

3.4 Application

Apply material in increments immediately after the application of the adhesive. The material is designed to be cured in increments up to a 2 mm depth/thickness.

Using Compules[®] Tips

1. Insert Compules[®] Tip into the notched opening of the Compules[®] Tips Gun. Be certain that the collar on the Compules[®] Tip is inserted first.
2. Remove the colored cap from the Compules[®] Tip. The Compules[®] Tip may be rotated 360° to gain the proper angle of entrance into the cavity or to the mixing pad. DO NOT USE EXCESSIVE FORCE.

Indirect use: to prevent Compules[®] Tip from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the Compules[®] Tip is handled off-side the dental unit with clean/disinfected gloves. Dispense the necessary amount of material onto a mixing pad and protect against light. Apply material into the cavity with a plastic instrument.

¹ Not registered trademarks of Dentsply Sirona, Inc.

- or -

Dispense the material directly into the cavity preparation using a slow, steady pressure.

3. To remove the used Compules® Tip, be sure that the Compules® Tips Gun plunger is pulled back completely by allowing the handle to open to its widest position. Apply a downward motion to the front end of the Compules® Tips and remove.

Excessive force.

Injury.

1. Apply slow and steady pressure on the Compules® Tips Gun.
2. Do not use excessive force. Compules® Tip rupture or ejection from Compules® Tips Gun may result.

3.5 Light curing

1. Light cure each area of the restoration surface with a suitable curing light². Minimum light output must be at least 500 mW/cm². Refer to curing light manufacturer's Instructions for Use for compatibility and curing recommendations.
2. Cure each layer according to the table below.

Insufficient curing.

Inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing cycle.
3. Check minimum irradiance.
4. Cure each area of each increment for the recommended curing time.

Curing time for 2 mm increments

Output	≥ 500 mW/cm²
Shades A2; A3; A3,5; B1	10 sec

3.6 Finishing and polishing

1. Contour the restoration using finishing burs or diamonds.
2. Use Enhance® finishing instruments and interproximal finishing strips. In patients with adequate oral hygiene, the high final luster of the restoration comes with use.

For contouring, finishing, and/or polishing, follow the manufacturer's Instructions for Use.

² Curing light designed to cure materials containing camphorquinone (CQ) initiator. Peak of spectrum in the range of 440-480 nm.

4 Hygiene

4.1 Compules® Tips Gun

For reprocessing instructions please refer to the Instructions for Use of the Compules® Tips Gun, which is available on our webpage at www.dentsply.eu/IFU. If requested, we will send you a free printed copy of the Instructions for Use in the language you require within 7 days. You can use the order form provided on our webpage for this purpose.

4.2 Compules® Tips – after intraoral use

Cross-contamination.

Infection.

1. Do not reuse Compules® Tips after intraoral use and/or if contaminated.
2. Dispose the intraorally used and/or contaminated Compules® Tips in accordance with local regulations.

5 Lot number () and expiration date ()

1. Do not use after expiration date.
ISO standard is used: “YYYY-MM” or “YYYY-MM-DD”.
2. The following numbers should be quoted in all correspondence:
 - Reorder number
 - Lot number
 - Expiration date

© Dentsply Sirona 2017-01-19

[These Instructions for Use are based on Master Version 02]

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Dentsply DeTrey GmbH