# **office** **illuminé™**

**In-Office Tooth Whitener**

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## Instructions for Tray Fabrication


Patient: \_\_\_\_\_

Dentist: \_\_\_\_\_

Date: \_\_\_\_\_

Whitening Technique:


- Illuminé home
- Illuminé office or Illuminé office & home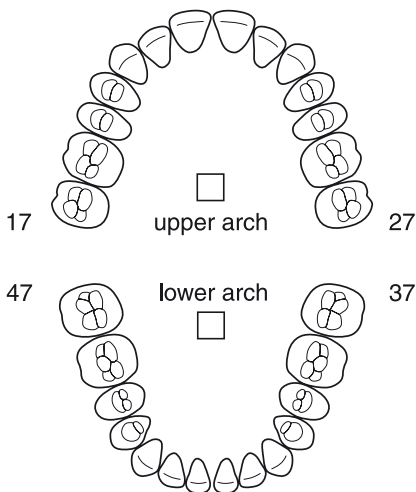


Teeth and arch to be treated are marked by a cross 

Spacer thickness:  
Illuminé home only


0.5 - 1 mm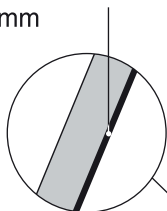

Illuminé office  
Illuminé office & home  
3 - 4 mm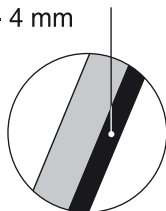

Spacer

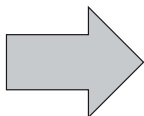gingival  
"seal"

Spacer: 1 mm  
short of gingiva

Tray: 0.5 mm  
short of gingiva

**Note:** Gel reservoir is created only for teeth to be treated

<b>Directions for Use</b>	<b>2</b>
<b>Gebrauchsanweisung</b>	<b>7</b>
<b>Mode d'emploi</b>	<b>13</b>
<b>Istruzioni per l'uso</b>	<b>19</b>
<b>Instrucciones de uso</b>	<b>25</b>
<b>Инструкция по применению</b>	<b>31</b>
<b>Bruksanvisning</b>	<b>37</b>
<b>Gebruiksaanwijzing</b>	<b>42</b>
<b>Instrukcja użytkowania</b>	<b>48</b>
<b>Brugsanvisning</b>	<b>54</b>
<b>Návod k použití</b>	<b>59</b>
<b>Kullanım Talimatları</b>	<b>64</b>

# office Illuminé™

## In-Office Tooth Whitener

**IMPORTANT NOTE:** Illuminé™ office is unique in both technique and formulation. Because it is different to other in-office tooth whiteners, it is imperative to read and understand the instructions thoroughly and to exercise application in vitro to confidently administer this new technique.

Illuminé office is a 15% hydrogen peroxide-based treatment which will provide your patients with significantly noticeable whitening results in 30 to 60 minutes. The process requires no light or heat activation.

Illuminé office is recommended as a pre-treatment "boost" to Illuminé home Tooth Whitening Gel, or as a stand-alone treatment, completed in 1 to 3 office visits. If used as a boost to Illuminé home, the take-home regimen may be abbreviated to a 3-day treatment.

Each kit contains two (2) syringe sets and an applicator tip for each. Each syringe set comprises one syringe containing a 30% hydrogen peroxide solution (syringe A) and one powder syringe (syringe B) with an attached stopcock. Each syringe set contains enough material to whiten 1 arch.

### COMPOSITION

Syringe A contains: 30% hydrogen peroxide

Syringe B contains: Poly (Methyl Vinyl Ether/Maleic Anhydride) mixed calcium/sodium salts, titanium dioxide

### INDICATIONS

1. Tooth discolouration caused by:
  - Staining from food, drinks and tobacco with penetration into the tooth substance
  - Age-dependent degenerative changes
  - Tetracycline or minocycline medicamentation
  - Fluorosis, especially brown pigmentation
  - Pulp necrosis and/or endodontic treatment.
2. Masking of enamel mottling.
3. Genetically determined (born with) discoloured teeth.
4. Whitening of discoloured teeth prior to restorative and/or prosthodontic measures, e. g. veneering.

The best results are with patients whose teeth are discoloured to yellow, orange or light brown.

### CONTRAINDICATIONS

1. Use during pregnancy or lactation.
2. Known allergy to hydrogen peroxide or other ingredients of the products.
3. Extrinsic stains which can easily be removed by professional tooth cleaning and which can be prevented by thorough oral hygiene.

4. Use in heavy smokers unless they refrain from smoking during the treatment.

## **WARNINGS**

1. Avoid exposure to heat and to light which may cause outgassing of the hydrogen peroxide solution. Hydrogen peroxide syringes (syringe A) may be under slight pressure. Remove cap carefully.
2. Syringe A contains 30% hydrogen peroxide which is corrosive and causes burns. The mixed material contains 15% hydrogen peroxide which is irritating.
3. Extrude the material into the tray immediately after mixing to avoid pressure build-up in the closed syringe set.
4. Avoid contact with oral tissues, eyes and skin. If accidental contact occurs, flush affected area with generous amounts of water. Contact may cause transient whitening. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.
5. Hydrogen peroxide causes damage to textile fabrics. Avoid contact.
6. Illuminé was designed to be used with a custom-made whitening tray. Because the material sets to a rubbery semi-solid consistency in the tray, no gingival isolation is required! Note however, if the material makes prolonged contact with the gingiva or mucous membrane during the treatment, it can cause irritation and does cause transient whitening of the gingiva.

## **PRECAUTIONS**

1. Familiarise yourself with the components of the syringe sets.
2. Wear gloves and protective eyewear when working with hydrogen peroxide.
3. Always mix the product away from the patient.
4. Ensure that the bleaching tray has definitive, self-contained reservoirs. The tray must fit well, and not permit material to extend onto the gingival tissue. If these conditions are not present, a new tray should be fabricated.
5. Patients should always be warned of the sensitivity of the gingival tissues during the treatment and that gingiva in contact with Illuminé will turn white but return to its normal colour after about 1 day.
6. Treat any uncared for caries and repair edge gaps in existing restorations, enamel gaps or cracks, and free-lying regions of dentine, prior to whitening, to reduce the probability of the material reaching the pulp, resulting in hypersensitivity or inflammatory reaction.
7. Avoid exposure of known hypersensitive teeth and teeth with expanded pulp cavities to the whitening agent.
8. Particular care should be given during insertion of the tray to carefully remove excess material as well as misapplied material from mucosal tissues and to avoid contact to skin and eyes.
9. Keep out of reach of children.

## **INTERACTIONS WITH DENTAL MATERIALS**

Existing tooth coloured restorations may not match the lighter shade of teeth after the treatment and may need to be replaced.

The Illuminé office System should be used before the placement of composites, veneers or crowns, in order to maintain a close match of tooth colour. It is also recommended that the patient wait a minimum of two weeks after the treatment to achieve optimum adhesion and to stabilise the tooth shade before performing adhesive/tooth-coloured anterior restorations.

## **ADVERSE REACTIONS**

Patients may experience sensitivity from the use of hydrogen peroxide-based products (exposed root surfaces, restorations with microleakage, exposed dentine, interproximal labial lingual areas).

## STEP-BY-STEP INSTRUCTIONS FOR DENTAL OFFICE

### Initial Consultation

1. Review the patient's medical and dental history.
2. Perform an intraoral examination of the soft tissue, periodontal and teeth health.
3. Inform the patient that depending on the nature of the stain, an extended treatment may be required to attain the desired effect.
4. A full prophylaxis (using Nupro® Prophy Paste) and fluoride treatment is recommended before starting the bleaching process.
5. The complete procedure should be discussed with the patient, reviewing any potential problems and side effects.
6. Inform the patient that any existing tooth coloured restorations may not match the lighter shade of teeth after the treatment and may need to be replaced.
7. Use a suitable shade guide, e. g. Biodent<sup>1</sup> or Vita<sup>®2</sup> Lumin Vacuum shade guide, to make a shade determination at baseline. Enter the determined shade into the patient's record.

### Take Impression

Make an alginate or elastomer impression of the arch to be treated. (It is highly recommended that the teeth be treated one arch at a time, so that the patient can see the changes.)

### Tray Fabrication and Communication with Dental Laboratory

It is recommended that the step-by-step instructions for the dental laboratory which are part of the starter kit of Illuminé home be used for communication with the dental laboratory. The tray for Illuminé office requires however the formation of reservoirs of at least 3 mm thickness as opposed to the 0.5 to 1 mm thickness if making a standard home tray. The Illuminé office whitening tray may also be used for subsequent treatment with Illuminé home. Directions for tray fabrication please also find in the appendix to this information.

### Gingival Isolation

Gingival isolation should not be required if the instructions are followed, and:

- a well-fitting bleaching tray has been prepared with adequate (3 to 4 mm depth) reservoirs for each tooth to be bleached
- the Illuminé material has been allowed to properly set, assuring that it will adhere to the tooth surface and not migrate, and
- care has been taken during placement in the mouth to assure that the material has not contacted the gingiva.

If, however, the instructions cannot be followed, or if edentulous or interdental gingival spaces such as diastemas are present, we strongly recommend that the gingival tissue in question be protected via a petrolatum jelly such as Vaseline®, an isolation resin (paint-on rubber dam) or a conventional rubber dam.

### Step-by-Step Instructions for Application of Illuminé office

(Numbers cited below refer to the respective illustrations.)

1. To connect both syringes together:
  - a.) Hold syringe A in an upright position (cap up, plunger down). As the syringe may be under slight pressure, draw back the plunger somewhat to release and carefully remove and discard the cap.
  - b.) Position syringe B (with the stopcock attached) over the "uncapped" syringe A.

---

<sup>1</sup> DENTSPLY DeTREY, Dreieich (D).

<sup>2</sup> Vita is a registered trademark of Vita Zahnfabrik H. Rauter GmbH & Co.

- c.) Securely fasten syringe A and B together by carefully threading the stopcock onto syringe A.
- d.) Do not over-tighten or strip the connection of these two parts.
2. Holding the coupled syringes horizontally, turn the stopcock to the ON position (parallel to the syringes).
3. Place thumbs securely on the ends of the plungers, while maintaining a horizontal syringe barrel position. Remove thumb from syringe B and apply thumb pressure to plunger A, quickly injecting the LIQUID contents of syringe A into the POWDER contents of syringe B. Reverse the process to inject the contents of syringe B into syringe A. Quickly mix the contents back and forth from syringe to syringe, no more than 3 to 6 times, until the powder is completely dissolved. DO NOT continue to mix once the powder is dissolved. Overmixing will cause the material to set up too quickly. IMPORTANT: Ensure that the mixed material is in syringe A, then gently push the mixed material up towards the tip to remove air in syringe A.
4. Turn the stopcock to the OFF position (perpendicular to the syringe barrel).
5. Disconnect syringe A in upright position from the stopcock and set syringe B with stopcock aside.
6. Place the applicator tip onto syringe A and twist to secure.
7. Working quickly, express the material into the reservoirs of the tray while holding the tray upright. Fill the centrals first, then work out to the laterals and canines, etc. Use the applicator tip to guide even placement of the material.
8. NOTE: The material will begin to turn from a liquid to a rubbery semi-solid approximately 30 to 60 seconds after mixing.  
Ensure that the material in the reservoirs is a semi-solid prior to seating the tray in the patient's mouth. The surface of the material will change from glossy to a dull finish when it becomes a semi-solid. If the material has not set, it could move onto the gingiva, causing irritation. DO NOT seat the tray if the material exhibits a glossy finish or appears to be liquid.
9. Seat the tray onto the patient's arch, gently pressing the reservoirs to ensure contact of the formulation with the tooth surface.
10. IMPORTANT: ENSURE that there is NO contact of the formulation with the gingival tissue. Use a scaler or explorer to gently remove any material that may be in contact with the gingiva.
11. The patient may now wait comfortably in the operatory or waiting room for 30 to 60 minutes.
12. Remove the tray(s) from the patient's mouth. Remove any remaining Illuminé material from the patient's teeth with gauze, using a gentle sweeping motion. Additional cleaning with a scaler or explorer or dry flossing may be required to remove material from the interproximal areas.

## **STORAGE**

Refrigerate (2 - 8 °C).

Avoid exposure to light.

## **BATCH NUMBER AND EXPIRY DATE**

The batch number should be quoted in all correspondence which requires identification of the product.

Do not use after expiry date.

If you have any questions, please contact:

**Manufacturer:**  
**DENTSPLY DeTREY GmbH**  
**De-Trey-Str. 1**  
**78467 Konstanz**  
**GERMANY**  
**Phone +49 (0) 75 31 5 83-0**

**Distributor:**  
**DENTSPLY Limited**  
**Building 1**  
**Aviator Park, Station Road**  
**Addlestone, KT15 2PG**  
**Phone (0 19 32) 85 34 22**

© DENTSPLY DeTREY 2001-05-31

\* **Appendix: Step by Step Instructions for Manufacture of Whitening Tray**

Generate Stone

Pour the disinfected, rinsed impression with dental stone.

Trim the stone cast so that the base is parallel to the occlusal plane of the posterior teeth and that the base extends 2 to 4 mm past the gingival border.

Creating the Reservoirs

1. Once the stone model has dried to an effective hardness apply an acrylic spacer (e. g. Triad<sup>1</sup> Gel) to the teeth on the model, thus creating reservoirs in the gel tray.
2. Apply the spacer only to teeth of aesthetic value.
3. Stay approximately 1 mm from the gingival margin when applying the spacer.
4. Do not cover any interproximal areas.
5. Do not apply spacer to occlusal surfaces or incisal edges.
6. Apply a spacer to the facial surfaces of the teeth to be whitened. For treatment with Illuminé office alone or in combination with short-time treatment with Illuminé home, the thickness of the spacer is 3 to 4 mm. Wait until spacer has set or light-cure spacer, respectively.
7. Wipe the cured spacer surfaces with a gauze dampened with alcohol to remove the slightly tacky surface that remains on the cured acrylic spacer.

Forming the Tray

1. Use a medium thick heat/vacuum sheet for tray fabrication.
2. Place the vacuum sheet (pebble finish towards the stone model) on a heat/vacuum tray-forming machine.
3. Soften the vacuum sheet until it sags 2 to 5 cm.
4. Engage the vacuum and slowly lower the heated gel tray onto the stone model to avoid generating wrinkles and folds.
5. Allow the sheet ample time under vacuum for good retention and definition.
6. Allow the sheet to cool prior to trimming.

Trimming the Tray

1. Remove the gel tray from the stone model.
2. Using scissors to trim the tray approximately 2 to 3 mm beyond the teeth.
3. Trim and refine the remaining portion with smaller precision scissors. (Scallop around the interdental areas).
4. Trim the tray approximately 1/4 to 1/2 mm short of the gingival margin.
5. Place the trimmed tray back onto the stone model to verify extensions.
6. Lightly flame polish tray edges.
7. Wash and disinfect the gel tray. Dry thoroughly.
8. Place in storage case and deliver to patient.

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