QuiXfil®

Posterior Restorative

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CAUTION: For dental use only.

USA: Rx only

Content	Page
1 Product description	2
2 Safety notes	3
3 Step-by-step instructions	5
4 Hygiene	7
5 Lot number and expiration date	7

1 Product description

QuiXfil® restorative is a resin-based restorative material specially designed for posterior use.

QuiXfil® restorative allows a simplified and fast application technique and is placed in layers up to 4 mm without compromising safety and efficacy.

QuiXfil® restorative utilizes a new filler technology resulting in a unique high filler load.

QuiXfil® restorative exhibits wear resistance and physical properties suitable for occlusal stress-bearing restorations.

QuiXfil® restorative is available in pre-dosed in Compules® Tips for direct intraoral application.

QuiXfil® restorative is available in one universal shade.

QuiXfil® restorative is used following application of Xeno® self etching adhesives or etch and rinse adhesives designed for use with QuiXfil® restorative.

1.1 Delivery forms

- Predosed Compules® Tips

1.2 Composition

- Urethane dimethacrylate (UDMA)
 - Triethyleneglycol dimethacrylate (TEGDMA)
 - Di- and trimethacrylate resins
 - Carboxylic acid modified dimethacrylate resin
 - Butylated hydroxy toluene (BHT)
 - UV stabilizer
 - Camphorquinone
 - Ethyl-4(dimethylamino)benzoate
 - Silanated-strontium-aluminum-sodium-fluoride-phosphate-silicate-glass
- The filler: matrix ratio is 85.5: 14.5 by weight and 66.4:33.6 by volume, respectively.

1.3 Indications

- Direct restorations of Class I and II cavities when esthetic considerations are not given priority.

1.4 Contraindications

- Use with patients who have a history of severe allergic reaction to methacrylate resins.
- Use as Core build-up for full ceramic crowns.

1.5 Compatible adhesives

The material is to be used in combination with a (meth)acrylate-based dentin/enamel adhesive such as the Dentsply Sirona Prime&Bond® and Xeno® range of adhesives.

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

The material contains methacrylates and polymerizable monomers which may be irritating to skin, eyes and oral mucosa and may cause sensitization by skin contact and allergic contact dermatitis in susceptible persons.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitisation or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- If refrigerated, allow material to reach room temperature prior to use.
- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- The Compules® Tips cannot be reprocessed. To prevent Compules® Tips when used for indirect application from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the devices are handled offside the dental unit with clean/disinfected gloves. Discard devices if contaminated.
- Compules® Tips may be used for direct intraoral application of restorative material into a cavity or for indirect application by first placing the restorative material on a pad. After intraoral use, discard Compules® Tips and do not reuse in other patients in order to prevent cross-contamination.
- Use of Compules® Tips with the Compules® Tips Gun is recommended.
- The Compules® Tips Gun is only intended to be used with Dentsply Sirona Compules® or Compula® Tips.
- For further information please refer to the Compules® Tips Gun Instructions for Use.
- The material should extrude easily. DO NOT USE EXCESSIVE FORCE. Excessive pressure may cause the Compules® Tip to rupture or to eject from the Compules® Tips Gun.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.
- Interactions:
 - Do not use eugenol- or hydrogen peroxide-containing materials in conjunction with this product since they may interfere with hardening of the product.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

2.4 Storage conditions

Inadequate storage conditions will shorten the shelf life and may lead to malfunction of the product.

- Store at temperatures between 10 °C and 24 °C (50 °F and 75 °F).
- Keep out of direct sunlight and protect from moisture.
- Do not freeze.
- Do not use after expiration date.

Humidity can adversely affect the properties of the material. For unsealed Compules® Tips, the following is recommended:

- Store in a dry environment (< 80 % relative humidity).
- Use within 4 weeks.

3 Step-by-step instructions

3.1 Shade consideration

QuiXfil® restorative is available in one universal, translucent shade to achieve a large curing-depth. When esthetic considerations are important use alternative restoratives (e.g. ceram.x®, Esthet•X® HD, Spectrum® TPH®3).

3.2 Cavity preparation

1. Clean the tooth surface with a fluoride free prophylaxis paste (e.g. Nupro® prophylaxis paste).
2. Prepare cavity (unless not necessary, e.g. cervical lesion).
3. Use adequate isolation such as rubber dam.
4. Rinse surface with water spray and carefully remove rinsing water. Do not desiccate the tooth structure.

3.2.1 Matrix placement and wedging (Class II restorations)

1. Place a matrix (e.g. AutoMatrix® matrix system or Palodent® matrix system) and wedge. Burnishing of the matrix band will improve contact and contour. Pre-wedging or BiTine® ring placement is recommended.

3.3 Pulp protection, tooth conditioning/dentin pre-treatment, adhesive application

Refer to adhesive manufacturer's Instructions for Use for pulp protection, tooth conditioning and/or adhesive application. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately to placement of the material.

3.4 Application

Apply material in increments immediately after the application of the adhesive. Due to the high depth of cure and low shrinkage for QuiXfil® restorative, placement in layers up to 4 mm thickness is possible.

Using Compules® Tips

1. Insert Compules® Tip into the notched opening of the Compules® Tips Gun. Be certain that the collar on the Compules® Tip is inserted first.
2. Remove the colored cap from the Compules® Tip. The Compules® Tip may be rotated 360° to gain the proper angle of entrance into the cavity or to the mixing pad.
DO NOT USE EXCESSIVE FORCE.
Indirect use: to prevent Compules® Tip from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the Compules® Tip is handled off-side the dental unit with clean/disinfected gloves. Dispense the necessary amount of material onto a mixing pad and protect against light. Apply material into the cavity with a plastic instrument.
- or -
Direct intraoral use: dispense the material directly into the cavity preparation using a slow, steady pressure.
3. To remove the used Compules® Tip, be sure that the Compules® Tips Gun plunger is pulled back completely by allowing the handle to open to its widest position. Apply a downward motion to the front end of the Compules® Tip and remove.

Excessive force.

Injury.

1. Apply slow and steady pressure on the Compules® Tips Gun.
2. Do not use excessive force. Compules® Tip rupture or ejection from Compules® Tips Gun may result.

3.4.1 Applying restorative

1. Fill the whole cavity incrementally. Each layer ≤ 4 mm.
2. Cure each layer according to the table below [3.5 Light curing].

3.5 Light curing

1. Light cure each increment with a suitable curing light.¹ Cure each increment for **10 seconds** when using a curing light as Dentsply Sirona SmartLite® PS or SmartLite® Focus® that has a minimum irradiance of 800 mW/cm².

Cure composite for **20 seconds** when minimum irradiance is between 500 mW/cm² and 800 mW/cm².

Refer to curing light manufacturer's Instructions for Use for compatibility and curing recommendations.

Curing time for 4 mm increments

 mW/cm ²	 sec	 mm
≥ 500	20	4
≥ 800	10	

Curing time table.

The curing time table above also appears on all outer packages.

Insufficient curing.

Inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing cycle.
3. Check minimum irradiance.
4. Cure each area of each increment for the recommended curing time.

3.6 Finishing and polishing

1. Contour the restoration using finishing burs or diamonds.
2. Use Enhance® finishing devices for additional finishing.

For contouring, finishing, and/or polishing, follow the manufacturer's Instructions for Use.

¹ Curing light designed to cure materials containing camphorquinone (CQ) initiator. Peak of spectrum in the range of 440-480 nm.

4 Hygiene

4.1 Compules® Tips Gun

For reprocessing instructions please refer to the Instructions for Use of the Compules® Tips Gun, which is available on our webpage at www.dentsply.eu/IFU. If requested, we will send you a free printed copy of the Instructions for Use in the language you require within 7 days. You can use the order form provided on our webpage for this purpose.

4.2 Compules® Tips – cross-contamination

Cross-contamination.

Infection.

1. Do not reuse Compules® Tips after intraoral use and/or if contaminated.
2. Dispose the intraorally used and/or contaminated Compules® Tips in accordance with local regulations.

5 Lot number () and expiration date ()

1. Do not use after expiration date.
ISO standard is used: “YYYY-MM” or “YYYY-MM-DD”.
2. The following numbers should be quoted in all correspondence:
 - Reorder number
 - Lot number
 - Expiration date

© Dentsply Sirona 2016-10-05

[These Instructions for Use are based on Master Version 03]

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Dentsply DeTrey GmbH