Sani-Tip®

D-1 Conversion Kit

Part No. 122240N

For Den-Tal-Ez/Star Air/Water Syringes

Kit de conversion D-1

Pièce N°. 122240N

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**Caution: United States Federal law restricts this device for sale by,
or on the order of, a licensed dental professional.**

INDICATIONS FOR USE

Used to transmit air or water to the operatory field during operative procedures.

CONTRAINDICATIONS


This Conversion Kit is not indicated for use with any other air/water syringes.

WARNINGS

- Sani-Tip® Conversion Kits are designed to function exclusively with Sani-Tip disposable syringe tips. Use only adapter components and clear Sani-Tip plastic syringe tips manufactured by DENTSPLY Professional for the Sani-Tip® System to ensure optimal performance and safety including reducing potential for cross-contamination. Follow the enclosed instructions.
- Sani-Tip® disposable syringe tips are for single use only. Dispose after use.
- It is the responsibility of the Dental Healthcare Professionals (DHCP) to determine the appropriate uses of this product and to understand the health of each patient, the dental procedures being undertaken, and industry and governmental agency recommendations, requirements, and regulations for safe practice of dentistry.
- To ensure adequate protection from cross contamination from other devices connected to the Dental Unit, it is highly recommended that the Dental Unit be installed with anti-retraction features. Additionally, the Dental Unit, including the anti-retraction features, must be properly maintained and periodically tested. For more information, please contact your Dental Unit manufacturer.
- Where asepsis is required or deemed appropriate in the best professional judgement of the DHCP, this product should not be used.


PRECAUTIONS

- Although this conversion kit may be used with all Sani-Tip disposable syringe tips, it is recommended for use with the Standard (76mm) Sani-Tip disposable syringe tips.
- To help reduce cross contamination, use a new Sani-Shield disposable air/water syringe sleeve for every patient.

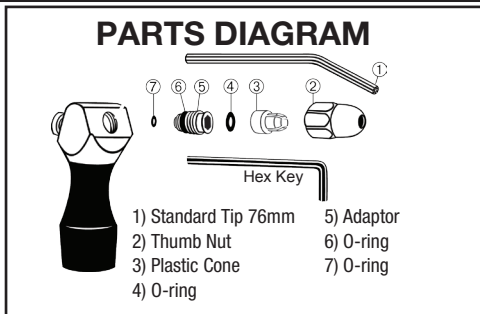D-1 CONVERSION KIT FOR AIR/WATER SYRINGES

ADAPTER INSTALLATION INSTRUCTIONS


* Important: If Den-Tal-Ez unit is manufactured after January 1997, syringe will require an A-1 Conversion Kit.

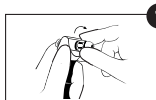
- 1** 1. Remove existing thumb nut and syringe tip.

- 4** 4. Verify o-ring **4** is in place inside large opening of white plastic cone **3**.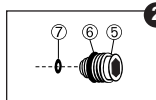


- 2** 2. Verify new adapter **5** has o-rings **6** and **7** in position as shown.

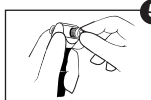- 5** 5. With o-ring in place, firmly snap large end of white plastic cone over adapter rim.


- 3** 3. Hand-thread adapter into syringe. Use hex key provided to tighten.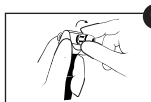


- 6** 6. Hand-thread thumb nut onto adapter. Do not tighten until new tip has been inserted.

Installation is complete.

USER INSTRUCTIONS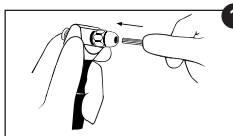

- 1** 1. With thumb nut loosened, insert new Sani-Tip disposable tip and push into adapter as far as possible. While still pushing inward, twist tip 1/2 turn clockwise to create a watertight seal over internal tapered water spindle.

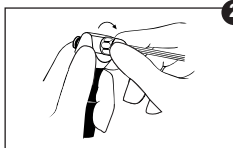- 2** 2. Hand tighten thumb nut. To redirect tip, push in on tip while rotating clockwise.

TEST PRIOR TO EACH USE: Express water, then express air only. If air contains moisture, or tip dislodges, refer to Troubleshooting section.

3. To replace tip, loosen thumb nut. Remove and discard used tip. Repeat steps 1 and 2.

INFECTION CONTROL PROCEDURES

WARNINGS

- Conversion Kits are not sterile upon receipt and must be sterilized prior to use in accordance with the following instructions.

LIMITATIONS ON PROCESSING

- Do not use chemical disinfectants prior to sterilization or rapid deterioration of the material may occur.
- Cold liquid disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use. Use may void warranty.

INSTRUCTIONS

Point of use:

Remove excess soil with disposable cloth or paper.

Containment and Transportation:

Protect conversion kit components from damage. Components should be reprocessed as soon as is reasonably practical following use. Components left standing wet may stain or corrode.

Preparation for decontamination:

Remove excess debris from the components using a disposable cloth or paper. Discard cloth/paper after use. Disassemble the conversion kit from the air/water syringe. Remove the plastic cone and adapter o-ring.

Cleaning: automated

None.

Cleaning: manual

1. Clean all external surfaces of the Sani-Tip® conversion kit, including the air/water syringe with a mild detergent and water solution. Dry with a lint-free cloth.
2. Remove the conversion kit from the air/water syringe.
3. Clean all internal surfaces of the adapter, thumb nut, plastic cone and o-ring with a mild detergent and water solution. Dry with a lint-free cloth.
4. Further process the adapter and thumb nut following the Sterilization procedure listed.
5. The plastic cone and o-ring are not steam autoclave sterilizable. Further process following the disinfection procedures listed.

Disinfection:

Disinfect the plastic cone and o-ring using standard disinfection solutions.

Packaging:

Place thumb nut and adapter in a paper or paper/plastic steam sterilization pouch. Components may be loaded into dedicated instruments tray.

Sterilization:

Use a steam autoclave. Place bagged adapter and thumb nut into the steam autoclave, paper side up

when using a paper/plastic pouch. After warm-up is completed, operate at a sterilizing temperature and pressure of 134°C/273°F to 137°C/279°F and pressure of 216kPa/31psi for 3 to 12 minutes.

Alternate method: Place non-bagged adapter and thumb nut into the steam autoclave. After warm-up is completed, operate at a sterilizing temperature of 134°C/273°F to 137°C/279°F and pressure of 216kPa /31psi for 3 to 4 minutes. Components should be used immediately.

Drying:

To dry, use the drying cycle of the autoclave. Set cycle for 20-30 minutes.

Maintenance, inspection and testing:

- Visually inspect to ensure all contamination is removed.
- Check for distortion, damage and wear. Discard damaged, worn or corroded components.

Storage:

To maintain sterility, instruments should remain bagged until ready to use.

MANUFACTURER CONTACT

In the United States, contact DENTSPLY Professional Customer Service Technical Support at 800-989-8826. For areas outside the United States, contact your local DENTSPLY Representative.

TROUBLESHOOTING

PROBLEM: Water leaking, wet air, and/or loose tips, indicates either:

- a.) Center hole of Sani-Tip disposable tip is not fully seated over water spindle inside new adapter.
- b.) Correct Sani-Tip Conversion Kit is not properly installed, and/or missing parts; thumb nut has not been fully tightened, or incorrect tips are being used.
- c.) Internal water spindle is damaged.

SOLUTION:

- a.) Loosen thumb nut, when interchanging Sani-Tip disposable tips. Insert new tip and push into adapter as far as possible. While still pushing inward, twist tip 1/2 turn clockwise to create a water-tight seal over internal tapered water spindle. Tighten thumb nut.
- b.) Make sure correct Sani-Tip Conversion Kit is properly installed. Confirm presence of plastic cone and all o-rings. Fully tighten thumb nut after inserting new tip. Verify tips are clear plastic and manufactured by DENTSPLY Professional for the Sani-Tip® System. **Important:** If Den-Tal-Ez unit is manufactured after 1/97, syringe will require A-1 Conversion Kit.
- c.) If internal water spindle inside Sani-Tip adapter is damaged, replace adapter.

PROBLEM: Tip does not fit through thumb nut.

SOLUTION: Loosen thumb nut a full turn to expand opening of plastic cone.

PROBLEM: Little or no water.

SOLUTION: Check for clogged or damaged water spindle inside adapter, or blocked water outlet in syringe head.

PROBLEM: Too much/little air or water pressure.

SOLUTION: Minor adjustments to air and water pressure can be made at the dental unit. Follow manufacturer's specifications.

PROBLEM: Little or no water due to off-center water outlet in syringe head.

SOLUTION: Occasional manufacturing variances result in an off-center water outlet on Den-Tal-Ez syringes. If this occurs, Sani-Tip® adaptor will not function properly.

TEST PRIOR TO USE:

Express water, then express air only. If air contains moisture, or tip dislodges, refer to first problem and solution.

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The enclosed adapter is covered, in whole or in combination with other parts of the SANI-TIP® System, by one or more of the following patents: U.S. 5,306,146; U.S. 5,489,205; AU 714134; CA 2,155,313; EP 0686017.

You do not have a license, express or implied, under any of these patents to make or sell the adapter. You have an express license only under Patent Numbers U.S. 5,306,146; and U.S. 5,489,205; and no implied license under any other of these patents, to use the adapter in combination with the enclosed syringe tip(s) furnished by DENTSPLY Professional.

If you use the adapter with syringe tips not furnished by DENTSPLY Professional, you are in violation of this license granted to you and you may be liable for willful patent infringement. If you use the adapter with syringe tips not furnished by DENTSPLY Professional, you do so entirely at your own risk and DENTSPLY Professional denies any responsibility for such use, including product liability and any express or implied warranties.

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EC|REP DENTSPLY DeTrey GmbH