# Sani-Tip®

## K-3 Conversion Kit

Part No. 122275N

For Kavo 573 0620 and 573 0630 (new versions) Systematica Air/Water Syringe without optics

## Kit de conversion K-3

Pièce N°. 122275N

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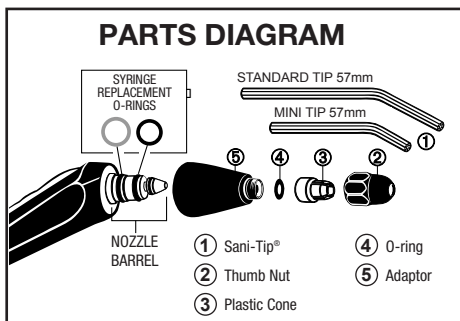
**Rx Only.**

## Indications for Use

Used to transmit air or water to the operatory field during operative procedures.

## Contraindications

This Conversion Kit is not indicated for use with any other air/water syringes.

## Warnings

- This conversion kit is only to be used for the latest version Kavo Systematica Syringe models 573 0620 and 573 0630. Only the latest version provides adequate specifications to safely convert to use the Sani-Tip system. Refer to the **IDENTIFY SYRINGE VERSION** section before continuing to determine if your Kavo air/water syringe can be converted at this time.
- Sani-Tip® Conversion Kits are designed to function exclusively with Sani-Tip® disposable syringe tips. To minimize physical harm to patient, or equipment damage and/or malfunction, use only adapter components and clear Sani-Tip plastic syringe tips manufactured by DENTSPLY Professional for the Sani-Tip® System. Follow the enclosed instructions.
- Sani-Tip® disposable syringe tips are for single use only. Dispose after use.
- It is the responsibility of the Dental Healthcare Professionals (DHCP) to determine the appropriate uses of this product and to understand the health of each patient, the dental procedures being undertaken, and industry and governmental agency recommendations, requirements, and regulations for safe practice of dentistry.
- To ensure adequate protection from cross-contamination from other devices connected to the Dental Unit, it is highly recommended that the Dental Unit be installed with anti retraction features. Additionally, the Dental Unit, including the anti-retraction features, must be properly maintained and periodically tested. For more information, please contact your Dental Unit manufacturer.
- Where asepsis is required or deemed appropriate in the best professional judgment of the DHCP, this product should not be used.

## Precautions


- Although this conversion kit may be used with all Sani-Tip disposable syringe tips, it is recommended for use with the Mini Sani-Tip disposable syringe tips.
- To help reduce cross contamination, use a new Sani-Shield disposable air/water syringe sleeve, in addition to a new Sani-Tip disposable tip, for every patient.

## IDENTIFY SYRINGE VERSION

### Compare O-Ring Grooves and O-Rings:


Both Systematica versions feature a 23mm nozzle barrel, however, only one offers o-ring grooves deep and wide enough for the heavy o-rings needed to retain the adapter under pressure. The actual-size diagrams and photos show a comparison of both versions. If your syringe nozzle barrel and corresponding o-rings match the diagram at top, continue with installation instructions. If your syringe matches the diagram at bottom, contact Kavo to upgrade before proceeding.

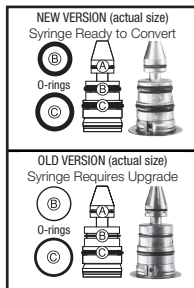KAVO 573-0620/30 SYSTEMATICA  
NOZZLE BARREL COMPARISON

## ADAPTER INSTALLATION INSTRUCTIONS

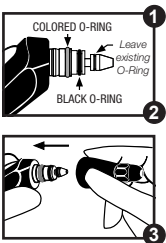
Follow syringe identification procedure first.

1. Remove existing syringe tip.
2. Replace existing large o-rings with spare o rings provided. Install colored o-ring at back of nozzle barrel and black o-ring at center as shown. Leave existing small o-ring in place.
3. Firmly push the Sani-Tip® adapter and thumb nut assembly as far as possible onto syringe. Adapter must be flush with syringe face to make the proper connection. Failure to fully mount adapter will result in leakage or cause adapter to disengage from syringe.

*Installation is complete.*

## USER INSTRUCTIONS

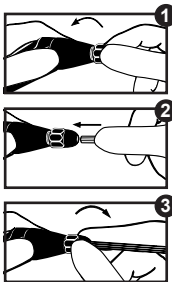1. Hold adapter body securely to prevent rotation. With other hand, loosen thumb nut.
2. With thumb nut loosened, insert a new Sani-Tip disposable tip and push into adapter as far as possible. While still pushing inward, twist tip 1/2 turn clockwise to create a watertight seal over internal tapered water spindle.
3. Continue to hold adapter body and hand-tighten thumb nut. To redirect tip, push in on tip while rotating clockwise.
4. Hold the syringe with tip vertically and spray water into an inverted cup for 30 seconds. Express air and water several times. If air still contains water, repeat steps. If condition persists, refer to Troubleshooting Section.
5. To replace tip, loosen thumb nut. Remove and discard used tip. Repeat steps 2, 3 and 4.

## INFECTION CONTROL PROCEDURES

### WARNINGS

- Conversion Kits are not sterile upon receipt and must be sterilized prior to use in accordance with the following instructions.

### LIMITATIONS ON PROCESSING

- Do not use chemical disinfectants prior to sterilization or rapid deterioration of the material may occur.
- Cold liquid disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use. Use may void warranty.

## INSTRUCTIONS

**Point of use:**

Remove excess soil with disposable cloth or paper.

**Containment and Transportation:**

Protect conversion kit components from damage. Components should be reprocessed as soon as is reasonably practical following use. Components left standing wet may stain or corrode.

**Preparation for decontamination:**

Remove excess debris from the components using a disposable cloth or paper. Discard cloth/paper after use. Disassemble the conversion kit from the air/water syringe. Remove the plastic cone and adapter o-ring.

**Cleaning: automated**

None

**Cleaning: manual**

1. Clean all external surfaces of the Sani-Tip® conversions kit, including the air/water syringe with a mild detergent and water solution. Dry with a lint free cloth.
2. Remove the conversion kit from the air/water syringe.
3. Clean all internal surfaces of the adapter, thumb nut, plastic cone and o-ring with a mild detergent and water solution. Dry with lint-free cloth.
4. Further process the adapter and thumb nut following the Sterilization procedure listed.
5. The plastic cone and o-ring are not steam autoclave sterilizable. Further process following the disinfection procedures listed.

**Disinfection:**

Disinfect the plastic cone and o-ring using standard disinfection solutions.

**Packaging:**

Place thumb nut and adapter in a paper or paper/plastic steam sterilization pouch. Components may be loaded into dedicated instrument trays.

**Sterilization:**

Use a steam autoclave. Place bagged adapter and thumb nut into the steam autoclave, paper side up when using a paper/plastic pouch. After warm-up is completed, operate at a sterilizing temperature and pressure of 134°C/ 273°F to 137°C/279°F and pressure of 216kPa /31psi for 3 to 12 minutes.

*Alternate method: Place non-bagged adapter and thumb nut into the steam autoclave. After warm-up is completed, operate at a sterilizing temperature and pressure of 134°C/273°F to 137°C/279°F and pressure of 216kPa/31psi for 3 to 4 minutes. Components should be used immediately.*

**Drying:**

To dry, use the drying cycle of the autoclave. Set cycle for 20-30 minutes.

**Maintenance, inspection and testing:**

- Visually inspect to ensure all contamination is removed.
- Check for distortion, damage and wear. Discard damaged, worn or corroded components.

**Storage:**

To maintain sterility, instruments should remain bagged until ready for use.

**MANUFACTURER CONTACT:**

In the United States, contact DENTSPLY Professional Customer Service Technical Support at 800-989-8826. For areas outside the United States, contact your local DENTSPLY Professional Representative.

**TROUBLESHOOTING**

**PROBLEM:** Water leaking, wet air, loose tip and/or adapter. Indicates either:

- Center hole of Sani-Tip tip is not fully seated over water spindle inside new adapter, or
- Correct Sani-Tip Conversion Kit is not properly installed, and/or is missing parts; thumb nut has not been fully tightened; existing syringe o-rings have not been replaced; incorrect tips are being used, or
- Adapter is not fully mounted onto syringe.
- Internal water spindle is damaged.
- Kavo syringe barrel nozzle must be upgraded.

**SOLUTION:**

- Hold adapter body to prevent rotation and loosen thumb nut when interchanging the Sani-Tip disposable tip. Insert a new tip and push into the adapter as far as possible. While still pushing inward, twist top 1/2 turn clockwise to create a watertight seal over internal tapered water spindle. Tighten thumb nut.
- Make sure the correct Sani-Tip Conversion Kit is properly installed. Confirm presence of plastic cone and o-ring. Hold adapter body to prevent rotation and fully tighten thumb nut after inserting new tip. Replace existing o-rings on syringe nozzle barrel with replacement o-rings provided, as described in the Installation Instructions. Verify tips are clear plastic Sani-Tip disposable tips manufactured by DENTSPLY Professional for the Sani-Tip® System.
- Firmly push adapter as far as possible onto syringe. Adapter rim must be flush with syringe face to make proper connection.
- If internal water spindle inside Sani-Tip adapter is damaged, replace adapter.
- The old version of Kavo syringe model 537 0620/0630 features a nozzle barrel with shallow o-ring grooves and excessively thin o-rings which cannot retain a Sani-Tip adapter under pressure. Contact Kavo to upgrade to the new version of syringe barrel nozzle before converting.

**PROBLEM:** Tip does not fit through thumb nut.

**SOLUTION:** Loosen thumb nut a full turn to expand opening of plastic cone.

**PROBLEM:** Little or no water.

**SOLUTION:** Check for clogged or damaged water spindle inside adapter, or blocked water outlet in syringe head.

**PROBLEM:** Too much/little air or water pressure.

**SOLUTION:** Minor adjustments can be made at the dental unit. Follow manufacturer's specifications.

**ALWAYS TEST PRIOR TO USE:**

*Express water, then express air only. If air contains moisture, or tip dislodges, refer to first problem and solution.*

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## IMPORTANT LICENSING NOTICE

The enclosed adapter is covered, in whole or in combination with other parts of the SANI-TIP® System, by one or more of the following patents: U.S. 5,049,071; U.S. 5,192,206; U.S. 5,236,356; U.S. 5,306,146; U.S. 5,489,205; Aus. 630108; Aus. 650567; CA 1,334,134; E.P. 0358272 and other U.S. and foreign patents pending.

The enclosed syringe tip(s) are covered, in whole or in combination with other parts of the SANI-TIP® System, by one or more of the following patents: U.S. 5,049,071; U.S. 5,192,206; U.S. 5,236,356; U.S. 5,342,195; Aus. 630108; Aus. 650567; CA 1,334,134; E.P. 0358272 and other U.S. and foreign patents pending.

You do not have a license, express or implied, under any of these patents to make or sell the adapter. You have an express license only under Patent Numbers U.S. 5,236,356; U.S. 5,306,146; U.S. 5,489,205; CA 1,334,134 and no implied license under any other of these patents, to use the adapter in combination with the enclosed syringe tip(s) furnished by DENTSPLY Professional.

If you use the adapter with syringe tips not furnished by DENTSPLY Professional, you are in violation of this license granted to you and you may be liable for willful patent infringement. If you use the adapter with syringe tips not furnished by DENTSPLY Professional, you do so entirely at your own risk and DENTSPLY Professional denies any responsibility for such use, including product liability and any express or implied warranties.

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**EC REP** DENTSPLY DeTrey GmbH