# Sani-Tip®

## L-3 Conversion Kit

Part No. 122326N  
For Luzzani  
MiniMate Air/Water Syringes

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## L-2 Conversion Kit

Part No. 122325N  
For Luzzani Minimatic Air/Water  
Syringes

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**Caution: United States Federal law restricts this device for sale by,  
or on the order of, a licensed dental professional.**

**INDICATIONS FOR USE**

Used to transmit air or water to the operatory field during operative procedures.

**CONTRAINDICATIONS**


This Conversion Kit is not indicated for use with any other air/water syringes.

**WARNINGS**

- Sani-Tip® Conversion Kits are designed to function exclusively with Sani-Tip® disposable syringe tips. To minimize physical harm to patient, or equipment damage and/or malfunction, use only adapter components and clear Sani-Tip plastic syringe tips manufactured by DENTSPLY Professional for the Sani-Tip® System. Follow the enclosed instructions.
- Sani-Tip® disposable syringe tips are for single use only. Dispose after use.
- It is the responsibility of the Dental Healthcare Professionals (DHCP) to determine the appropriate uses of this product and to understand the health of each patient, the dental procedures being undertaken, and industry and governmental agency recommendations, requirements, and regulations for safe practice of dentistry.
- To ensure adequate protection from cross contamination from other devices connected to the Dental Unit, it is highly recommended that the Dental Unit be installed with anti-retraction features. Additionally, the Dental Unit, including the anti-retraction features, must be properly maintained and periodically tested. For more information, please contact your Dental Unit manufacturer.
- Where asepsis is required or deemed appropriate in the best professional judgement of the DHCP, this product should not be used.

**PRECAUTIONS**


- Although this conversion kit may be used with all Sani-Tip disposable syringe tips, it is recommended for use with the mini (57 mm) Sani-Tip disposable syringe tips.
- To help reduce cross contamination, use a new Sani-Shield disposable air/water syringe sleeve, in addition to a new Sani-Tip disposable tip, for every patient.

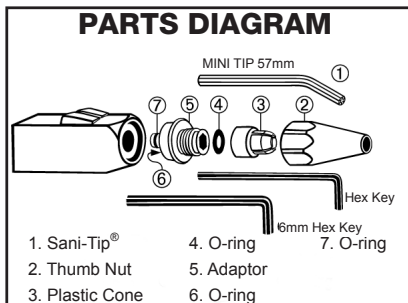**L-3 CONVERSION KIT FOR MINILIGHT & MINIMATE AIR/WATER SYRINGES**


## L-3 CONVERSION KIT ADAPTER INSTALLATION INSTRUCTIONS

1. Remove all existing nozzle parts. Nozzle receptacle must be free of all obstructions and/or loose parts before proceeding. Use 6mm hex key provided to remove existing adapter.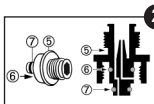


2. Verify new adapter (3) has o-rings (6) and (7) in position as shown.

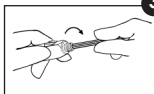
3. Hand-thread adapter into syringe. Use hex key provided to tighten. **WARNING:** Do not over-tighten! Damage to syringe will result.

4. Verify o-ring (4) is in place inside large opening of white plastic cone (3).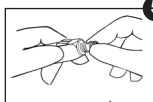


5. With o-ring in place, firmly snap large end of white plastic cone over adapter rim.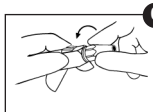


6. Hand-thread thumb nut onto adapter. Do not tighten until new tip has been inserted.

**Installation is complete.**

## L-3 CONVERSION KIT USER INSTRUCTIONS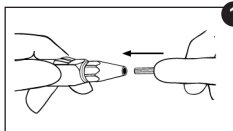

1. With thumb nut loosened, insert new Sani-Tip® disposable tip and push into adapter as far as possible. While still pushing inward, twist tip 1/2 turn clockwise to create a watertight seal over internal tapered water spindle.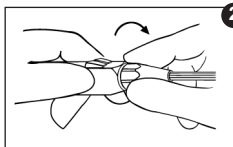


2. Hand tighten thumb nut. To redirect tip, push in on tip while rotating clockwise.

**TEST PRIOR TO EACH USE:** Express water, then express air only. If air contains moisture, or tip dislodges, refer to Troubleshooting section.


3. To replace tip, loosen thumb nut. Remove and discard used tip. Repeat steps 1 and 2.

## L-2 CONVERSION KIT FOR MINIMATIC AIR/WATER SYRINGES


## PARTS DIAGRAM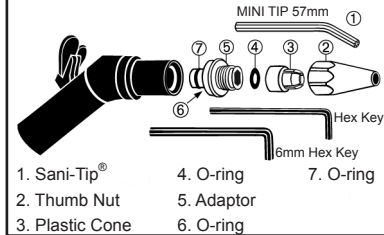


## L-2 CONVERSION KIT ADAPTER INSTALLATION INSTRUCTIONS

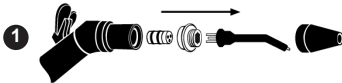
1. Remove all existing nozzle parts. Nozzle receptacle must be free of all obstructions and/or loose parts before proceeding. Use 6mm hex key provided to remove existing adaptor.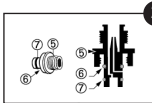


2. Verify new adapter ⑤ has o-rings ⑥ and ⑦ in position as shown.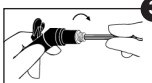


3. Hand-thread adapter into syringe. Use hex key provided to tighten.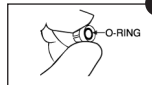

4. Verify o-ring ④ is in place inside large opening of white plastic cone ③.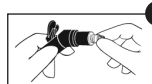


5. With o-ring in place, firmly snap large end of white plastic cone over adapter rim.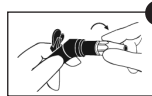


6. Hand-thread thumb nut onto adapter. Do not tighten until new tip has been inserted.

**Installation is complete.**

## L-2 CONVERSION KIT USER INSTRUCTIONS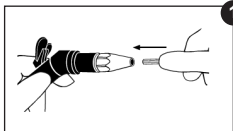

1. With thumb nut loosened, insert new Sani-Tip® disposable tip and push into adapter as far as possible. While still pushing inward, twist tip 1/2 turn clockwise to create a watertight seal over internal tapered water spindle.

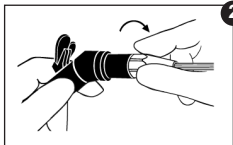2. Hand tighten thumb nut. To redirect tip, push in on tip while rotating clockwise.

**TEST PRIOR TO EACH USE:** Express water, then express air only. If air contains moisture, or tip dislodges, refer to Troubleshooting section.

3. To replace tip, loosen thumb nut. Remove and discard used tip. Repeat steps 1 and 2.

## INFECTION CONTROL PROCEDURES

### WARNINGS

- Conversion Kits are not sterile upon receipt and must be sterilized prior to use in accordance with the following instructions.

### LIMITATIONS ON PROCESSING

- Do not use chemical disinfectants prior to sterilization or rapid deterioration of the material may occur.
- Cold liquid disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use. Use may void warranty.

### INSTRUCTIONS

#### Point of use:

Remove excess soil with disposable cloth or paper.

#### Containment and Transportation:

Protect conversion kit components from damage. Components should be reprocessed as soon as is reasonably practical following use. Components left standing wet may stain or corrode.

#### Preparation for decontamination:

Remove excess debris from the components using a disposable cloth or paper. Discard cloth/paper after use. Disassemble the conversion kit from the air/water syringe. Remove the plastic cone and adapter o-ring.

**Cleaning: automated**

None.

**Cleaning: manual**

1. Clean all external surfaces of the Sani-Tip® conversion kit, including the air/water syringe with a mild detergent and water solution. Dry with a lint-free cloth.
2. Remove the conversion kit from the air/water syringe.
3. Clean all internal surfaces of the adapter, thumb nut, plastic cone and o-ring with a mild detergent and water solution. Dry with a lint-free cloth.
4. Further process the adapter and thumb nut following the Sterilization procedure listed.
5. The plastic cone and o-ring are not steam autoclave sterilizable. Further process following the disinfection procedures listed.

**Disinfection:**

Disinfect the plastic cone and o-ring using standard disinfection solutions.

**Packaging:**

Place thumb nut and adapter in a paper or paper/plastic steam sterilization pouch. Components may be loaded into dedicated instruments tray.

**Sterilization:**

Use a steam autoclave. Place bagged adapter and thumb nut into the steam autoclave, paper side up when using a paper/plastic pouch. After warm-up is completed, operate at a sterilizing temperature and pressure of 134°C/273°F to 137°C/279°F and pressure of 216kPa/31psi for 3 to 12 minutes.

*Alternate method: Place non-bagged adapter and thumb nut into the steam autoclave. After warm-up is completed, operate at a sterilizing temperature and pressure of 134°C/273°F to 137°C/279°F and pressure of 216kPa /31psi for 3 to 4 minutes. Components should be used immediately.*

**Drying:**

To dry, use the drying cycle of the autoclave. Set cycle for 20-30 minutes.

**Maintenance, inspection and testing:**

- Visually inspect to ensure all contamination is removed.
- Check for distortion, damage and wear. Discard damaged, worn or corroded components.

**Storage:**

To maintain sterility, instruments should remain bagged until ready to use.

**MANUFACTURER CONTACT**

In the United States, contact DENTSPLY Professional Customer Service Technical Support at 800-989-8826. For areas outside the United States, contact your local DENTSPLY Representative.

**TROUBLESHOOTING**

**PROBLEM:** Water leaking, wet air, and/or loose tips, indicates either:

- a.) Center hole of Sani-Tip disposable tip is not fully seated over water spindle inside new adapter.
- b.) Correct Sani-Tip Conversion Kit is not properly installed, and/or missing parts; thumb nut has not been fully tightened, or incorrect tips are being used.
- c.) Internal water spindle is damaged.

**SOLUTION:**

- a.) Loosen thumb nut when interchanging Sani-Tip® disposable tips. Insert new Sani-Tip disposable tip and push into adapter as far as possible. While still pushing inward, twist tip 1/2 turn clockwise to create a watertight seal over internal tapered water spindle. Tighten thumb nut.
- b.) Make sure correct Sani-Tip® Conversion Kit is properly installed. Confirm presence of plastic cone and all o-rings. Fully tighten thumb nut after inserting new tip. Verify tips are clear plastic and manufactured by DENTSPLY Professional for the Sani-Tip® System.
- c.) If internal water spindle inside Sani-Tip® adapter is damaged, replace adapter.

**PROBLEM:** Difficulty installing new adapter, water leaking, and/or loose tips.

**SOLUTION:** Check that all existing parts have been removed from syringe nozzle receptacle prior to installing new adapter.

**PROBLEM:** Tip does not fit through thumb nut.

**SOLUTION:** Loosen thumb nut a full turn to expand opening of plastic cone.

**PROBLEM:** Little or no water.

**SOLUTION:** Check for clogged or damaged water spindle inside adapter, or blocked water outlet in syringe head, or existing parts still inside nozzle receptacle.

**PROBLEM:** Too much/little air or water pressure.

**SOLUTION:** Minor adjustments to air and water pressure can be made at the dental unit. Follow manufacturer's specifications.

**ALWAYS TEST PRIOR TO USE:**

*Express water, then express air only. If air contains moisture, or tip dislodges, refer to first problem and solution.*

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The enclosed adapter is covered, in whole or in combination with other parts of the SANI-TIP® System, by one or more of the following patents: U.S. 5,306,146; U.S. 5,489,205; AU 714134; CA 2,155,313; EP 0686017.

You do not have a license, express or implied, under any of these patents to make or sell the adapter. You have an express license only under Patent Numbers U.S. 5,306,146; and U.S. 5,489,205; and no implied license under any other of these patents, to use the adapter in combination with the enclosed syringe tip(s) furnished by DENTSPLY Professional.

If you use the adapter with syringe tips not furnished by DENTSPLY Professional, you are in violation of this license granted to you and you may be liable for willful patent infringement. If you use the adapter with syringe tips not furnished by DENTSPLY Professional, you do so entirely at your own risk and DENTSPLY Professional denies any responsibility for such use, including product liability and any express or implied warranties.

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**EC REP** DENTSPLY DeTrey GmbH