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# Sani-Tip® Disposable Air/Water Syringe Tip

## INDICATIONS FOR USE

The SANI-SHIELD device is intended to be used as a barrier product to reduce gross contamination for use with air/water syringes.

## CONTRAINDICTIONS

Not for use in sterile procedure environments.

## WARNINGS

- Sani-Tip® syringe tips are designed to function exclusively with air/water syringes converted using a Sani-Tip® Conversion Kit. If your syringe has not been converted for Sani-Tip® disposable tip use, the appropriate Sani-Tip® Conversion Kit **MUST** be installed before proceeding. Contact your dealer for a complete list of available conversion kits.
- Use only adapter components and clear Sani-Tip plastic syringe tips manufactured by DENTSPLY Professional for the Sani-Tip® System to ensure optimal performance and safety including reducing potential for cross-contamination. Follow the enclosed instructions.
- For **SINGLE USE ONLY**. Disposable tips are not autoclavable. Dispose after use.
- Where asepsis is required or deemed appropriate in the best professional judgement of the DHCP, this product should not be used.

## PRECAUTIONS

- Air/water syringe users should follow the syringe and dental unit manufacturers' instructions to maintain water quality and to minimize biofilm.
- When properly installed, Sani-Tip air/water syringe tips do not contribute to cross-contamination due to retraction of oral flora. To prevent system "suckback," add an anti-retraction (anti-siphon) device in an appropriate location. Consult local sanitary codes.
- The risks of re-use include equipment malfunction and cross-contamination.

## INSTALLATION INSTRUCTIONS

1. **LOOSEN THUMB NUT** on air/water syringe.

2. Insert new Sani-Tip® syringe tip and **PUSH** into adaptor as far as possible. While still pushing inward, twist tip 1/2 turn clockwise to create a watertight seal over internal tapered water spindle.

## ADVERSE REACTIONS

None Known.

## TROUBLESHOOTING

**PROBLEM:** Water leaking, wet air, and/or loose tips. Indicates either:

- a) Center hole of Sani-Tip® syringe tip is not fully seated over water spindle inside new adaptor, or
- b) Correct Sani-Tip® Conversion Kit is not properly installed, and/or is missing parts; thumb nut has not been fully tightened, or incorrect tips are being used.
- c) Internal water spindle is damaged.

**SOLUTION:**

- a) **LOOSEN THUMB NUT** when interchanging Sani-Tip® syringe tip. Insert new tip and **PUSH** into adaptor as far as possible. While still pushing inward, twist tip 1/2 turn clockwise to create a watertight seal over internal tapered water spindle. Tighten thumb nut.
- b) Make sure correct Sani-Tip® Conversion Kit is properly installed. Confirm presence of plastic cone and all O-rings. Fully tighten thumb nut after inserting new tip. Verify tips are clear plastic and manufactured by DENTSPLY Professional for the Sani-Tip® System.
- c) If internal water spindle inside Sani-Tip® adaptor is damaged, replace adaptor.

**PROBLEM:** Tip does not fit through thumb nut.

**SOLUTION:** Loosen thumb nut a full turn to expand opening of plastic cone.

**PROBLEM:** Little or no water.

**SOLUTION:** Check for clogged or damaged water spindle inside adaptor, or blocked water outlet in syringe head.

**PROBLEM:** Too much/little air or water pressure.

**SOLUTION:** Minor adjustments to air and water pressure can be made at the dental unit. Follow manufacturer's specifications.

**TEST PRIOR TO EACH USE**

3. Hand-tighten thumb nut. To redirect tip, push in on tip while rotating clockwise.

4. **Test prior to each use.** Express water only, then air only. If air contains moisture, or tip dislodges, refer to the Troubleshooting section of your Conversion Kit Directions for Use.
5. To replace tip, **LOOSEN THUMB NUT**. Remove and discard used tip. Repeat steps 2 and 3.

# Sani-Shield® Disposable Covers (Air/Water Syringe Polyethylene Sleeve)

## INDICATIONS FOR USE

Sani-Shield® Barrier Sleeves and Covers are intended to serve as a disposable barrier for dental instruments and equipment. This device is non-sterile and intended for single patient use only.

## INSTRUCTIONS

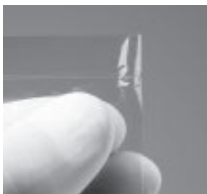
1. Insert new Sani-Tip® syringe tip. Test for proper air and water separation.

4. Slide tip along top edge of sleeve, toward tip opening.

7. A slight tug at open end of sleeve will create a seal around tip at thumb nut. For best results: push sleeve over tip, do not pull.

2. Locate tip opening at sealed end of sleeve.

5. Guide tip through opening. Slide opening along tip to thumb nut.

3. Slip Sani-Tip® syringe tip into open end of sleeve and aim toward tip opening.

6. Slide sleeve down over syringe handle.

This product does not replace cleaning and disinfection of dental instruments. Please clean and disinfect dental instruments after each patient as described by the instrument manufacturer.

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## IMPORTANT LICENSING NOTICE

You do not have a license, express or implied, under any of these patents to make or sell the adapter. You have an express license only under Patent Numbers U.S. 5,306,146; and U.S. 5,489,205; and no implied license under any other of these patents, to use the adapter in combination with the enclosed syringe tip(s) furnished by DENTSPLY Professional.

If you use the adapter with syringe tips not furnished by DENTSPLY Professional, you are in violation of this license granted to you and you may be liable for willful patent infringement. If you use the adapter with syringe tips not furnished by DENTSPLY Professional, you do so entirely at your own risk and DENTSPLY Professional denies any responsibility for such use, including product liability and any express or implied warranties.

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DENTSPLY DeTrey GmbH