X-flow[®]

Universal Flowable Restorative

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CAUTION: For dental use only.

USA: Rx only

Content	Page
1 Product description	2
2 Safety notes	3
3 Direct restorations: Step-by-step instructions	5
4 Luting of indirect restorations: Step-by-step instructions	7
5 Hygiene	8
6 Lot number and expiration date	9

1 Product description

X-flow[®] is a smooth flowing, light curing, radiopaque composite material which makes it ideal for restoration of small cavities, for lining and for luting of ceramic or composite inlays and onlays.

X-flow[®] is delivered in Compula[®] Tips¹ for direct intraoral and for indirect application.

X-flow[®] is available in five universal VITA^{®2} shades (A1/B1; A2; A3; A3,5; A4) and one dentin-like opaque shade (O-A3).

X-flow[®] adapts to the cavity walls without the use of hand instruments.

1.1 Delivery forms

- Predosed Compula[®] Tips

1.2 Composition

- Multifunctional acrylate resin
- Difunctional methacrylate resin
- Diethylene glycol dimethacrylate (DGDMA)
- UV stabilizer
- Ethyl-4(dimethylamino)benzoate
- Camphorquinone
- Butylated hydroxy toluene (BHT)
- Strontium-alumino-sodium-fluoro-phosphor-silicate glass
- Highly dispersed silicon dioxide
- Iron oxide pigments
- Titanium dioxide

¹ The Compula[®] Tip combines a Compules[®] Tip with a steel cannula for precise dosage and placement.

² Not a registered trademark of Dentsply Sirona, Inc.

1.3 Indications

- Minimal invasive restorations – this comprises:
 - Small restorations without direct antagonist contact in anterior and posterior teeth.
 - Extended pit & fissure sealings.
 - Shallow cervical lesions.
- Margin repair of direct and indirect restorations.
- Cavity lining.
- Adhesive luting of porcelain or composite inlays and onlays, if they allow sufficient light transmission to the tooth restoration interface.
- Blocking out undercuts in connection with preparation of indirect restorations.

1.4 Contraindications

- Use with patients who have a history of severe allergic reaction to methacrylate resins.
- Permanent restorations of occlusal stress-bearing cavities in permanent posterior teeth.
- Core build-up for full ceramic crowns.
- Luting of crowns or veneers.
- Luting of inlays and onlays that are impenetrable by dental curing lights or the walls of which exceed a thickness of 5 mm.

1.5 Compatible adhesives

The material is used following application of a suitable dentin/enamel adhesive. The material is chemically compatible with methacrylate-based dentin/enamel adhesives including Dentsply Sirona adhesives designed for use with visible light cured composite restoratives (see Directions for Use of selected adhesive).

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Directions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

The material contains methacrylates which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitisation or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Directions for Use. Any use of this product inconsistent with these Directions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.
- Use of Compula® Tips with the Compules® Tips Gun is recommended.
- The Compules® Tips Gun is only intended to be used with Dentsply Sirona Compules® or Compula® Tips.
- Compula® Tips may be used for direct intraoral application of restorative material into a cavity or for indirect application by first placing the restorative material on a pad. After intraoral use, discard Compula® Tips and do not reuse in other patients in order to prevent cross-contamination.
- To prevent Compula® Tips (when used for indirect application) from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the devices are handled offside the dental unit with clean/disinfected gloves. Discard devices if contaminated.
- The material should extrude easily. DO NOT USE EXCESSIVE FORCE. Excessive pressure may cause the Compula® Tip to rupture or to eject from the Compules® Tips Gun.
- Interactions:
 - Do not use eugenol- or hydrogen peroxide-containing materials in conjunction with this product since they may interfere with hardening of the product.
 - If mineral-impregnated (e.g. ferric compounds) retraction cords and/or hemostatic solutions are used in conjunction with adhesive procedures, marginal seal may be adversely affected, allowing microleakage, subsurface staining and/or restoration failure. If gingival retraction is necessary, use of plain, non-impregnated cord is recommended.
 - If refrigerated, allow material to reach room temperature prior to use.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store in sealed packs at temperatures between 10 °C and 24 °C (50 °F and 75 °F).
- Keep out of direct sunlight and protect from moisture.
- Do not freeze.
- Do not use after expiration date.

Humidity can adversely affect the properties of Compula® Tips. For unsealed Compula® Tips, the following is recommended:

- Store in a dry environment (< 80% relative humidity).
- Use unsealed Compula® Tips within 4 weeks.

3 Direct restorations: Step-by-step instructions

3.1 Shade selection

Shade selection is made whilst the teeth are hydrated and prior to isolation and cavity preparation.

1. Remove any extraneous plaque or surface stain.
2. Use the VITAPAN^{®3} classical shade guide. Use the central part of the respective VITA[®] teeth for shade selection.

3.2 Cavity preparation

1. Clean the tooth surface with a fluoride free prophylaxis paste (e.g. Nupro[®]).
2. Prepare cavity (unless not necessary, e.g. cervical lesion).
3. Use adequate isolation such as rubber dam.
4. Rinse surface with water spray and carefully remove rinsing water.
Do not desiccate the tooth structure.

3.2.1 Matrix placement and wedging (class II restorations)

1. Place a matrix (e.g. Palodent[®] V3 Sectional Matrix System or AutoMatrix[®] Retainerless Matrix System) and wedge. Burnishing of the matrix band will improve contact and contour.

3.3 Pulp protection, tooth conditioning/dentin pre-treatment, adhesive application

Refer to adhesive manufacturer's Directions for Use for pulp protection, tooth conditioning and/or adhesive application. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately to placement of the material.

3.4 Application

Apply material in increments immediately after the application of the adhesive. The material is designed to be cured in increments up to a 2 mm depth/thickness.

Using Compula[®] Tips

1. Insert Compula[®] Tip into the notched opening of the Compules[®] Tips Gun. Be certain that the collar on the Compula[®] Tip is inserted first.
2. Remove the colored cap from the Compula[®] Tip. The Compula[®] Tip may be rotated 360° to gain the proper angle of entrance into the cavity or to the mixing pad.
DO NOT USE EXCESSIVE FORCE.

Indirect use: to prevent Compula[®] Tip from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the Compula[®] Tip is handled offside the dental unit with clean/disinfected gloves. Dispense the necessary amount of material onto a mixing pad and protect against light. Apply material into the cavity with a plastic instrument.

- or -

Direct intraoral use: dispense the material directly into the cavity preparation using a slow, steady pressure.

3. To remove the used Compula[®] Tip, be sure that the Compules[®] Tips Gun plunger is pulled back completely by allowing the handle to open to its widest position. Apply a downward motion to the front end of the Compula[®] Tip and remove.

Excessive force.

Injury.

1. Apply slow and steady pressure on the Compules[®] Tips Gun.
2. Do not use excessive force. Compules[®] Tip rupture or ejection from Compules[®] Tips Gun may result.

³ Not a registered trademark of Dentsply Sirona, Inc.

3.5 Light curing

1. Light cure each area of the restoration surface with a suitable curing light designed to cure materials containing camphorquinone (CQ) initiator, i.e. spectral output containing 470 nm. Minimum light output must be at least 500 mW/cm². Refer to curing light manufacturer's Directions for Use for compatibility and curing recommendations.
2. Cure each layer according to the table below.

Insufficient curing.

Inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing cycle.
3. Check curing output.

Curing Light Output ≥ 800 mW/cm²

Shades	Increment/layer	Curing time
A1/B1, A2	2 mm	10 sec
A3, A3,5, A4	2 mm	20 sec
O-A3	2 mm	30 sec

Curing Light Output **500-800** mW/cm²

Shades	Increment/layer	Curing time
A1/B1, A2, A3	2 mm	20 sec
A3,5, A4, O-A3	2 mm	40 sec

3.6 Finishing and polishing

1. Contour the restoration using finishing burs or diamonds.
2. Use Enhance[®] finishing and polishing instruments and interproximal finishing and polishing strips.

For contouring, finishing, and/or polishing, follow the manufacturer's Directions for Use.

4 Luting of indirect restorations: Step-by-step instructions

4.1 Pre-Treatment of Restoration

The internal surfaces should be treated according to the manufacturers' recommendations.

4.2 Moisture Control

Surface cleanliness is paramount for the development of adhesion.

4.3 Enamel and Dentin Pre-Treatment

Clean enamel and dentin prepared during a previous appointment with a rubber cup and pumice or a cleaning paste such as Nupro® Prophylaxis Pastes. Remove paste and rinse surface with water spray. Clean freshly cut enamel and dentin with water spray. Remove rinsing water. Do not desiccate the tooth structure.

4.4 Pulp protection, tooth conditioning/dentin pre-treatment, adhesive application

Refer to adhesive manufacturer's Directions for Use for pulp protection, tooth conditioning and/or adhesive application. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately to placement of the material.

4.5 Dispensing of X-flow®

Dispense X-flow® onto the restoration and/or onto the tooth preparation.

4.6 Placement of Restoration

Place restoration. Apply pressure and then release to allow the hydraulic pressure to dissipate. Repeat this procedure at least one more time and ensure that the restoration is completely seated.

Remove as much excess as possible before the composite is light cured.

If any excess composite remains after curing, remove with curette, scalers or finishing burs/discs, or strips.

4.7 Light Curing

Proceed according to step 3.5.

Light cure each area of the restoration surface for at least 40 seconds.⁴

Depending on the number of surfaces of the restoration, the diameter of the curing light outlet, and the size of the tooth, this results in up to 5 curing cycles per restoration.

4.8 Occlusal Adjustments and Finishing

Adjust occlusion, remove excess material and finish and polish according to step 3.6.

⁴ Check curing light for minimum curing output of at least 500 mW/cm².

5 Hygiene

5.1 Compules® Tips Gun

NOTICE

Wrong cleaning or disinfection method.

Damage to the device.

1. Always disassemble device before reprocessing.
2. Do not clean or disinfect with highly aggressive agents (e.g. solutions based on orange oil or acetone).

5.1.1 Disassembly and removal of restorative material

1. Partially close the Compules® Tips Gun and place your thumb on the rear part of the hinge.
2. Push upwards and lift hinge. The Compules® Tips Gun is separated into two parts and the plunger is exposed.
3. Remove restorative material, if any, with a soft paper tissue and alcohol of 70% vol.

5.1.2 Manual cleaning

1. Discard gloves according to local regulations.
2. Disinfect hands with an appropriate bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations. Use according to disinfectant solution manufacturer's Directions for Use.
3. Use a new clean pair of examination gloves.
4. Clean device with a soft brush, warm water, and an appropriate cleaning solution until free of visible contamination. Pay special attention to device seams and insertions.
5. Remove cleaning solution residue with a cloth soaked with water.
6. Dry device with a lint-free single-use cloth.

5.1.3 Manual disinfection

1. After cleaning [5.1.2 Manual cleaning] thoroughly wipe all device surfaces with a single-use cloth in combination with an alcohol-based, bactericidal, virucidal and fungicidal disinfection solution approved according to local regulations and use according to disinfectant solution manufacturer's Directions for Use. Make sure disinfectant solution is compatible with cleaning solution. Pay special attention to device seams and insertions.
2. Remove disinfectant solution residue with a cloth soaked with water.
3. Dry device with a lint-free single-use cloth.

5.1.4 Automatic Washer-Disinfector

As an alternative to manual cleaning and disinfection, the device can also be processed in a washer-disinfector.⁵ Use only properly maintained, calibrated, and approved washer-disinfector according to ISO 15883-1.

1. Disassemble device.
2. Remove residual adhesive with tissue.
3. Place device in the washer-disinfector allowing water and detergent to enter into and drain off from device openings during washer-disinfection program and follow washer-disinfector's Directions for Use.
4. Run washer-disinfection program with A0 value ≥ 3000 (e.g. 5 min at ≥ 90 °C) using appropriate detergents.⁵

5.1.5 Sterilization (optional)

The device can also be autoclaved.

1. Steam autoclave device unwrapped at 134 °C/2 bar = 273 °F/29 psi for 3 minutes 30 seconds.

⁵ For example Miele Washer-Disinfector G7835 CD using program vario TD with neodisher® MediClean (alkaline detergent) and neodisher® Z (acid neutralization and cleaning detergent); both detergents from Dr. Weigert, Hamburg, Germany (no products or registered trademarks of Dentsply Sirona, Inc.)

5.1.6 Reassembly

1. After reprocessing insert plunger into the Compules® Tips Gun cylinder.
2. Press components together and snap hinge mechanism in place.

Do not continue to use damaged or soiled Compules® Tips Guns.

Compules® Tips Gun can be reprocessed for up to 100 times.

5.2 Compula® Tips – after intraoral use and/or if contaminated

Cross-contamination.

Infection.

1. Do not reuse Compula® Tips after intraoral use and/or if contaminated.
2. Dispose the intraorally used and/or contaminated Compula® Tips in accordance with local regulations.

6 Lot number () and expiration date ()

1. Do not use after expiration date.
ISO standard is used: “YYYY-MM” or “YYYY-MM-DD”.
2. The following numbers should be quoted in all correspondence:
 - Reorder number
 - Lot number
 - Expiration date

© Dentsply Sirona 2013-07-02

[These Directions for Use are based on Master Version 04]

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Dentsply DeTrey GmbH