Airblock™

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Airblock™

Gel for the prevention of an oxygen inhibited layer

CAUTION: For dental use only.

USA: Rx only.

Content	Page
1 Product description	2
2 Safety notes	3
3 Step-by-step instructions	4
4 Hygiene	5
5 Lot number and expiration date	6

1 Product description

Airblock™ is a gel which is light transmitting but impermeable to air. It allows the complete curing of the composite material including the surface layer. Thus **Airblock™** improves the surface quality and simplifies the finishing procedure.

When curing composite materials a layer of un-polymerized material remains on those surfaces which are not covered by matrices and therefore come into contact with oxygen. This so called oxygen inhibited layer can be prevented or reduced by the application of **Airblock™**.

1.1 Delivery forms

- Syringes with disposable Luer-Lock type applicator tips.

1.2 Composition

- Glycerol
- Highly dispersed silicone dioxide
- Surfactant

1.3 Indications

- For prevention or reduction of the oxygen inhibition layer on the surface of direct composite fillings, fissure sealants.
- For prevention or reduction of the oxygen inhibition layer on the surface of the margins of indirect restorations.

1.4 Contraindications

- None.

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

- **Avoid eye contact.** If accidental contact with the eyes occurs, immediately rinse with plenty of water.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.
- Tightly close syringes immediately after use.
- Always make sure the Luer-Lock type applicator tip is properly connected to Luer-Lock before application. The material should extrude easily.
- Discard Luer-Lock type applicator tip after use, as the Luer-Lock applicator tip may clog if gel is allowed to dry inside.
- The Airblock™ syringe cannot be reprocessed. To prevent the Airblock™ syringe from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe is handled with clean/disinfected gloves.
- As additional precautionary measure, the Airblock™ syringe may be protected from gross contamination but not from all contamination by applying the single use Disposa-Shield® protective barrier¹.
- Disposa-Shield® protective barrier and Airblock™ Luer-Lock type applicator tip are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
- Interactions:
 - If refrigerated, allow material to reach room temperature prior to use.

2.3 Adverse reactions

- None known.

2.4 Storage conditions

Inadequate storage conditions will shorten the shelf life and may lead to malfunction of the product.

- Store at temperatures between 10 °C and 24 °C (50 °F and 75 °F).
- Keep out of direct sunlight.
- Do not freeze.
- Do not use after expiration date.

¹ Disposa-Shield® Infection Control Barrier for syringe-type delivery systems, ReOrder A88013S1.

3 Step-by-step instructions

3.1 Application

Clogging.

Separation of Luer-Lock type applicator tip from syringe.

1. Do not allow material to dry inside.
2. Do not reuse Luer-Lock type applicator tip.
3. Always make sure Luer-Lock type applicator tip is properly connected to Luer-Lock before application. The material should extrude easily.

1. Remove cap from syringe.
2. Mount new Luer-Lock type applicator tip.
3. Drip Airblock™ from the application syringe onto the surface of which the oxygen inhibition layer should be prevented.
4. Light cure composite through the gel using a dental polymerization light. If using auto-curing products leave gel on the surface until the material has set.
5. After polymerization of the composite remove the gel by means of suction and air-water-spray.
6. After treatment discard the Luer-Lock type applicator tip.
7. Remove protective barrier and/or disinfect syringe.
8. Close syringe with cap.

Excessive force.

Injury.

1. Apply slow and steady pressure on the syringe.
2. Do not use excessive force. Excessive pressure may cause material and not fully seated Luer-Lock type applicator tip to eject from the syringe.

4 Hygiene

4.1 Airblock™ syringe

Cross-contamination.

Infection.

1. Syringe cannot be reprocessed.
2. To prevent the Airblock™ syringe from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe is handled with clean/disinfected gloves. Do not reuse syringe if contaminated.
3. Dispose contaminated syringe in accordance with local regulations.

4.2 Disposa-Shield® protective barrier

The use of protective barriers is an additional precautionary measure against gross contamination.

1. Right after use, carefully remove protective barrier together with cannula without contaminating the device.

Cross-contamination.

Infection.

1. Protective barriers are intended for single use only. Do not reuse in order to prevent cross-contamination.
2. Discard in accordance with local regulations.

4.3 Luer-Lock type applicator tip – single use

1. Right after use, carefully remove protective barrier together with Luer-Lock type applicator tip without contaminating the device.

Cross-contamination.

Infection.

1. Do not clean, disinfect or reuse Luer-Lock type applicator tip.
2. Properly dispose the used Luer-Lock type applicator tip in accordance with local regulations.

Clogging.

Separation of Luer-Lock type applicator tip from syringe.

1. Do not allow material to dry inside.
2. Do not reuse Luer-Lock type applicator tip.
3. Always make sure Luer-Lock type applicator tip is properly connected to Luer-Lock before application. The material should extrude easily.

5 Lot number () and expiration date ()

1. Do not use after expiration date. ISO standard is used: "YYYY-MM"
2. The following numbers should be quoted in all correspondence:
 - Reorder Number
 - Lot number
 - Expiration date

© Dentsply Sirona 2014-12-01

[These Instructions for Use are based on Master Version 05]

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Dentsply DeTrey GmbH