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- Disconnect cartridge dispensing gun, cartridge and attached using mixing tip for storage
- Immediately prior to subsequent usage, remove the used mixing tip, bleed cartridge, install a new mixing tip.
- Dispose of cartridge when empty. Do not attempt to refill or consolidate used cartridges.

3.3 DECA' cartridge loading

NOTE: Heavy Cartridge: Dropping may result in damage to the cartridge rendering it unusable. The DECA' cartridge fits snugly into the dispenser and must be inserted to the Volume Mix dynamic mixers. Carefully remove dispensing cartridge from the package. Also refer to separate dynamic mixer manufacturer's instructions.

- Remove the cap from the cartridge.
- Insert the cartridge into the dynamic mixer.
- Bleed the cartridge before installing a new mixing tip.
- Install a mixing tip. Use only DENTSPYL' mixing tips. Other tips will not seal properly.
- Place locking ring over the mixing tip and turn until ring is locked in place.
- Dispense the mixed base and catalyst into a suitable tray.
- Immediately prior to subsequent usage, remove the used mixing tip, bleed cartridge, install a new mixing tip.

3.4 Loading and using the digit power' Dispenser

To test functionality, dispense at least two directions below, extruding one cartridge, prior to use on patients.

- Prepare a digit power' Dispenser Single or Multi cartridge for use by snapping the circular disk off the cartridge. Insert unit dose cartridge into mixing tip.
- Enroll the cartridge into the square holes on the side of the cartridge to prevent extrusion.
- Lock the Cap onto the Dispenser with a clockwise turn until the alignment notches line up.
- To reduce dispenser contamination, placing a digit power' protective barrier sleeve over Dispenser is recommended.
- Depress the foot pedal and, away from the patient, bleed a small amount through the intrasulcular tip to assure proper flow and a good mix. Immediately proceed to the intrasulcular application.
- Start and stop the flow of material using the foot pedal.
- The dispensing rate can be adjusted by adjusting the Regulator speed control knob.
- To dissipate, relieve pressure on the Dispenser plunger by stepping off of the foot pedal and place Regulator back into the tool holder. Unscrew Cap and remove. Remove used cartridge/mixing tip/intrasulcular tip. Replace Cap prior to Dispenser disinfection.
- The blue plastic components – Cap, Grip and Adapter – can be autoclaved. Properly discard used cartridge/mixing tip/intrasulcular tip and prepare Dispenser for reuse. (See Hygiene)

3.5 Aqualis Ultra Cordless Tissue Managing Impression Technique: Dual Phase one-step ("double-mix"). Technique with a Rapid Tray with Double Arch Technique.
NOTE: Tray and wash material mixing time: It is highly recommended to utilize an immediate use pre-contraction gauge.

- Assure hemostasis and clean the field with air water spray.

• Blood and Fluid Control

- Fatigue, toothache, numbness and control of sulcular fluid may lead to inadequate margin capture and ill-fitting restorations.
- If blood and/or fluid cannot be adequately controlled, select another impression technique or postpone impression until tissue healing has occurred.

Aqualis Ultra Cordless Tissue Managing Wash Impression Material: Prepare digit power' Dispenser, attaching mixing tip/intrasulcular tip to be used as single or Multi digit power' cartridge/load in Dispenser and/or Dispenser with barrier sleeve.
Aqualis Ultra Cordless Tissue Managing Tray Impression Material: Dispense material (assistant) into impression tray. Do not overflow tray.
• Simultaneous with waxing, bleed a small amount through the mixing tip onto a pad, then immediately gently insert intrasulcular tip into the gingival sulcus of the preparation slightly apical to the preparation margin.

Periodontal Tissue or tooth injury

- Do not inject Aqualis Ultra Cordless Tissue Managing Impression material into sulcus of periodontally involved teeth. Removal may be difficult.
- Avoiding any contact to teeth with open functions or compromised periodontal attachment.
- Select other material or impression technique.

- Depress the foot pedal to begin the flow of material. Material should flood the sulcus.
- Dispensing material ahead of the intrasulcular tip, gently move the intrasulcular tip completely around the preparation margin. Do not lift the intrasulcular tip out of the sulcus to minimize air entrapment.
- Once the sulcus is completely filled, with the foot pedal still depressed and material flowing, gently withdraw the intrasulcular tip, continuing to dispense mixed base and catalyst until the preparation margin is completely filled.
- For single preparations (Single cartridge): Entire cartridge may be dispensed onto the single preparation. Tray must be sealed immediately upon completion of dispensing.
- For multiple preparations (Multi cartridge): Move intrasulcular tip to the subsequent preparation(s) and repeat, covering at least the gingival 1/3 of each preparation to be impressed. The entire cartridge may be dispensed. Tray must be sealed immediately upon completion of dispensing.

NOTE: Tray and wash material insertion timing
Single: Load tray and dispense at least 35 seconds from time of first spraying of Aqualis Ultra Cordless Tissue Managing Single Wash Material intrasulcularly.
Multi: Loaded tray must be inserted within 1 minute from time of first spraying of Aqualis Ultra Cordless Tissue Managing Multi Wash Material intrasulcularly.

- Insert filled tray into the mouth.
- The Minimum Removal Time is 3 minutes for Single, 4 minutes 30 seconds for Multi from the start of mix.
- Follow disinfection procedures before shipping, scanning and/or pouring.

Timing Guide

Begin simultaneous syringing and tray loading

Complete syringing and seat loaded tray

4. HYGIENE

Cross-contamination

- Do not reuse single use products. Dispose in accordance with local regulations.
- Reprocess reusable products as described below.

4.1 Cleaning the digit power' Dispenser and Cartridge Dispensing Gun

Remove the cap and discard. The digit power' Dispenser and cartridge dispensing gun may be cleaned by scrubbing with a towel soaked with hot water and soap or detergent. Do not allow solution to penetrate the casing. To clean digit power' Dispenser, remove Cap and depress foot pedal to advance plunger. Excess material may be cleaned with a toothbrush. Refer to the separate cleaning manufacturer's instructions for Dispenser cleaning and disinfecting instructions.

4.2 Disinfection of the digit power' Dispenser and Cartridge Dispensing Gun

- Disinfect Dispenser(s) and tips left in place on cartridges for storage with a hospital-level tuberculocidal/disinfectant according to national and local regulations.
- Sodium hypochlorite (0.525%), water based dual phenolics, alcohol based dual phenolics, and dual or synergized quaternary ammoniums are approved disinfectants.
- Use impregnated wipes not sprays. Do not immerse the Dispenser in disinfectant. The disinfectant manufacturer's directions should be followed properly for optimum results.

NOTE: Vigorous wiping can destroy the label. Wipe cartridge gently. Some phenolic based agents and iodophor based products may cause surface staining.

The digit power' Dispenser blue plastic components – Cap, Grip and Adapter – can be steam autoclaved, at 132°C for a minimum of 5 minutes (unwrapped). Cap, Grip and Adapter may be autoclaved at least 200 times. Use of other sterilization procedures/methods is at the discretion and sole responsibility of the practitioner.

4.3 Disinfection of the Impression material

The impression should be disinfected with a hospital-level tuberculocidal disinfectant according to the manufacturer's directions.

- Iodophors, sodium hypochlorite (0.525%), chlorine dioxide, and dual or synergized quaternary ammoniums are approved disinfectants.
- Thoroughly soak by spraying or immerse the impression in any recommended disinfectant for the contact time recommended by the disinfectant manufacturer for optimum results.
- Water-based disinfectant solutions are preferred.

4.4 Casting

- The impression should be removed from the disinfectant, rinsed with water and lightly air dried.
- The impression may be poured in plaster or stone 30 minutes after disinfection or delayed up to fourteen days. For epoxy dies, delay pour 60 minutes.
- Do not store impression in direct sunlight.
- If the impression is to be shipped, use suitable packaging to preclude distortion.
- The material is compatible with a range of metal plasters and die stones. Impressions may be silver or copper coated.
- Aqualis Ultra Cordless Tissue Managing Impression Material may be scanned by dental numbering units designed for digital CAD/CAM procedures.

5. LOT NUMBER AND EXPIRATION DATE

- Do not use after expiration date. ISO standard uses: "YYYY/MM."
- The following numbers should be quoted in all correspondences:
 - Expiration number
 - Lot number
 - Recorder date

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