Blueprint[®] Xcreme

Zelgan[®]+

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
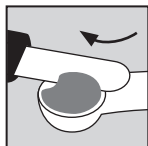
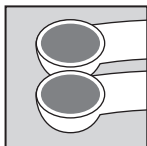
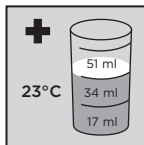
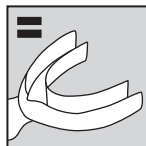
Medium Tray

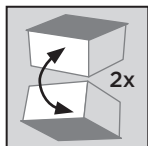Handling

Loosen Powder
in Tub turning the
box over at least
two times

max. 30 Sec.

max. 1 Min.

min. 1 Min.

← 2 Min. 30 Sec. →

Blueprint® Xcreme

Zelgan®+

Dust-Free Alginate Impression Materials

CAUTION: For dental use only.

USA: Rx only

Content	Page
1 Product description	3
2 Safety notes	4
3 Step-by-step instructions	5
4 Hygiene	6
5 Lot number and expiration date	6

1 Product description

Blueprint® Xcreme and Zelgan®+ alginates are irreversible hydrocolloid materials for taking dental impressions.

Blueprint® Xcreme and Zelgan®+ alginate impressions allow the option of delayed pouring either in the office or shipped to an off-site laboratory.

Blueprint® Xcreme and Zelgan®+ alginate powders have been especially formulated to eliminate dust during mixing. The materials reduce the concentration of airborne particles to a concentration lower than the limit of detection.

Blueprint® Xcreme and Zelgan®+ alginates comply with ISO 1563 and EN 21563, respectively.

While **Blueprint® Xcreme** is of a light-blue color, **Zelgan®+** is peach-colored.

1.1 Delivery forms

- **Blueprint® Xcreme** alginate is available in either 1 kg refill package (containing two 500 g pouches) or 10 kg bulk packages (containing twenty 500 g pouches), with each package containing a storage container, a powder measure scoop, a water measuring cup and Instructions for Use.
- **Zelgan®+** alginate is available in either a single 500 g pouch, 1 kg refill package (containing two 500 g pouches) or 10 kg bulk packages (containing twenty 500 g pouches), with each package containing a storage container, a powder measure scoop, a water measuring cup and Instructions for Use.

1.2 Composition

- Diatomaceous earth (amorphous SiO₂)
- Potassium alginate
- Calcium sulfate
- Paraffin oil
- Sodium phosphate
- Potassium hexafluorotitanate
- Magnesium oxide
- Pigments
- Spearmint oil

1.3 Indications

Making of dental impressions for the fabrication of casts for:

- Case study models.
- Orthodontic models.
- Opposing models.
- Removable partial denture frameworks not involving transfers and/or precision attachments.
- Removable retainers and splints.
- Provisional restorations.

1.4 Contraindications

- Severe allergy against any of the components.

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

Blueprint® Xcreme and Zelgan®+ alginates contain diatomaceous earth (amorphous SiO₂).

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- **Avoid inhalation:** In case of inhalation, move subject to fresh air. Drink water to clear throat and blow nose to evacuate material. If irritation or soreness persists, seek medical attention.
- **Prevent ingestion.** If accidental swallowing occurs, have patient drink plenty of water. If nausea or illness develops, seek medical attention.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses in accordance with local best practice.
- Replace storage container lid tightly immediately after dispensing.
- The temperature and hardness of the water used for mixing will affect the working time and setting time. Recommended water temperature is 23 °C/73 °F.

- Blueprint® Xcreme and Zelgan®+ alginates are not recommended for transfer impressions or final impressions for fixed restorations.
- Do not store impressions submerged in water.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Keep out of direct sunlight.
- Store in a tightly closed container at temperatures between 5 °C and 27 °C (41 °F and 80 °F).
- Use the product at room temperature.
- Protect from moisture.
- Do not freeze.
- Do not use after expiration date.

3 Step-by-step instructions

3.1 Preparation and dosage¹

1. Transfer Blueprint® Xcreme or Zelgan®+ alginates from pouch into the storage container.
2. Alginate impression materials tend to settle during transit and storage. Always loosen the powder before use by inverting the storage container twice.
3. For correct proportioning use Blueprint® Xcreme or Zelgan®+ alginates water measuring cup (17, 34 and 51 ml marks) and powder measure scoop (25 ml).
4. Scoop powder lightly and level off; do not tap the scoop.

Sectional impression:	1 level scoop of powder (25 ml) + lower mark of water measure (17 ml)
Small full impression:	2 level scoops of powder (50 ml) + middle mark of water measure (34 ml)
Large full impression:	3 level scoops of powder (75 ml) + upper mark of water measure (51 ml)

5. If a thinner or thicker mix is required use slightly more or less water.

3.2 Mixing

1. Machine mix for 8-12 seconds or according to manufacturer's Instructions for Use.
2. Hand mix vigorously in a suitable bowl for 30 seconds.

3.3 Working and setting time²

Working and setting times can be modified by altering the temperature of the mixing water. The time of placement may vary by altering working and setting times. In general, at normal room temperature, the working time is 1 minute and the impression should remain in the mouth for at least 1 minute. This leads to a total setting time of 2 minutes and 30 seconds from start of mixing.

¹ For testing according to ISO 1563: Powder : water ratio = 7 g powder : 15 ml water.

² The quoted working time is based on the use of distilled water but there is no significant change if tap water, of any hardness, is used.

3.4 Impression taking

1. Have the patient rinse the mouth with warm water.
2. Load and seat the tray into the mouth immediately after completion of mixing and retain, under slight pressure, for at least 1 minute.
3. Remove the impression from the mouth and rinse free from saliva and debris under cold running water.

3.5 Cleaning and disinfection

1. Immerse the impression in a suitable disinfection solution (e.g. Zeta 7 Solution³ or Impresept³) for 10 minutes.
2. Rinse impression under running water for approximately 15 seconds.

3.6 Storage of impressions

1. It is recommended that the model be cast from the impression immediately after cleaning (and disinfection).
2. If this is not possible store the impression next to a damp napkin in a hermetically sealed polyethylene bag at room temperature. Under these conditions the casting can be delayed for up to 5 days after impression taking.

3.7 Casting

Compatibility with cast materials has been confirmed for the type 3 plaster Elite Model Fast³ and type 4 plaster Elite Rock³.

4 Hygiene

Cross-contamination.

Infection.

1. In order to prevent cross-contamination do not touch storage container, powder measure scoop, water measuring cup and Instructions for Use with contaminated hands. Protect from aerosols.
2. The storage container, powder measure scoop and water measuring cup may be cleaned by scrubbing with hot water and soap or detergent.

5 Lot number () and expiration date ()

1. Do not use after expiration date.
ISO standard is used: "YYYY-MM" or "YYYY-MM-DD".
2. The following numbers should be quoted in all correspondence:
 - Reorder number
 - Lot number
 - Expiration date

© Dentsply Sirona 2011-12-16

[These Instructions for Use are based on Master Version 07]

³ Not registered trademarks of Dentsply Sirona, Inc.

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³ Ikke registrerte varemerker som tilhører Dentsply Sirona, Inc.
Dentsply DeTrey GmbH