Calibra® Bio Bioceramic Luting Cement

DIRECTIONS FOR USE – ENGLISH

For dental use only. U.S.A.: Rx only.

1. PRODUCT DESCRIPTION

Calibra® Bio Bioceramic Luting Cement is a permanent, self-curing, radiopaque, water-based dental luting cement. Calibra® Bio Cement is a hybrid between a glass ionomer cement and a ceramic cement based on calcium aluminate. When the powder is mixed with the liquid in a capsule mixer/triturator, the resulting chemical reactions of the calcium aluminate and the reactive ionomer glass form a biocompatible cement material. When exposed with saliva in the oral environment, Calibra® Bio Cement has the ability to create hydroxyapatite on its surface.

Calibra® Bio Bioceramic Luting Cement complies with ISO 9917-1.

Capsule Extruder 2 is the device designed for extrusion and direct application of Calibra® Bio Bioceramic Luting Cement capsules.

1.1 INDICATIONS

Permanent cementation of:

- Metal and porcelain fused to metal crowns and bridges
- Gold inlays and onlays
- Cast or prefabricated metal posts
- High-strength ceramic crowns and bridges suitable for conventional cementation (e.g. zirconia, alumina, and lithium disilicate)

1.2 CONTRAINDICATIONS

Calibra® Bio cement is contraindicated for use as a direct pulp-capping agent. In case of close proximity to the pulp, use accepted standard clinical procedures.

1.3 DELIVERY FORMS

Calibra® Bio cement is available in:

- Predosed (334 mg powder, 196mg liquid) mixing capsule for direct intra-oral application with a minimal dispensable amount of 0.17 ml.
- Universal white shade

1.4 COMPOSITION

Powder: Calcium aluminate; Glass powder; inert strontium fluoride

Liquid: Water; polyacrylic acid; tartaric acid; lithium chloride; nitrotriacetic acide/trisodium salt

2. GENERAL SAFETY NOTES

Be aware of the following general safety notes and the special safety notes in other sections of these directions for use.

Safety alert symbol

This is the safety alert symbol. It is used to alert you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury.

2.1 WARNINGS

Calibra® Bio cement contains polyacrylic acid and tartaric acid which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.

- **Avoid Eye Contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- **Avoid Skin Contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs immediately remove material with cotton and alcohol and wash well thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid Contact with Oral Soft Tissues/Mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

Do not use on patients with known allergy to polyacrylic acid. In rare cases, the product may cause sensitivity to some persons. If such reactions are observed, discontinue the use of the product and consult a physician.

2.2 PRECAUTIONS

1. This product is intended to be used only as specifically outlined in the Directions for Use. Any use of this product inconsistent with the Directions for Use is at the discretion and sole responsibility of the practitioner.
2. Wear suitable protective eyewear, clothing and gloves. Protective eyewear is recommended for patients.
3. Devices marked "single use" on the labeling are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
4. The material should extrude easily. DO NOT USE EXCESSIVE FORCE. Excessive pressure may cause the capsule to rupture.
5. Increased temperature and/or humidity reduce available work time and accelerate set time. Material will set faster in the mouth than at the bench. Do not use bench cure rates to judge mouth cure rates.
6. Slow speed amalgamators or slow speeds on amalgamators are not recommended.
7. Interactions
 - Do not use any pre-treatment agents, e.g. silanes, primers or bonding agents when cementing with Calibra® Bio cement. Use of any such pre-treatments may compromise the bonding of the cement. See step-by-step instructions below for recommended tooth and restoration treatment.

2.3. ADVERSE REACTIONS

Product may irritate the eyes and skin. Eye contact: irritation and possible corneal damage. Skin contact: irritation or possible allergic response. Reddish rashes may be seen on the skin. Mucous membranes: inflammation (see Warnings).

2.4. STORAGE CONDITIONS

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product. Do not open the aluminum pouch until immediately before use. Store in sealed packs at temperatures between 4 °C and 20 °C (39 °F and 68 °F). Protect from moisture. Do not freeze. Do not use after expiration date.

3. STEP-BY-STEP INSTRUCTIONS

3.1 TOOTH PREPARATION

1. Finish preparation or remove the temporary restoration and any remaining temporary cement.
2. If needed, clean the tooth using pumice or prophylactic paste (with or without fluoride). Clean enamel and dentin with water spray and carefully dry with oil free air spray or cotton pellets. Do not desiccate the tooth structure.
3. Pulp Protection: In deep cavities/preparations, cover the dentin close to the pulp (less than 1mm) with a hard setting calcium hydroxide liner (Dycal® Liner).
4. Isolate the site with cotton rolls. Avoid contamination.

3.2. PREPARATION OF THE RESTORATION

- High strength glass ceramic suitable for conventional cementation: Etch with HF acid according to manufacturer's instructions. Do not use any pre-treatment agents, e.g. silanes, primers or bonding agents.
- All other restorations: Internal surfaces of restorations should be clean and dry prior to cementation. Internal surface microetching (sandblasting with 50µm alumina) of internal surfaces of the restoration is recommended.

Compromised adhesion due to improper treatment

- Tooth structure should be moist; Gently dry with air or blot with dry cotton
- Restoration surfaces should be clean and dry. Clean with alcohol and rinse thoroughly if contamination occurs.
- Do not use any pre-treatment agents, e.g. silanes, primers or bonding agents when cementing with Calibra® Bio Cement. Use of any such pre-treatments will destroy the bond between cement and restoration

3.3 ACTIVATION

-  Work Quickly

Time after activation

Mixing	8 seconds
Working Time - fill and seat crown	Up to 2 minutes from start of mixing
Remove Excess - rubbery phase	2 minutes after restoration seating
Stabilize	4 minutes
Final Set	6 minutes after restoration seating (6-8 minutes from start of mixing)

The reaction between the liquid and powder starts directly after activation. Start mixing immediately following activation.

1. Open the aluminum pouch and remove the capsule.
2. Hold the capsule firmly against a hard, flat surface with the plunger facing down.
3. Push down firmly, forcing the plunger into the body of the capsule until it stops.

3.4 MIXING

1. Immediately place the capsule in a high-frequency oscillating (4,000 to 5,000 cpm) or rotating capsule mixer.
2. Mix for 8 seconds

Clinical experience with your capsule mixer is the best guide to mixing the Calibra® Bio cement capsule. Variation in capsule mixers (even from the same manufacturer) will occur and the trituration speed of capsule mixers will vary with the line voltage on which they are operated.

High-speed capsule mixers are recommended for best results.

Brand/Manufacturer	Speed
ProMix® Amalgamator (Dentsply Sirona)	High: Rabbit (4200 RPM)
Crescent® Wig-L-Bug® Mixer Model MSD (Dentsply Sirona)	5000 RPM
Dentsply Sirona Capsule Mixer	N/A (Fixed at 4550 RPM)
Ultramast S* (SDI)	N/A (Fixed at 4550 RPM)

*Not registered products of Dentsply Sirona

3.5. APPLICATION

1. Immediately insert the capsule into the Capsule Extruder 2.
2. Click the trigger of the Capsule Extruder 2 until cement is seen through the clear nozzle. Immediately start dispensing.
3. Without delay, using gentle pressure, apply a thin, uniform layer of cement to the entire internal surface of the restoration directly from the capsule tip. At room temperature, Calibra® Bio Cement offers a minimum work time of 2 minutes.

The capsule may be rotated in the Capsule Extruder 2 to allow direct intraoral access for placement of cement into preparations with internal anatomy. For endodontic post spaces, use of a Lentulo Spiral or metal file to aid placement in the post space is recommended.

Danger of injury due to excessive force

- Apply slow and steady pressure on the extruder
- Do not use excessive force – capsule rupture may result

3.6. WORKING AND SETTING TIME

Working time (from end of mixing) at 23 °C/73 °F: 2 minutes.
Setting time (from end of mixing): 6 to 8 minutes.

Temperatures above 23 °C/73 °F will accelerate setting and reduce the working time available.

3.7. CEMENTATION TECHNIQUE

1. Immediately seat the restoration in the mouth. Verify complete seating. A gentle rocking or vibratory motion may be helpful to insure optimal seating.
2. Stabilize the restoration under pressure until the cement reaches a rubber-like consistency, approximately 2 minutes.
3. Complete excess cement removal using an instrument such as a rubber tip, a scaler or an explorer. **Note:** Cement within the crown has not yet set. Do not move, torque, or disturb the crown during cleanup.
4. Protect restoration from contamination and movement for 4 minutes, until the final set of the cement (6 minutes from restoration seating, 6-8 minutes from start of mix).
5. Check the area around the margins of the restoration (especially the gingival sulcus) and remove any remaining cement.
6. Advise the patient not to load their restoration for the first hour after cementation.

**Inadequate retention due to insufficient stabilization**

- Stabilize restoration during cleanup
- Protect restoration from movement during setting
- Allow cement to self-cure without disturbing for 6 minutes

Calibra® Bio Cement is radio-opaque, with a 1 mm radio-opacity equivalent to 1.5 mm radio-opacity of aluminum. Aluminum has a radio-opacity equivalent to that of dentin. Thus 1 mm of material having a radio-opacity equivalent to 1 mm of aluminum has a radio-opacity equivalent to that of dentin.

4. HYGIENE**Cross-contamination**

- Do not reuse single use products. Dispose of in accordance with local regulations.
- Reprocess reusable products as described below.

NOTICE**Wrong cleaning or disinfection method.**

Damage to the device.

1. Always disassemble used capsule from the Capsule Extruder 2 before final reprocessing.
2. Do not submerge into disinfection solutions.
3. Do not ultrasonic clean.
4. Start reprocessing within 1 hour after use.

For Manual Cleaning and Disinfection

- Discard used gloves according to local regulations.
- Disinfect hands with an appropriate bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations. Use according to disinfectant solution manufacturer's Instructions for Use.
- Use a new clean pair of examination gloves.

INSTRUCTIONS FOR CLEANING, DISINFECTING AND STERILIZING CAPSULE EXTRUDER 2

WARNINGS	<ul style="list-style-type: none"> • Intermediate level disinfection is appropriate for the Capsule Extruder 2. • Steam autoclaving sterilization is recommended for the Capsule Extruder 2. • It is not recommended to submerge Capsule Extruder 2 device into disinfection solutions. • Do not allow the device to exceed 140° C.
LIMITATIONS ON REPROCESSING	<ul style="list-style-type: none"> • Repeated processing has minimal effect on the Capsule Extruder 2 • The Capsule Extruder 2 can be subjected to the manual cleaning and disinfection process up to 1000 times. • The Capsule Extruder 2 can be subjected to the automated washer/ disinfectant process up to 200 times. • The Capsule Extruder 2 can be subjected to the steam autoclave sterilization process up to 1000 times. • Cold liquid immersion disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use.
POINT OF USE	<ul style="list-style-type: none"> • Remove excess soil with disposable cloth / paper wipe in combination with an alcohol-based, tuberculocidal, quaternary ammonium solution with a label claim for cleaning <p>Prior to Cleaning and Disinfection, the Capsule Extruder 2 should be disassembled:</p> <ol style="list-style-type: none"> 1. Remove used capsule: Press the release button until the push rod releases. 2. Slightly rotate and withdraw the used capsule. Do not attempt to refill or consolidate used capsules. Dispose of used capsules in accordance with local regulations. <p>It is recommended that the Capsule Extruder 2 be reprocessed as soon as is reasonably practical following use.</p>
CONTAINMENT AND TRANSPORTATION	No particular requirements. Handle with care.
CLEANING AND DISINFECTION: AUTOMATED	Thermal disinfectant: Manufacturer's specification according to DIN EN ISO 15883. Cleaning program (AO value > 3000 or, at least 5 min at 90 °C/194 °F) as indicated by the manufacturer in the operating instructions. Follow manufacturer's recommendation for use of detergent and neutralizer, observing concentrations and contact times.
CLEANING AND DISINFECTION (INTERMEDIATE-LEVEL):	As an alternative to automated cleaning and disinfection, the Capsule Extruder 2 has to be cleaned by scrubbing with hot water and enzymatic detergent solution prepared according to the manufacturer's instructions.
MANUAL	<ul style="list-style-type: none"> • Clean with a soft brush. • Rinse under running potable water. • Dry with a lint-free single-use cloth. <p>Follow by disinfection as outlined below.</p> <p>Wipe all device surfaces with a single-use cloth in combination with an alcohol-based, tuberculocidal, quaternary ammonium solution e.g., BBraun Meliseptol® Rapid.</p> <ul style="list-style-type: none"> • Thoroughly disinfect the surface with the soaked cloth paying special attention to any cavities and holes. • Ensure the entire surface remains wet for the entirety of the required contact time in accordance with the disinfectant solution manufacturer's instructions for use. • Remove disinfectant solution residue with a cloth soaked with deionized water. • Dry with a lint-free single-use cloth
PACKAGING	Paper / plastic steam sterilization pouches e.g., AssurePlus® Sterilization Pouches, may be used, but are not required.
STERILIZATION	<p>After Cleaning and Disinfecting, steam autoclaving is recommended.</p> <ul style="list-style-type: none"> • Gravity Displacement Steam Sterilization: Bagged/pouched: 134 °C/273°F for 3 minutes 30 seconds • Prevacuum Steam Sterilization: Full Cycle, bagged/pouched: 134 °C/273°F for 3 minutes 30 seconds <p>Follow manufacturer's instructions for loading and operation cycle</p>
DRYING	Use the drying cycle of the autoclave, minimum 30 minutes. Allow the Capsule Extruder 2 to fully air dry before storage
MAINTENANCE	<p>Visually inspect to ensure that all contamination has been removed. If the Capsule Extruder 2 is discolored, damaged, worn, or distorted it should be discarded.</p> <p>Regularly apply dental instrument lubricant.</p>

STORAGE	Store the cleaned, disinfected and sterilized Capsule Extruder 2 at room temperature, away from moisture or excessive humidity. Instruments steam autoclaved bagged should remain bagged until ready for use. Prior to subsequent re-use, inspect the sterilization pouch and the Capsule Extruder 2. If the integrity of the sterilization pouch has been compromised, the Capsule Extruder 2 must be reprocessed prior to use.
	Instruments steam autoclaved unwrapped should be used immediately.
MANUFACTURER CONTACT	Within the United States, call Dentsply Sirona at 1-302-422-4511. For areas outside the United States, contact your local Dentsply Sirona representative.

5. LOT NUMBER AND EXPIRATION DATE

1. Do not use after expiration date. ISO standard is used: "YYYY-MM-DD"
2. The following numbers should be quoted in all correspondence.
 - Reorder Number
 - Lot number
 - Expiration date

Manufactured by
Dentsply LLC
38 West Clarke Avenue
Milford, DE 19963 USA
Tel.: 1-302-422-4511
www.dentsplysirona.com

Dentsply De Trey GmbH
De-Trey-Str. 1
78467 Konstanz
Germany
Tel.: 49-7531-583-0
www.dentsplysirona.com

Distributed by
Dentsply Canada
161 Vinyl Court
Woodbridge, Ontario
L4L 4A3 Canada

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