# Calibra®

## Silane Coupling Agent

### DIRECTIONS FOR USE - ENGLISH

For dental use only

**CAUTION:** U.S. federal law restricts this device to sale by or on the order of a dentist.

### PRODUCT DESCRIPTION

Single-component Calibra® Silane Coupling Agent, when applied to the etched portion of a ceramic restoration, forms a surface which is chemically compatible with adhesives such as Prime & Bond® NT™ or Prime & Bond® NT™ Dual Cure Nano-Technology Universal Dental Adhesive and resin luting cements such as Calibra®.

Convenient syringe and needle tip delivery allows easy, precise placement onto areas to be bonded.

### INDICATIONS:

Application to bonding surfaces of ceramic, porcelain and composite inlays/onlays, crowns and veneers prior to adhesive cementation.

### CONTRAINDICATIONS:

Calibra® Silane Coupling Agent is contraindicated for use on hard and soft tissue including tooth structure, oral mucosa, skin, eyes and dental pulp tissue.

### Warnings:

- Flammable: Calibra® Silane Coupling Agent contains acetone. Keep away from sources of ignition. Do not breathe vapor.
- If accidental contact with the eyes occurs, immediately rinse with plenty of water and seek medical attention. Do not take internally.

### Precautions:

- Calibra® Silane Coupling Agent should be kept out of direct sun light and stored in a well ventilated place at room temperature not exceeding 25°C/77°F.
- Calibra® Silane Coupling Agent should extrude easily. DO NOT USE EXCESSIVE FORCE. Replace original cap tightly after each use to avoid evaporation. Discard syringe needle tip after use.
- The Calibra® Silane Coupling Agent container should be tightly closed immediately after use.

### Adverse Reactions:

Allergic contact dermatitis and other allergic reactions may occur in susceptible individuals.

### Step by Step Instructions:

#### Treatment of Ceramic/Porcelain/Composite Restorations

Microetching (sandblasting) with 50µ alumina or hydrofluoric acid chemical etching of the internal surfaces of a ceramic restoration is recommended.

Check the fit and esthetics of the restoration.

**Technique Tip:** Occlusal adjustment of inlays/onlays is best accomplished after final cementation.

Once fit and esthetics are verified, thoroughly rinse any try-in paste from restoration and preparation surfaces using water.

The internal surface of the restoration should be clean and dry prior to application of Calibra® Silane Coupling Agent. Organic debris accumulated during try-in may be removed by cleaning with ethanol or acetone followed by cleaning in water in an ultrasonic cleaner. Prior to applying the silane, clean the internal surface of the restoration with Caulk 34% Tooth Conditioner Gel (34% phosphoric acid). Apply for 30 seconds and rinse thoroughly with water for 20 seconds. Air-dry the internal surface of the restoration.

#### Restoration Silanation

Attach supplied needle tip to end of the Calibra® Silane Coupling Agent syringe. Gently apply pressure to syringe plunger. DO NOT USE EXCESSIVE FORCE. Calibra® Silane Coupling Agent should express easily, one drop at a time. Directly apply the silane to the etched, clean internal surface of the restoration and allow to air dry.

Proceed to Adhesive cementation according to the manufacturer's directions.

**NOTE:** Avoid application of the silane to the external surface of the restoration. Contact with the external surface will cause bonding of the cement to the external surface, complicating clean-up and necessitating removal of the glazed portion of the external surface of the restoration.

If silanated surface becomes contaminated, clean surface with alcohol or acetone, allow to air dry and repeat application as outlined above.

1In Europe: DeTrey Conditioner 36

 Manufactured by  
Dentsply Caulk  
38 West Clarke Avenue  
Milford, DE 19963 USA  
Tel.: 1-302-422-4511  
www.dentsplysirona.com

 Dentsply DeTrey GmbH  
De-Trey-Strasse 1  
78467 Konstanz, Germany  
Tel.: 49-7531-583-0  
www.dentsply.eu

Distributed by  
DENTSPLY Canada Ltd.  
161 Vinyl Court  
Woodbridge, Ontario  
Canada, L4L 4A3

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