ChemFil® Rock Capsule Extruder 2

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ChemFil® Rock Capsule Extruder 2

Advanced Glass Ionomer Restorative

CAUTION: For dental use only.

USA: Rx only.

Content	Page
1 Product description	2
2 Safety notes	3
3 Step-by-step instructions	5
4 Hygiene	7
5 Lot number and expiration date	8

1 Product description

ChemFil® Rock restorative is a high strength zinc-reinforced glass ionomer restorative offering the following features:

- Resistance to fracture and wear.
- Marginal seal in dentin and enamel comparable with composites.
- Self-adhesive to enamel and dentin.
- Releases fluoride.
- Packable, non-sticky and sculptable consistency.
- Bulk placement and self-cure setting.
- High contrast to tooth substance through high opacity.
- Effective capsule application with ergonomic applicator.
- Available shades: A1¹, A2¹, A3¹, A4¹ Opaque and Contrast White.

Capsule Extruder 2 is the device designed for extrusion and direct application of ChemFil® Rock restorative capsules.

1.1 Delivery forms

- Predosed mixing capsule for direct intra-oral application with a minimal dispensable amount of 0.1 ml.

¹ According to the VITAPAN® classical shade guide. Not a registered trademark of Dentsply Sirona, Inc.

1.2 Composition

- Calcium-aluminum-zinc-fluoro-phosphor-silicate glass
- Polycarboxylic acid
- Iron oxide pigments
- Titanium dioxide pigments
- Tartaric acid
- Water

1.3 Indications

Especially designed for:

- Semi-permanent restorations of Class I and II cavities in posterior teeth.

In addition suitable for:

- Restoration of deciduous teeth.
- Restoration of Class V lesions and cavities.
- Restoration of Class III cavities.
- Base/core-build-up. When used as a core-build-up, 2/3 of the remaining coronal dentin or at least 2 mm of circumferential coronal dentin should be left for retention.

1.4 Contraindications

- Direct or indirect pulp-capping.
- Use in patients with known allergies to calcium-aluminum-zinc-fluoro-phosphor-silicate glass or any of the other components.

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

The mixed material contains polycarboxylic acid and tartaric acid and may be irritating to skin, eyes and oral mucosa.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes rinse immediately with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and wash thoroughly with soap and water.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective eyewear, mask, clothing and gloves for the dental team and protective eyewear for patients.
- Capsules are intended for single use only. Discard after use. Do not reuse.
- If material has started to set excessive pressure may result in rupture of the capsule.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store in sealed packs at temperatures between 4 °C and 28 °C (39 °F and 82 °F). Refrigerated storage is not recommended.
- Do not freeze.
- Protect from moisture.
- Do not use after expiration date.

3 Step-by-step instructions

3.1 Shade selection

ChemFil® Rock restorative offers five shade options: A1, A2, A3, A4 Opaque, Contrast White.

3.2 Cavity preparation

1. In all cavity classes cavity preparation may be kept to the minimum required for caries removal and for preparation of butt joint margins.
2. Prepare for a restoration thickness of more than 1 mm.
3. Prepare butt joints – avoid bevelled margins.
4. Rinse surface with water spray and carefully remove excess rinsing water with air spray. Do not desiccate the tooth substance – cavity surface should be moist.
5. Use cotton rolls or rubber dam to isolate the cavity from contamination.

3.3 Placement of matrix

1. In Class II cavities place a matrix (e.g. AutoMatrix® or Palodent® matrix system) and wedge. Burnishing of the matrix band will improve contact and contour. Pre-wedging or BiTine® ring placement is advocated.
2. Use a dead soft, thin matrix band. Contact surface of steel matrix can be coated with a water soluble or petroleum lubricating jelly.
3. In Class V cavities cervical matrices may be used.

3.4 Pulp protection

1. In deep cavities cover the dentin close to the pulp (less than 1 mm) with a hard-setting calcium hydroxide liner (Dycal® calcium hydroxide liner, see complete Instructions for Use) leaving the rest of the cavity surface free for bonding.

3.5 Application

Time after activation

Mixing	15 seconds unless stated otherwise in Table 1
Working time	1 minute 30 seconds
Waiting time before further manipulation/finishing	6 minutes

3.5.1 Activation of the capsule

1. Activate the capsule by pressing the capsule onto a stable surface and depressing the plunger to its limit (at this position the plunger will overlap by about 2 mm).

3.5.2 Mixing

1. **Immediately** place the activated capsule in a capsule mixer (4000-4600 oscillations/minute) and mix for **15 seconds unless stated otherwise in Table 1**:

Table 1: Recommended mixing times² with standard capsule mixers³ (4000-4600 oscillations/minute).

Trademark/Manufacturer	Min (sec)	Max (sec)
Capmix Amalgam Mixer⁴/3M ESPE	12	15
ProMix Amalgamator/Dentsply Sirona	15	15
Rotomix⁴ – without centrifugation (2850 rpm)/3M ESPE	12	15
Rotomix⁴ – with centrifugation (3 sec) (2950 rpm)/3M ESPE	10	10
Silamat S6⁴/Ivoclar Vivadent	10	15
Softly Capsule Amalgamator⁴/Satelec	15	15
Ultramat 2⁴/SDI	12	12
Unodent Capsule Mixer⁴/Dental Directory	15	15

3.5.3 Application of ChemFil® Rock restorative

1. Immediately remove the capsule from the capsule mixer and place into the Capsule Extruder 2. The capsule may be rotated 360° to gain the proper angle of entrance into the cavity. Do not apply excessive force.
2. Click the trigger of the Capsule Extruder 2 until glass ionomer paste is seen through the clear nozzle. **Immediately start dispensing.** Begin dispensing at the deepest part of the cavity, keeping tip close to cavity floor. Gradually withdraw tip as cavity is filled. Avoid lifting the tip out of dispensed material while dispensing to minimize air entrapment. At the completion of dispensing, wipe tip against cavity wall while withdrawing from the operative field.
3. Start packing, excess removal and contouring **immediately after placement.**
4. After application, press the release button of the Capsule Extruder 2 until the push rod releases. Slightly rotating the capsule, the empty capsule can easily be removed.

3.5.4 Working and setting time⁵

- After activation the working time is **1 minute 30 seconds.**
- Wait at least **4 minutes 30 seconds after end of working time** before further manipulation.

Gross contamination during placement (cavity full of rinsing water) and setting (restoration flooded by saliva).

Dehydrated or diluted glass ionomer restorative.

1. Do not use high-volume suction close to restoration to prevent desiccation.
2. Prevent saliva contamination during application and early setting.

² Internal data.

³ Device available in 2012.

⁴ Not registered trademarks of Dentsply Sirona, Inc.

⁵ Setting time according to ISO-9917-1:2007 is 6 minutes.

3.5.5 Finishing

1. Start finishing no earlier than **4 minutes 30 seconds after end of working time** (i.e. 6 minutes after activation) with cups, discs, and points (Enhance® Finishing System).

4 Hygiene

Cross-contamination.

Infection.

1. Do not reuse single use products. Dispose in accordance with local regulations.
2. Reprocess reusable products as described below.

4.1 Capsule Extruder 2

NOTICE

Wrong cleaning or disinfection method.

Damage to the Capsule Extruder 2.

1. Do not submerge into disinfection solutions.
2. Do not ultrasonic clean.
3. Start reprocessing within 1 hour after use.

4.1.1 Manual cleaning

1. Right after use, remove capsule from the Capsule Extruder 2 and discard capsule according to local regulations.
2. Clean Capsule Extruder 2 using a soft brush and running cold tap water.
3. Discard gloves according to local regulations and disinfect hands with an appropriate bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations. Use according to disinfectant solution manufacturer's Instructions for Use.
4. Use a new clean pair of examination gloves.
5. Clean Capsule Extruder 2 with a soft brush, tap water, and liquid soap until free of visible contamination. Pay special attention to device seams and insertions.
6. Remove cleaning solution residue with a cloth soaked with tap water.
7. Dry Capsule Extruder 2 with a lint-free, single-use cloth.

The Capsule Extruder 2 can be manually cleaned for up to 1000 times.

4.1.2 Manual disinfection

1. After cleaning [4.1.1 Manual cleaning] thoroughly wipe all surfaces of the Capsule Extruder 2 with a single-use cloth in combination with an aldehyde- and alkylamine-free, alcohol-based, bactericidal, virucidal and fungicidal disinfection solution (e.g.: BBraun Meliseptol® Rapid⁶) approved according to local regulations and use according to disinfectant solution manufacturer's Instructions for Use. Make sure disinfectant solution is compatible with cleaning solution. Pay special attention to Capsule Extruder 2 seams and insertions.

⁶ Not a registered trademark of Dentsply Sirona, Inc.

2. After 1 minute remove disinfectant solution residue with a cloth soaked with deionized water.
3. Dry Capsule Extruder 2 with a lint-free single-use cloth.

The Capsule Extruder 2 can be manually disinfected for up to 1000 times.

4.1.3 Automatic washer-disinfector

As an alternative to manual cleaning and disinfection, the device can also be processed in a washer-disinfector⁷. Use only properly maintained, calibrated, and approved washer-disinfector according to ISO 15883-1.

1. Right after use, remove capsule from the device and discard capsule according to local regulations.
2. Clean Capsule Extruder 2 with a soft brush and running cold tap water.
3. Place device in the washer-disinfector allowing water and detergent to enter into and drain off from device openings during washer-disinfection program. Follow washer-disinfector's Instructions for Use.
4. Run washer-disinfection program with A0 value ≥ 3000 (e.g. 5 min at ≥ 90 °C), using appropriate detergents (e.g.: neodisher® MediClean [0.5%] (alkaline detergent) and neodisher® Z [0.1%] (acid neutralization and cleaning detergent); both detergents from Dr. Weigert, Hamburg, Germany⁸).

Device can be washer-disinfected for up to 200 times.

4.1.4 Autoclaving (optional)

After cleaning [4.1.1 Manual cleaning] and disinfection [4.1.2 Manual disinfection] the device can also be autoclaved.

1. Steam autoclave device unwrapped at 134 °C/2 bar = 273 °F/29 psi for 3 minutes 30 seconds.

Device can be autoclaved for up to 1000 times.

4.1.5 Lubrication

1. Regularly apply dental instrument lubricant.

Do not continue to use damaged or soiled Capsule Extruder 2.

5 Lot Number () and expiration date ()

1. Do not use after expiration date. ISO standard is used: "YYYY-MM"
2. The following numbers should be quoted in all correspondence:
 - Reorder number
 - Lot number
 - Expiration date

© Dentsply Sirona 2016-11-03

[These Instructions for Use are based on Master Version 16]

⁷ For example Miele Washer-Disinfector G7835 CD using program Vario TD.

⁸ Not registered trademarks of Dentsply Sirona, Inc.

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Dentsply DeTrey GmbH
Distributed in U.S.A by