# Dispersalloy®

## Dispersed Phase Alloy

### TABLETS AND POWDER

DENTSPLY

#### DIRECTIONS FOR USE — ENGLISH

Caution: For dental use only. USA: Rx only.

#### 1. PRODUCT DESCRIPTION

Dispersalloy® Alloy is a dispersed phase admix amalgam, containing lathe-cut particles and silver/copper eutectic spheres.

When combined with dental mercury as directed, it contains 50% mercury by weight.

##### 1.1 Delivery forms

Dispersalloy® Alloy is available in:

- 150 gram bottled powder in Fast Set or Regular Set
- 389mg pre-dosed tablets in Fast Set or Regular Set

##### 1.2 Composition

	Powder	Tablets
Silver	69.5 %	.270g
Tin	17.7 %	.069g
Copper	11.8 %	.046g
Zinc	1.0 %	.004g

A 1:1 mercury/alloy ratio is recommended. When dosed and mixed properly, Dispersalloy® Alloy contains 50% mercury by weight.

##### 1.3 Performance data

Compressive Strength (24hrs).....	>350MPa
Dimensional Change during hardening (%) .....	-0.1% – 0.15%
Working Time (min) .....	Fast Set: 2'30" – 3'30" Regular Set: 3'30" – 4'30"

##### 1.4 Indication for use

Dispersalloy® Alloy is designed for use in stress bearing restorations (Class I and II), when other restorative materials or restoration techniques are not indicated.

##### 1.5 Contraindication

The use of amalgam is contraindicated in patients with known allergies to mercury or any of the components.

##### 1.6 Compatible adhesives

Dispersalloy® Alloy is compatible with DENTSPLY adhesives and adhesive liners designed for use with dental amalgam materials. For details, see complete directions for use of the selected adhesive system. The use of other dentin and enamel adhesive systems is at the discretion and sole responsibility of the dental practitioner.

#### 2. GENERAL SAFETY NOTES

Be aware of the following general safety notes and the special safety notes in other chapters of these directions for use.

##### Safety alert symbol

This is the safety alert symbol. It is used to alert you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury.

##### 2.1 Warnings

1. When mixed and used as designed, this product will contain mercury. Properly mixed Alloy amalgam products contain mercury, which can cause skin sensitization or other allergic reaction. Although the mercury in this product is bound within the amalgam following trituration, proper care should be taken to prevent exposure to mercury. These preventive measures include the wearing of safety glasses and gloves, good ventilation, the use of an enclosed amalgamator, and the use of a mercury absorbing compound in the event of spillage. It has been suggested by the Ad-hoc working group established by the European Commission that the following information, be kept in mind when considering the use of dental amalgams.
  - After placement or removal of dental amalgam restorations increased mercury concentration in blood and urine has been observed. According to available scientific knowledge this increase has not been associated with any adverse health effects.
  - If placed in close contact with other metal restorations galvanic effects may occur. In most cases they will be of short lasting duration. If the effect persists the user should consider replacement of the dental amalgam filling with another material.
  - There are no proven adverse effects on the fetus associated with the placement or presence of dental amalgam fillings in the mother. It is sensible, however, where clinically feasible, to minimize health interventions during pregnancy and avoid any unnecessary chemical exposure of the fetus. This precaution should be observed with the use of all dental materials.

It has also been recommended by certain regulatory authorities that consideration be given to the use of amalgam fillings in children and that amalgams not be used in patients with severe renal disease.

##### 2. Health Canada Warnings

- Non-mercury filling materials should be considered for restoring the primary teeth of children where the mechanical properties of the material are suitable.
  - Whenever possible, amalgam fillings should not be placed in or removed from the teeth of pregnant women.
  - Amalgam should not be placed in patients with impaired kidney function.
  - In placing and removing amalgam fillings, dentists should use techniques and equipment to minimize the exposure of the patient and the dentist to mercury vapor and to prevent amalgam waste from being flushed into municipal sewage systems.
  - Dentists should advise individuals who may have allergic hypersensitivity to mercury to avoid the use of amalgam. In patients who have developed hypersensitivity to amalgam, existing amalgam restorations should be replaced with another material where this is recommended by a physician.
3. Properly mixed Alloy amalgam products contain mercury, which can cause skin sensitization or other allergic reaction. Proper care should be taken to prevent exposure to mercury.
    - Avoid skin Contact with this product by wearing protective safety glasses, clothing and gloves. If contact with skin occurs immediately wipe off and flush with generous amounts of water, then wash well with soap and water after contact. If skin rash, irritation, sensitization or other allergic reaction occurs, discontinue use and seek medical attention immediately.
    - Avoid eye contact as product may be irritating to eyes. Before using this product wear protective glasses as well as covering the patient's eyes to protect from excess. The use of an enclosed amalgamator is recommended. In case of accidental contact with eyes, rinse eyes immediately with plenty of water and seek medical attention.
    - Avoid ingestion of this product. Do not swallow or take internally. Whenever possible, the use of protective barriers such as rubber dam isolation and protective mask during placement and removal is recommended. If accidental swallowing occurs, drink lots of water. If nausea or illness develop, seek medical attention immediately. Contact regional Poison Control Center immediately.
    - Avoid inhalation as this product may be harmful if vapors are inhaled. Use with adequate ventilation. Whenever possible, the use of protective barriers such as rubber dam isolation, high-volume evacuation and protective mask during placement and removal is recommended. Give oxygen or artificial respiration if necessary.
  4. This alloy contains Zinc; amalgam made therefrom may show excessive expansion if moisture is introduced during mixing and condensing.

##### 2.2 Precautions

1. This product is intended to be used only as specifically outlined in the Directions for Use. Any use of this product inconsistent with the Directions for Use is at the discretion and sole responsibility of the practitioner.
2. Wear suitable protective eyewear, clothing, mask and gloves. Protective eyewear, and whenever possible, barrier techniques such as rubber dam is recommended for patients. (See Warnings)
3. Devices marked "single use" on the labeling are intended for single use only. Discard after use. Do not reuse.
4. Mercury reacts with and embrittles particular metals and their alloys. Avoid unnecessary contact between mercury and those metals (and their alloys).
5. Mercury presents a health hazard if incorrectly handled. Mercury is toxic by vapor inhalation and the effect is cumulative. Spillages of mercury should be removed immediately, including places which are difficult to access. Use a plastic syringe to draw it up. Smaller quantities can be covered by sulfur powder and removed. Individual small droplets can be picked up by tin (i.e. Sn) foil and removed. Avoid inhalation of the vapor. During and after a clean-up thoroughly ventilate the area where the spill has occurred. Vacuum cleaners should not be used.
6. Regulations for disposal must be observed. Adherence to the American Dental Association's current "Best Management Practices For Amalgam Waste (BMPs)" and "Dental Mercury Hygiene Recommendations" is strongly recommended. Among these practices:
  - Salvage and store non-contact amalgam scrap in well-sealed containers and recycle.
  - Salvage and store contact amalgam pieces from restorations, traps and filters in well-sealed containers and recycle.
  - Do not dispose of amalgam scrap, waste or extracted teeth in regulated infectious waste containers that will be incinerated, or in regular garbage.
  - Recycle used disposable amalgam capsules.
7. Waste material and all primary containers that have held mercury shall be disposed of following appropriate management practice.
8. DISPERSALLOY® ALLOY CONTAINS ZINC; If moisture is introduced into the dental amalgam before it has set, properties such as strength and corrosion resistance can be affected adversely. If the alloy contains zinc, such contamination can result in an excessive expansion (delayed expansion). Use a dry field, whenever it is possible.
9. Manufacturer's variation in amalgamators and differences in local electrical current may necessitate adjusting trituration time to obtain an optimal mix.

10. Insufficient data exist to support the use of amalgamators not listed in the chart below. Use of amalgamators other than those listed may result in less than optimally triturated amalgam. The operator must investigate non-listed equipment compatibility and/or consult amalgamator manufacturer for recommendations.
11. Do not attempt to place improperly triturated amalgam, or amalgam that has exceeded working time (dry, crumbly).
12. Drilling, polishing and grinding dental fillings should always be combined with water-cooling and suction under a vacuum.
13. Do not place dental amalgam in direct contact with other types of metals to reduce risk of corrosion and galvanic reaction.
14. The following precautions are required by the General Director AFSSAPS (Agence française de sécurité sanitaire des produits de santé – French agency for Sanitary Security of Health Products):
  - Store amalgam capsules in a cool and ventilated space.
  - Work in ventilated rooms with decontaminable non-textile surfaces.
  - Realize shaping and polishing of amalgam always under cooling and suction of the operation field.
  - Condense amalgam with classical methods (plugger) and do not use ultrasound condensers.
  - Do not place amalgam in direct proximity of other metallic restorations in order to avoid all risk of corrosion.
  - Avoid placing and removing amalgam during pregnancy and lactation.
  - If local reactions, in particular lichenoid lesions in the proximity of amalgam, occur, it is justified to remove the filling.

### 2.3 Adverse reactions (when mixed with mercury as intended)

1. Product may irritate skin, eyes. Skin contact: irritation or possible allergic response. Reddish rashes may be seen on the skin. Eye contact: irritation and possible corneal damage. (See Warnings and Precautions)
2. Inhalation of vapors may cause varying degrees of damage to the affected tissue and also increased susceptibility to respiratory illness. (See Warnings and Precautions)
3. Product may cause neurotoxic, nephrotoxic or other serious health effects if inhaled or ingested. (See Warnings)

### 2.4 Storage

Dispersalloy® Alloy powder or tablets should be stored in a well ventilated place at temperature not to exceed 25°C/77°F. Do not allow exposure to moisture or excessive humidity. Do not use after expiration date.

## 3. STEP-BY-STEP INSTRUCTIONS

1. Treatment of the preparation: Clean freshly instrumented enamel and dentin with water spray and then air dry. The use of pulpal protection, bases and/or cavity liners is at the discretion of the operator, as dictated by the clinical presentation. If desired, Dispersalloy® Alloy may be placed over an adhesive liner or adhesively bonded by following the instructions contained in the compatible Adhesive System (available separately).
2. Proportions: As with all amalgam alloys, the ratio of mercury to alloy is important. When using either Dispersalloy® Tablets or Powder, a 1:1 mercury/alloy ratio is recommended. Use of an automatic powder and mercury proportioner is strongly recommended. It will automatically and precisely dispense measured amounts of mercury and alloy. Refer to the instructions provided by the proportioner manufacturer.
3. Trituration: Trituration speed and time is an important factor in preparing the amalgam, in order to achieve proper amalgam mix consistency. Use of the DENTSPLY Rinn® Crescent® Wig-L-Bug® Screw Type Capsule and Pestle is recommended. The chart that follows provides guidelines in determining amalgamator trituration time and speed setting for Dispersalloy® Alloy:

Approximate Trituration Time Range (in seconds)

Tablets	Amalgamator	Setting	Fast Set		Regular Set	
			1 Tablet	2 Tablets	1 Tablet	2 Tablets
	Caulk ProMix®	Turtle	14±3	14±3	14±3	12±3
	Caulk Vari-Mix IIM	M2	14±3	14±3	14±3	12±3
	Caulk Vari-Mix III	M2	14±3	14±3	14±3	12±3
	ESPE CapMix*	--	5±1	4±1	NR	NR
	Wig-L-Bug DS80	High	13±2	13±2	13±2	9±2
	Wig-L-Bug MSD	3800	16±3	16±3	14±3	12±3
	ADEC*	5	14±3	14±3	14±3	12±3
	Silamat*	4200	5±1	5±1	NR	NR
	Silamat Plus*	P-2 Slow	8±2	6±2	6±2	NR

Powder	Amalgamator	Setting	Fast Set			Regular Set		
			400mg	600mg	800mg	400mg	600mg	800mg
	Caulk ProMix®	Turtle	14±3	14±3	12±2	14±3	14±3	12±2
	Caulk Vari-Mix IIM	M2	14±3	14±3	12±2	14±3	14±3	12±2
	Caulk Vari-Mix III	M2	14±3	14±3	12±2	14±3	14±3	12±2
	ESPE CapMix*	--	5±1	5±1	5±1	5±1	5±1	5±1
	Wig-L-Bug DS80	High	14±3	14±3	12±2	14±3	14±3	12±2
	Wig-L-Bug MSD	3800	14±3	14±3	12±2	14±3	14±3	12±2
	ADEC*	5	14±3	14±3	14±3	14±3	14±3	14±3
	Silamat*	4200	5±1	5±1	5±1	5±1	5±1	5±1
	Silamat Plus*	P-2 Slow	7±1	6±1	6±1	7±1	6±1	6±1

\* Not products of DENTSPLY International

- The triturated amalgam should be bright and have a plastic consistency. If the amalgam is excessively splashy and wet looking, INCREASE trituration time or amalgamator speed. Amalgam that appears excessively dry indicates trituration time or amalgamator speed should be DECREASED.
- Suggestions for obtaining optimum mix: When performing adjustments, it is recommended to adjust trituration time in 2 second increments, until optimum mix is obtained. Greater adjustments may indicate amalgamator incompatibility or compromised functionality.

Decrease trituration time: - to lengthen working time.  
- if mix is too hot.  
- if mix is too dry.

Increase trituration time: - to shorten working time.  
- if mix is too wet.

- Properly triturated Dispersalloy® Amalgam does not require mulling to produce a smooth plasticity; however you may mull for 1-2 seconds after removing pestle to collect amalgam into a single mass.
4. Condensation: Dispersalloy® Alloy is now ready for immediate placement and condensation. A gentle tap may be required to free amalgam from the capsule. Do not attempt to manually express mercury from the triturated mass prior to placement.
    - Condense immediately after mix is completed. Use a clean amalgam carrier to transfer increments to the cavity. Pack each increment into angles and undercuts with small-faced plugger and sufficient pressure to insure good adaptation. Build the restoration with additional portions, until the cavity is slightly overfilled. Remove any mercury-rich amalgam from the surface that may develop during condensation.
  5. Carving/Burnishing: Carving can begin immediately after condensation. A sharp carver will give best results. The anatomy should be carved proceeding from the proximal margins towards the center of the restoration. Before the matrix band is removed, the marginal ridge should be supported by a condenser. After removal of the matrix band, gingival margins may be smoothed with a curved explorer. Burnishing setting amalgam is recommended to improve surface smoothness.
  6. Polishing: The quality of the Dispersalloy® Alloy restoration will be enhanced by polishing. The amalgam should be allowed to set 24 to 48 hours before polishing. See selected polishing instrument manufacturer's directions for use.

## 4. HYGIENE

- Properly dispose of the amalgam scrap in accordance with local regulations.

## 5. LOT NUMBER AND EXPIRATION DATE

1. Do not use after expiration date. ISO standard is used: "YYYY/MM"
2. The following numbers should be quoted in all correspondence.
  - Reorder number
  - Lot number on package
  - Expiration date on package

Manufactured by  
DENTSPLY Caulk  
38 West Clarke Avenue  
Milford, DE 19963 USA  
Tel.: 1-302-422-4511  
www.dentsply.com

DENTSPLY DeTrey GmbH  
De-Trey-Str. 1  
78467 Konstanz  
Germany  
Tel.: (49) (07531) 583-0

Distributed by  
DENTSPLY Canada  
161 Vinyl Court  
Woodbridge, Ontario  
L4L 4A3 Canada

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