Fynal® Kronen- und Brückenzement

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For dental use only. USA: Rx only.

1. PRODUCT DESCRIPTION

Fynal® Crown & Bridge Cement is a zinc oxide-eugenol composition cement designed for the final cementation of finished metal and PFM crowns, bridges, and inlays.

1.1 Delivery forms

Fynal® Cement is available in: powder and liquid

1.2 Composition

Powder: Zinc oxide; Polymethylmethacrylate; Zinc Acetate; Yellow Iron Oxide pigment

Liquid: Eugenol; Acetic acid

1.3 Indications

Luting of Indirect Restorations

- Final cementation of all metal crowns and bridges, onlays and inlays including precious, semi-precious and non-precious metals.
- Final cementation of conventional porcelain fused to metal crowns and bridges with metal margins.

1.4 Contraindications

- Fynal® Cement is contraindicated for use with patients who have a known hypersensitivity or severe allergic reaction to eugenol or any of the components.
- Fynal® Cement is contraindicated for direct application to dental pulp tissue (direct pulp capping).

1.5 Compatible materials

Use of chemical agent dentin/enamel cavity cleansers, desensitizers, sealing and/ or dentin/enamel adhesive systems is not required with Fynal® Cement. Any use of such agent(s) in conjunction with Fynal® Cement is at the discretion and sole responsibility of the dental practitioner.

2. GENERAL SAFETY NOTES

3. STEP-BY-STEP INSTRUCTIONS FOR USE

3.1 Preparation of the Restoration

Internal surfaces of restorations should be clean and dry prior to cementation. Internal surface microetching (sandblasting with 50µ alumina) of metal surfaces of the restoration is recommended.

3.2 Preparation of the Tooth

Remove temporary restoration and excess temporary cement using an explorer, a rubber cup and a prophy paste or water/flour of pumice. Rinse thoroughly and carefully blot dry preparations with cotton. Avoid prolonged air drying. Isolate field of operation in usual manner. Avoid contamination. Etching of tooth surfaces is NOT recommended.

3.3 Dispensing

- Thoroughly tumble the powder bottle before dispensing to assure uniform bulk density of contents. Fill provided measuring scoop to excess without packing—then level off scoop with spatula. Place powder onto the mixing pad.
- Using the dropper provided, dispense use eight (8) drops of liquid onto the mixing pad. Keep powder and liquid separated until ready for spatulation. After dispensing liquid, replace original cap immediately to prevent evaporation and subsequent contamination.

NOTE: Premature setting of material. Keep powder and liquid separated until ready for spatulation.

Technique Tip: Do not add water to accelerate set time. Setting is accelerated by higher temperatures and humidities found in the mouth or during mixing.

- The powder/liquid ratio is 2.5-3.0g powder to 1mL liquid. Depending upon the size and number of appliances to be cemented, the mix may be increased by like powder/liquid increments. Do not use more liquid than recommended; the result will be cement of reduced strength. However, more powder may be added if a thicker consistency is desired; the result will be cement mixes with higher strength, but will increase cement film thickness and may impeded complete seating of the restoration.

3.4 Mixing

In mixing, use any spatulation technique which quickly combines the powder and liquid. After powder and liquid have been thoroughly incorporated, “strop” the mix vigorously for 5 to 10 seconds over a large area of the mixing pad. This type of mix results in a working consistency that is smooth and non-runny. The mix should be completed in less than one (1) minute.

3.5 Seating the Restoration

- At room temperature, Fynal[®] Cement offers a work time of up to 2 minutes on the mixing pad. Excess Fynal[®] Cement left on the mixing pad will normally take 6 minutes or more to set.

- Apply a generous layer of mixed cement to the internal surface of the restoration. It is not necessary to apply cement to the abutment. Seat the restoration in the mouth. Verify complete seating.

- Remove gross excess from marginal areas. Use an instrument such as a blunted explorer or periodontal probe. Restoration should not be moved or torqued during removal of gross excess cement. Special attention should be paid to interproximal areas using floss to remove excess cement.
- Following placement, Fynal[®] Cement will self-cure set in the mouth in approximately 5 minutes. Protect restoration from contamination and movement during the setting time.
- Apply moderate and consistent pressure to the restoration throughout the set time of 5 minutes from start of mix. Burnishing, if applicable, should be completed before initial set. Following the set, check and adjust occlusion as necessary.

4. HYGIENE

Cross-contamination

Use protective barrier or touch bottle only with non-contaminated gloves or hands. If not protected keep bottle out of spray of bodily fluids.

4.1 Cleaning and disinfection instructions

Use of protective barriers is recommended to avoid package contamination. Repeated disinfection may cause damage to the labels.

Cleaning And Disinfection Of Powder Measure Scoop

The powder (scoop) measure may be cleaned by scrubbing with hot water and soap or detergent.

- Disinfect scoop, if exposed spatter or spray of body fluids, contaminated hands, or oral tissues with a hospital-level disinfectant solution according to national/local regulations. Use of chlorine- (bleach), glutaraldehyde-, phenolic-, iodophor- or organic solvent (e.g. alcohol) based or containing products may over time affect the scoop.
- Follow the disinfectant manufacturer’s directions for optimum results.
- After disinfection, thoroughly rinse and dry the scoop before storage.
- Do not autoclave the scoop.

5. LOT NUMBER AND EXPIRATION DATE

- Do not use after expiration date. ISO standard uses: “YYYY/MM.”
- The following numbers should be quoted in all correspondences:
 - Reorder number
 - Lot number on package
 - Expiration date

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