6. Application of Matrix and Wedge

- Grasp the matrix with the tweezers using the hole located on the occlusal tab. If the ring was placed properly before tooth preparation, then the matrix should slide easily into place adjacent to the space to be restored from the occlusal approach. Ensure matrix is apical to preparation margin with no gap. **Technique Tip:** If WedgeGuard wedge prevents matrix from fully seating gingivally, slide wedge out slightly. Place matrix, then reposition wedge.
- Close the gingival margin by placing a contoured wedge or anatomic wooden wedge. **Palodent® Plus** wedges contain a placement tab with hole, which is securely grasped by the pin tweezers. This assures secure placement and minimizes the likelihood of dropping small accessories. **Technique Tip:** Palodent® Plus wedges' hollow design allows for easy placement of a second wedge from the opposing side (if needed) without disploding the first. **Technique Tip:** Alternatively, the fork design allows placement of the Palodent® Plus retaining rings if desired. The wedge can then be easily moved through the fork line without disploding the rings.
- Replace the Palodent® Plus retaining ring in the proximal plane, engaging the matrix and the wedge, thus stabilizing it against tooth traction. The forks of the ties should be sitting over the wedge.
- Lightly burnish the matrix against the adajcent tooth both buccally and lingually to further adapt it to form proper contours. **NOTE:** Over burnishing may distort matrix and compromise contours. The sin tweezers has a built-in ball burnisher for this purpose. The matrix placement tab may be bent over the adjacent marginal ridge to facilitate restoration marginal ridge contour. **Technique Tip:** When restoring an MOD preparation, two rings may be placed facing opposite directions, one mesially and one distally or both pointing in the same direction, usually mesially, one over the other placing the mesial matrix and ring first.
- Placement of Restorative Material

Place restorative material and establish proximal contact as per usual technique. Light cure and/or allow material to reach initial set. Goo contouring may be accomplished with matrix in place.
- Matrix Removal

Remove ring(s) using forceps, remove wedge(s) and sectional matrices using pin tweezers. Finish contouring and verify appropriate proximal contact. **Technique Tip:** It is advisable to "post-cure" light cured restorative materials following matrix removal from all surfaces, including proximo-facial and lingual line angles.

4. HYGIENE

Cross-contamination

- Do not reuse single use products. Dispose of in accordance with local regulations. During incineration, Wedges and WedgeGuards may produce irritating and highly toxic gases.
- WedgeGuards may produce irritating and highly toxic gases.
- Reprocess reusable products as described below.

Instructions for Disinfecting/Sterilizing the Palodent® Plus Placement forceps, pin tweezers and retaining rings

- Warnings
- High level disinfection by itself is not appropriate for the Palodent® Plus placement forceps, pin tweezers and retaining rings.
 - It is not recommended to submerge Palodent® Plus placement forceps, pin tweezers and retaining rings into disinfection solutions.
 - Do not allow the device to be exposed to 134°C.
 - The Palodent® Plus sectional matrix band, Wedges and WedgeGuards are intended for single use only. Do not attempt to reprocess and/or re-use. Properly dispose Wedges, WedgeGuards and matrices with other contaminated sharps.
 - WedgeGuards and the use-and/or contaminated matrices Wedges or WedgeGuards in accordance with local regulations. During incineration, Wedges and WedgeGuards may produce irritating and highly toxic gases. Ensure that phosphoric acid etchants are removed from the retaining ring ties prior to steam sterilization.

Limitations on Reprocessing	<ul style="list-style-type: none">The Palodent® Plus placement forceps, pin tweezers and retaining rings can be subjected to the cleaning and automated disinfection up to 500 times, and sterilization process up to 1000 times. Repeated processing has minimal effect on the Palodent® Plus placement forceps, pin tweezers and retaining rings. Cold liquid disinfectant/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use.
-----------------------------	---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

Point of Use	<ul style="list-style-type: none">Remove excess soil with disposable cloth/paper wipe. Most ultrasonic cleaning solutions will not affect the Palodent® Plus rings, although chemicals in some cleaners may cause tire discoloration. The discoloration is only cosmetic, but if that is a concern put the rings in a beaker with a water/detergent solution and place that in the ultrasonic cleaner to prevent contact with chemicals. It is recommended that the Palodent® Plus placement forceps, pin tweezers and retaining rings are reprocessed as soon as is reasonably practical following use.
Containment & Transportation	<ul style="list-style-type: none">No particular requirements.
Cleaning & Disinfection- Automated:	<ul style="list-style-type: none">Use only a properly maintained, inspected, calibrated, and approved washer/disinfecter (according to ISO 15883). Perform a cleaning program cycle (the following cycle is recommended): <ul style="list-style-type: none">-1 minute pre-wash/rinse in cold tap water. - 5 minute wash with enzymatic detergent at 55°C. - 1 minute enzymatic neutralization, - 1 minute rinse in deionized water. - Properly dry in autoclave. - Thermal disinfection with minimum A0 of 3000 (eg. 5 minutes at 90°C). - 30 minutes drying at 60°. Follow manufacturer’s recommendation for use of detergent and neutralizer, observing concentrations and contact times.
Cleaning & Disinfection- Manual:	<ul style="list-style-type: none">As an alternative to automated cleaning and disinfection, the Palodent® Plus placement forceps, pin tweezers and retaining rings have to be cleaned by scrubbing with hot water and soap or detergent. - Use a new clean pair of examination gloves. - After clean of thoroughly wash all device surfaces with a single-use cloth in combination with an alcohol-based, bactericidal, virucidal and fungicidal disinfection solution approved according to local regulations and use according to disinfectant solution manufacturer’s Instructions for Use. Make sure disinfectant solution is compatible with cleaning solution. Pay special attention to device seams and insertions. - Remove disinfectant solution residue with a cloth soaked with water. - Discard gloves according to local regulations and disinfect hands with an appropriate bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations. Use according to disinfectant solution manufacturer’s instructions for Use. - Dry device with a lint-free single-use cloth.
Packaging Sterilization	<ul style="list-style-type: none">Paper/plastic steam sterilization pouches may be used, but are not required. Use a steam autoclave. Do not exceed 134°C. Steam autoclaving – Full cycle: wrapped or unwrapped at 134°C, 2 bar/273F, 29 psi for 3 minutes.
Drying	<ul style="list-style-type: none">Use the drying cycle of the autoclave. Do not exceed 134°C. Allow the Palodent® Plus placement forceps, pin tweezers and retaining rings to fully dry before storage. Avoid leaving the Palodent® Plus placement forceps, pin tweezers and retaining rings in an autoclave overnight or for long periods, as this may cause staining from the autoclave itself.
Maintenance	<ul style="list-style-type: none">If the Palodent® Plus placement forceps, pin tweezers or retaining rings are damaged, worn, or distorted they should be discarded. Before use, visually inspect the ties for damage, such as cracking or splitting. The Palodent® Plus forceps and pin tweezers require no additional maintenance and should not be oiled or greased. If a Palodent Plus retaining ring is deformed by over stretching during placement or removal, the ring should be discarded.
Storage	<ul style="list-style-type: none">Store the Palodent® Plus placement forceps, pin tweezers and retaining rings at room temperature, away from moisture and excess humidity. Palodent® Plus placement forceps, pin tweezers and retaining rings should be disinfected/sterilized and dried prior to storage. Sterilization of new matrices, Wedges and WedgeGuards pre-use (if desired) may be accomplished (one time) by steam autoclaving, 134°C, 3 minutes minimum prior to storage and/or use.
Manufacturer Contact	<ul style="list-style-type: none">Within the United States, call Dentsply Sirona at 1-302-422-4511. For areas outside the United States, contact your local Dentsply Sirona representative.

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5. LOT NUMBER AND EXPECTED USEFUL LIFE

1. The following numbers should be quoted in all correspondence:

- Recorder Number
- Lot number on the package
- Copy of proof of purchase

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