Dentsply Sirona

Regisil® Rigid Vinyl Polysiloxan Bissregistrierungsmaterial

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INTERAZIONI

Mixing tray may be gently cured by slow access to the tooth preparation if desired. To dispense material, apply a slow, steady pressure to plunger (using palm or thumb, as preferred). Excessive force is not necessary. If resistance is encountered, or if excessive force is required, remove the syringe from the patient and check for plunger obstructions. If cartridge/mixing tip obstruction is suspected, remove cartridge/mixing tip assembly and replace, following steps above. When the tray is completely filled, remove the tray and attempt disassembly. To disassemble, dislodge the plunger, straighten mixing tip if bent, and tap the mixing tip against the counter to dislodge the cartridge/mixing tip assembly. Properly dispose used cartridge/mixing tip assembly. 7. Prepare the digi† Targeted Delivery System syringe for subsequent reuse by following instructions below for cleaning and disinfection/sterilization.

Interocclusal Bite Registration

Dispensing Material

- Instruct material directly onto the patient’s occlusal surfaces analy/or occlusion rms. The areas where teeth have been prepared should be injected first. Usually, only teeth of the upper or lower arch need be covered. However, in recording the anterior teeth, it is suggested that material be placed over the teeth of both arches. A layer of about 5mm of Rigid Bite Registration Material is usually sufficient. Do not exceed 30˚ while injecting. **Technique Tip:** This material may be used with or without a bite registration tray, but in most applications, a tray will not be required. In those situations where use of a tray is desired, the material should be placed on both sides of the tray to a depth of 2-3mm. This material has been found to work well with mesh fabric type trays.
- Gently guide or instruct patient to close into desired position. Due to the thixotropic nature, Regisil® Rigid Bite Registration Material offers no resistance to closure during the 30 seconds work time.
- Instruct the patient to maintain, or manually maintain jaw relationship without movement until 1 minute from start of mix. Check material in the mouth (not on the bench) to be sure it is firm, resilient, and non-tacky before removal.
- Removing Regisil® Rigid Bite Registration Material.**
 - After 1 minute from the start of mixing, the material may be removed from the patient’s mouth. Have the patient open and carefully release the bite registration from the teeth.
 - Check for residual material in undercut areas.
 - Rinse registration under cold water and blow dry.

Matrix for fabrication of provisional (temporary) restorations

Consult manufacturer’s complete Directions for Use of selected provisional material for vinyl polysiloxane matrix’s technique compatibility. Prior to preparation of the tooth/teeth, evaluate existing contours and anatomy. To duplicate existing anatomy proceed to matrix impression. If contours or anatomy are deficient, light cured composite may be placed and cured without bonding agent on the tooth/teeth as a mock up to approximate the desired contours. A missing tooth may be temporarily replaced with an acrylic denture tooth prior to matrix impression.

Matrix Impression

- Select and/or prepare a suitable quadrant metal tray, a firm quadrant disposable tray or a quadrant custom tray. Use rigid trays of sufficient size to provide at least 2-3mm thickness of matrix impression material.
- Dispense Regisil® Rigid Bite Registration Material directly into impression tray from cartridge. Do not layer the material or place it into rows. This technique ensures the least amount of air incorporation. **Technique Tip:** Loaded tray must be inserted within 30˚ (thirty seconds) from time of first dispensing.
- Seat loaded tray. **Tray must be seated within 30˚ (thirty seconds).** Retain matrix impression in position until firm and cured. (**NOTE:** The material sets firmer in the mouth than on the bench). Check matrix impression in the mouth (not on the bench) to be sure it is firm, resilient, and non-tacky before removal. The Minimum Removal Time is 1 minute from the start of mix. At 72°F (22°C) Regisil® Rigid Bite Registration Material has a minimum work time of 30 seconds. Variations in temperature from that stated above will affect work and set times. Higher temperatures reduce work times and lower temperatures increase them.
- Remove matrix impression by pulling slowly to break seal, then snap out along the long axis of the teeth. Check for residual material in undercut areas. Rinse matrix impression under cold water and blow dry.
- Complete preparations and fabricate provisional (temporary) restorations as the mixed provisional material manufacturer’s Directions for Use.

HYGIENE

CAUTION

CLEANING

The cartridge dispensing gun and Targeted Delivery System syringe may be cleaned by scrubbing with hot water and soap or detergent. The digi† Targeted Delivery System syringe plunger should be fully retracted. Excess material may be cleaned with alcohol-moistened gauze.

DISINFECTION

Disinfectors, cartridges and used tips left in place on cartridges for storage, exposed to splatter or spray of body fluids or that may have been touched by contaminated hands, or oral tissues, hospital-level disinfectant. Acceptable disinfectants are those that are EPA-registered as tuberculocidal, iodophors, sodium hypochlorite (5.25%), chlorine dioxide and dual or synergized quaternary ammoniums are approved disinfectants. Water-based disinfectant solutions are preferred. Use impregnated wyes not sprays.

Note: Wipe cartridge GENTLY, vigorous wiping may destroy the label.

Some phenolic based agents and iodophor based products may cause surface staining. Agents containing organic solvents, such as alcohol may tend to dissolve the plastic dispenser and plunger. The disinfectant manufacturer’s directions should be followed properly for optimum results.

Technique Tip: Unset Regisil® Rigid Bite Registration Material can be removed from clothing with dry cleaning solvent.

DISINFECTION AND/OR STERILIZATION

Follow following as outlined above, digi† Targeted Delivery System syringe exposed to splatter or spray of body fluids or that may have been touched by contaminated hands, or oral tissues, should be steam autoclaved following autoclave manufacturer’s recommendations.

Alternatively, the digi† Targeted Delivery System syringe may be disinfected with a hospital-level disinfectant as outlined above. Water-based disinfectant solutions are preferred. Some phenolic-based agents and iodophor-based products may cause surface staining. Agents containing organic solvents, such as alcohol may tend to dissolve the plastic barrel and plunger. The disinfectant manufacturer’s directions should be followed properly for optimum results.

NOTE: As with any plastic instrument, the digi† Targeted Delivery System syringe may weaken over time.

To reassemble, insert plunger into syringe barrel, and press components together. Prior to each use check to make sure that the digi† Targeted Delivery System syringe plunger is fully engaged and in good working order.

Disinfection of the Bite Registration Material

The registration should be disinfected with a hospital-level disinfectant. Acceptable disinfectants are those that are EPA-registered as tuberculocidal, iodophors, sodium hypochlorite (5.25%), chlorine dioxide, and dual or synergized quaternary ammoniums are approved disinfectants. To disinfect polyvinyl siloxane material, thoroughly soak by spraying or immersing the registration in any recommended hospital-level disinfectant except neutral glutaraldehyde for the contact time recommended by the disinfectant manufacturer for optimum results. Water-based disinfectant solutions are preferred.

Model Placement into Registration

The registration should be removed from the disinfectant, rinsed with water and exposed to air to dry for at least the same amount of time it was exposed to the disinfectant solution before it is used. Do not store registration in direct sunlight. If the registration is to be shipped, use suitable packaging to preclude distortion.

Any excess material extending onto soft tissue areas, into interproximal areas or gingival to the height of contour of the teeth should be removed to obtain maximum accuracy when seating the registration on the model. Best results are obtained by trimming with a sharp scalpel blade or fine, sharp scissors.

LOT NUMBER AND EXPIRATION DATE

- Do not use after expiration date. ISO standard uses: “YYYY/MM”
- The following numbers should be quoted in all correspondences:
 - Reorder number
 - Expiration on the cartridge
 - Lot number date

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