# Snap-A-Ray® Xtra

## Film & Phosphor Plate Holder

For anterior and posterior bisecting angle technique radiographs

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### REF

540501 Snap-A-Ray Xtra Holder (1)  
540503 Snap-A-Ray Xtra Holder (3)

**Posteriors Molar** Radiograph media can be angled for hard-to-reach areas.

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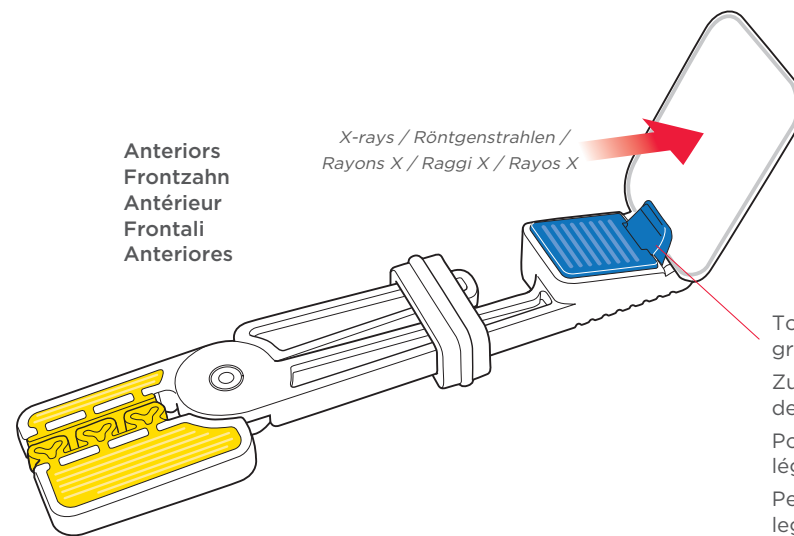
X-rays / Röntgenstrahlen /  
Rayons X / Raggi X / Rayos X

**Anteriors  
Frontzahn  
Antérieur  
Frontali  
Anteriores**

To insert film or phosphor plate, pull blue media grip forward slightly.

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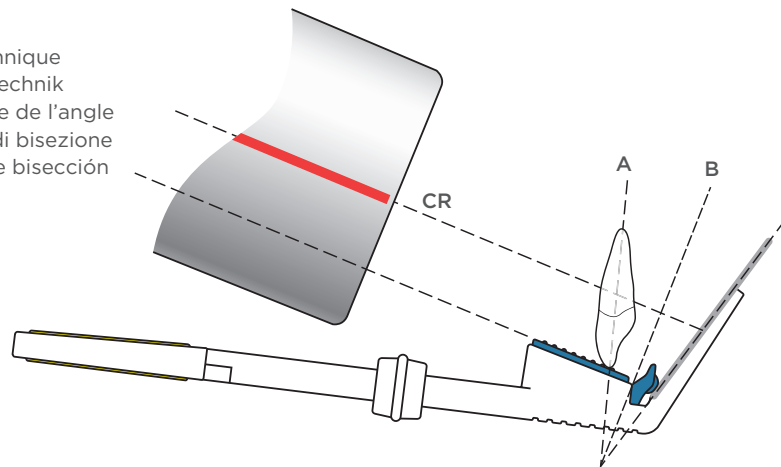
In most cases, the x-ray cone should be aligned parallel to the blue bite surface.

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**Caution:** U.S. Federal law restricts this device to be used by or on the order of a licensed dental professional.

**Indications for use:** Snap-A-Ray Xtra is indicated for holding film or PSP for intraoral dental radiographs.

**Contraindications:** None known.

**Warnings:** Sterilize before use. Only sterilize with steam autoclaving according to cleaning and sterilization instructions. It is the user's responsibility to qualify alternative methods of sterilization or disinfection. Failure to follow properly validated sterilization processes and approved aseptic techniques may result in cross contamination.

**Precautions:** Follow sterilization instructions carefully to prevent damage to plastic pieces during sterilization.

**Adverse reactions:** None known.

**Step-by-Step Instructions:**

1. Insert film or PSP according to positioning guide.
2. Position holder in patient's mouth and align x-ray tube.
3. Take exposure.
4. Adjust as needed for the next desired position.

**Reprocessing:** Disassemble instrument(s) and separate all parts prior to reprocessing.

**Manual Cleaning and Disinfection:**

1. Rinse components under running tap water using a soft-bristled brush to remove gross soil using an enzymatic detergent per manufacturer's recommendation, paying special attention to any cavities or holes.
2. Rinse under running tap water and dry with cloth or disposable wipe.
3. After thoroughly cleaning, wipe all surfaces with a disposable wipe in combination with a bactericidal, virucidal, and fungicidal instrument disinfection solution (such as VoloWipe) following manufacturer's instructions. Pay special attention to any cavities or holes.
4. Rinse components by wiping them with a cloth soaked in pure water or by running under running tap water.

**Automated Cleaning and Disinfection:**

As an alternative to manual cleaning and disinfection, the devices can also be processed in a washer-disinfector. Use only properly maintained, calibrated, and approved washer-disinfectors according to ISO 15883-1.

1. Place devices in washer-disinfector allowing water and detergent to wet all surfaces and to drain off afterwards during washer-disinfector program following manufacturer's instructions. Run a washer disinfectant program with A0 value  $\geq 600$ .

**Sterilization:**

1. Put components in sterilization pouches, and place in the middle tray of the autoclave, away from autoclave walls and heating element. Plastic parts must be in a separate pouch from metal arms to avoid melting or warping.
2. Cycle steam autoclave at 270°F (132° C) for 10 minutes at 30 PSI (206.8 kPa). Do not exceed 273°F (134°C).
3. To maintain sterility, instruments should remain bagged until ready for use.

**Storage:** Store at room temperature (18-25°C / 65-77°F).

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© 2017 Dentsply Sirona Preventive Printed in the USA 054901 Rev. 7 (0117)
Rinn® and Snap-A-Ray® Xtra are trademarks of DENTSPLY International Inc.