For better dentistry

DENTSPLY

SDR[®]

Smart Dentin Replacement

Posterior Bulk Fill Flowable Base

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Caution: For dental use only.

In the USA: Rx only

Content	Page
1 Product description.....	2
2 General safety notes	3
3 Step-by-step instructions	5
4 Hygiene.....	7
5 Lot number and expiration date.....	7

1 Product description

SDR[®] Posterior Bulk Fill Flowable Base is a one-component, fluoride-containing, visible light-cured, radiopaque resin composite restorative material. It is designed to be used as a base in class I and II restorations. It is also suitable as a stand-alone restorative material in non-occlusal-contact applications. **SDR[®]** material has handling characteristics typical of a “flowable” composite, but can be placed in 4 mm increments with minimal polymerization stress. **SDR[®]** material has a self-leveling feature that allows intimate adaptation to the prepared cavity walls. It is available in one universal shade. When used as a base/liner, it is designed to be overlaid with a methacrylate based universal/posterior composite for replacing missing occlusal/facial enamel.

1.1 Delivery forms

SDR[®] material is available in:

- Predosed Compula[®] tips for direct intra-oral application
- Small syringes for direct intra-oral application
- One universal shade

1.2 Composition

- Barium-alumino-fluoro-borosilicate glass
- Strontium alumino-fluoro-silicate glass
- Modified urethane dimethacrylate resin
- Ethoxylated Bisphenol A dimethacrylate (EBPADMA)
- Triethyleneglycol dimethacrylate (TEGDMA)
- Camphorquinone (CQ) Photoinitiator
- Photoaccelerator
- Butylated hydroxyl toluene (BHT)
- UV Stabilizer
- Titanium dioxide
- Iron oxide pigments
- Fluorescing agent

1.3 Indications

- Base and liner for direct Class I and II restorations.
- Restorative (without the need of an additional occlusal material) for direct, small class I restorations of permanent molars and for class I and II restorations of deciduous molars.
- Pit & Fissure sealant.
- Core Build-up.

1.4 Contraindications

SDR® material is contraindicated for:

- Use with patients who have a known hypersensitivity to methacrylate resins.

1.5 Compatible adhesives

SDR® material is used following application of a suitable dentin/enamel adhesive and is chemically compatible with conventional methacrylate-based dentin/enamel adhesives including DENTSPLY adhesives designed for use with visible light-cured composite restoratives (see complete Instructions for Use of selected adhesive).

1.6 Compatible enamel replacement restoratives

SDR® material is used in conjunction with a suitable universal/posterior restorative material* as an occlusal/facial enamel replacement (see Step-By-Step Instructions) and is chemically compatible with conventional methacrylate-based composite restorative materials including DENTSPLY visible light-cured universal/posterior composite restorative materials designed for posterior class I & II occlusal restorations (see complete Instructions for Use of selected occlusal restorative material).

* Enamel replacement material overlay is not required for Pit & Fissure Sealant, direct, small class I restorations of permanent molars, class I and II restorations of deciduous molars, or Core Buildup applications.

2 General safety notes

Be aware of the following general safety notes and the special safety notes in other sections of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

SDR® material contains polymerizable methacrylate monomers which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, red-dish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and alcohol and wash well thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

2.2 Precautions

1. This product is intended to be used only as specifically outlined in these Instructions for Use. Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.
2. Contact with saliva and blood during composite placement may cause failure of the restoration. Use of rubber dam or adequate isolation is recommended.
3. Wear suitable protective eyewear, mask, clothing and gloves. Protective eyewear is recommended for patients.
4. Devices marked "single use" on the labeling are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
5. SDR[®] material should extrude easily. **DO NOT USE EXCESSIVE FORCE.** Excessive pressure may result in unanticipated extrusion of the material or cause the Compula[®] tip to eject from the Compules[®] tips gun.
6. Use of Compula[®] tips with the DENTSPLY Compules[®] tips gun is recommended.
7. Tightly close syringes with original cap immediately after use. Do not allow dispensing tip to remain as a syringe cap.
8. Only use the dispensing tips supplied by DENTSPLY for placement of the material.
9. SDR[®] material is available in one, semi-translucent universal shade. If placed to the cavosurface margin, e.g., occlusal surface or wide proximal box extension cavities, the tooth/restorative demarcation may be visible. Heavily stained dentin may be visible through the cured SDR[®] material. Use of an appropriately shaded and/or opaque restorative material of adequate thickness in visible areas/surfaces is recommended where esthetics is paramount.
10. When placing SDR[®] material as a base or liner, allow at least 2 mm for selected occlusal restorative material.
11. Insufficient data exist to support the use as a class I restoration with isthmus width beyond central groove areas, or when replacing a functional cusp. Excessive wear or restoration failure may result.

12. Interactions

- Eugenol- and hydrogen peroxide-containing materials should not be used in conjunction with this product since they may interfere with hardening and cause softening of the polymeric components of the material.
- SDR[®] material is light-cured material. Therefore, it should be protected from ambient light. Proceed immediately once the material has been placed.
- If mineral-impregnated (e.g., ferric compounds) retraction cords and/or hemostatic solutions are used in conjunction with adhesive procedures, marginal seal may be adversely affected, allowing microleakage, subsurface staining and/or restoration failure. If gingival retraction is necessary, use of plain, non-impregnated cord is recommended.

2.3 Adverse reactions

Product may irritate the eyes and skin.

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

2.4 Storage conditions

Inadequate storage conditions will shorten the shelf life and may lead to malfunction of the product.

- Keep out of direct sunlight and store in a well ventilated place at temperatures between 2-24 °C/35-75 °F.
- Allow material to reach room temperature prior to use.
- Protect from moisture.
- Do not freeze.
- Do not use after expiration date.

3 Step-by-step instructions

3.1 Cavity preparation

1. Prepare the cavity so that no residual amalgam or restorative material is left.
2. Rinse surface with water spray and carefully dry it with air spray. Do not desiccate the tooth structure.
3. Use a dental dam or cotton rolls to isolate the cavity from contamination.

3.2 Placement of matrix

For optimal proximal contacts proceed as follows:

1. Place a matrix (e.g. AutoMatrix® matrix system or Palodent®/Palodent® Plus Sectional matrix system) and wedge. Burnishing of the matrix band will improve contact and contour. Pre-wedging ring placement is advocated.
2. In class II cavities use a deadsoft, thin matrix band.

3.3 Pulp protection, tooth conditioning/dentin pretreatment, adhesive application

Refer to adhesive manufacturer's instructions for pulp protection, tooth conditioning and/or adhesive application. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately to placement of SDR® material.

3.4 Dispensing SDR® material

Danger of injury due to excessive force.

1. Apply slow and steady pressure on the syringe plunger or applicator gun.
2. Do not use excessive force – Syringe or Compula® tip rupture or ejection from applicator gun may result.

Compula® tips

The pre-dosed Compula® tip provides the combination of a unit dose Compules® tip with a metal cannula applicator tip.

1. Load Compules® tips gun with pre-dosed Compula® tip. Insert Compula® tip into the notched opening of the Compules® tips gun barrel. Be certain that the collar on the Compula® tip is inserted first.
2. Remove the colored cap from the Compula® tip. The Compula® tip may be rotated 360° to gain the proper angle of entrance into the cavity.
3. Dispense the material into the cavity preparation using a slow, steady pressure. **DO NOT USE EXCESSIVE FORCE.**
4. To remove the used Compula® tip, be sure that the Compules® tips gun plunger is pulled back completely by allowing the handle to open to its widest position. Apply a downward motion to the front end of the Compula® tip and remove.

Syringes

1. Remove cap from the end of the syringe. To assure free flow of material from syringe, express a small amount onto pad, away from the patient field.
2. Attach disposable, black dispensing tip to end of the syringe. Turn tip clockwise ¼ to ½ turn to assure that it is fully seated. Tug on tip to be sure that it is locked into the collar of the syringe.
3. Material should flow freely with gentle pressure. **DO NOT USE EXCESSIVE FORCE.** If more than gentle pressure is required, remove from patient field and check for obstruction.
4. **DISCARD AND PROPERLY DISPOSE DISPENSING TIP IMMEDIATELY AFTER USE. REPLACE ORIGINAL CAP.** Do not store syringe with black dispensing tip in place. **STORE ONLY WITH ORIGINAL CAP.**

Note: It is recommended to pull back slightly on the syringe plunger after use to prevent excessive flow of material.

3.5 Placement of SDR® material

Note: As SDR® material is designed to conform to cavity shape, proper contact areas must be established with the matrix system selected. Material will not distend matrix band. If needed, manually distend matrix band with a suitable hand instrument such as a condenser or ball burnisher prior to and/or during light-curing.

1. Dispense SDR® material directly into preparation site/tooth surface using slow, steady pressure. Begin dispensing at the deepest portion of the cavity, keeping tip close to cavity floor. Gradually withdraw tip as cavity is filled. Avoid lifting the tip out of dispensed material while dispensing to minimize air entrapment. At the completion of dispensing, wipe tip against cavity wall while withdrawing from the operative field.
2. Within a few seconds, dispensed SDR® material will self-level, eliminating the need for further manipulation with hand instruments. In case of overfill, use a flocked applicator tip wetted with residual adhesive to remove excess. Any visible air bubbles should be pierced with a clean, sharp explorer prior to curing.
3. When used as a Bulk Fill Base material, most cavities may be filled in one bulk increment up to 4 mm as needed to fill the cavity 2 mm short of the occlusal cavosurface*. In deeper preparations, place material in 4 mm increments, thoroughly light-curing each increment.
4. Alternatively, SDR® material may be placed in a thin layer as a traditional “flowable” liner on exposed dentin.

* When used as a Pit & Fissure Sealant, direct, small class I restorations of permanent molars, class I and II restorations of deciduous molars or Core Buildup material, material may be placed to full contour including up to the occlusal cavosurface margin.

3.6 Curing

SDR® material is designed to be cured in increments up to a 4 mm depth/thickness.

- Light-cure each area of the restoration surface with a suitable visible light-curing unit designed to cure materials containing camphorquinone (CQ) initiator, i.e. spectral output containing 470 nm. Minimum light output must be at least 550 mW/cm² exposure for at least 20 seconds. Refer to curing light manufacturer’s recommendations for compatibility and curing recommendations.

Inadequate polymerization due to insufficient curing.

1. Check compatibility of curing light.
2. Check curing cycle.
3. Check curing output before each procedure.

3.7 Completion

1. When used as a base/liner, proceed immediately to complete the restoration using a methacrylate based universal/posterior restorative material following manufacturer’s Instructions for Use. Technique Tip: In most applications, a body shade of occlusal restorative material provides the best esthetic result. In case of heavily stained dentin, use of an opaque shade may be required. Do not disturb or allow contamination of cured surface or exposed adhesive. If contamination occurs, follow adhesive manufacturer’s Instructions for Use for re-application of adhesive. If surfaces remain undisturbed, additional application of adhesive between increments is neither necessary nor recommended.
2. If contouring, finishing, and/or polishing of cured SDR® material is necessary, use traditional rotary devices following manufacturer’s Instructions for Use.

4 Hygiene

Cross-contamination.

1. Do not reuse single use products. Dispose in accordance with local regulations.
2. Reprocess reusable products as described below.

4.1 Cleaning and Disinfection

To prevent syringes from exposure to spatter or spray of body fluids or contaminated hands, or oral tissues, use of a protective barrier is recommended to avoid syringe contamination.

Disinfect contaminated syringes with a water-based hospital-level disinfection solution according to national/local regulations.

Repeated disinfection may damage label.

Note: Destruction of the label by vigorous wiping.

Wipe syringe gently.

4.2 Sterilizing the Compules® tips gun

Disassembling

1. Partially close the applicator gun and place your thumb on the rear part of the hinge.
2. Push upwards and lift hinge. The applicator gun is separated in two parts and the plunger is exposed.
3. Remove restorative material with a soft paper tissue and alcohol of 70%.

Sterilization

Sterilize the applicator gun by steam autoclave (2.1-2.4 bar and 135-138 °C, 30-35 PSI and 275-280 °F).

It is not recommended to submerge Compules® tips gun into disinfection solutions.

Reassembling

1. After sterilization insert plunger into the applicator gun cylinder.
2. Press components together and snap hinge mechanism in place.
3. Do not continue to use damaged or soiled applicator guns.

5 Lot number () and expiration date ()

1. Do not use after expiration date. ISO standard is used: "YYYY-MM".
2. The following numbers should be quoted in all correspondences:
 - Reorder number
 - Lot number
 - Expiration date

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© DENTSPLY Caulk 2014-02-03
DENTSPLY Limited
DENTSPLY Italia S.r.l.
DENTSPLY DeTrey GmbH