SmartLite® Focus® Pen-Style LED Curing Light

“Pen-Style” LED Polymerisationslampe

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
SmartLite® Focus® curing light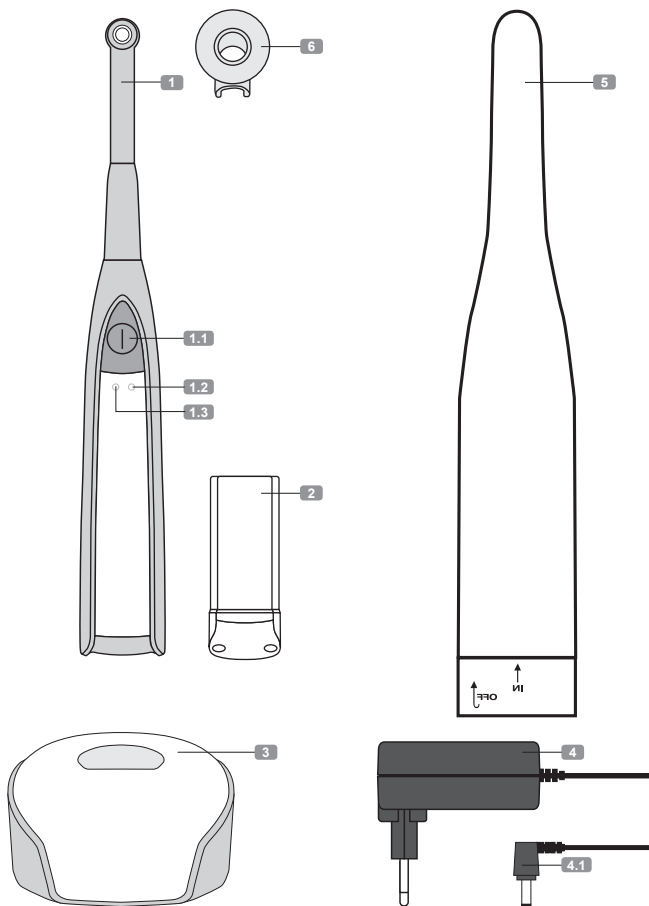


- 1 SmartLite® Focus® curing light
- 1.1 ON/OFF key
- 1.2 and 1.3 Battery lights
- 2 SmartLite® Focus® battery pack
- 3 SmartLite® Focus® charging base


- 4 Power connector incl. adapters
- 4.1 Jack
- 5 SmartLite® Sleeves
- 6 SmartLite® Focus® Shields

SmartLite® Focus® curing light – operation at a glance

Procedure A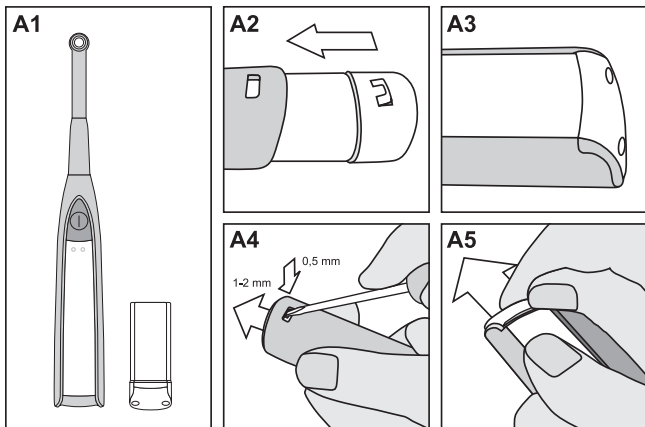

Procedure B

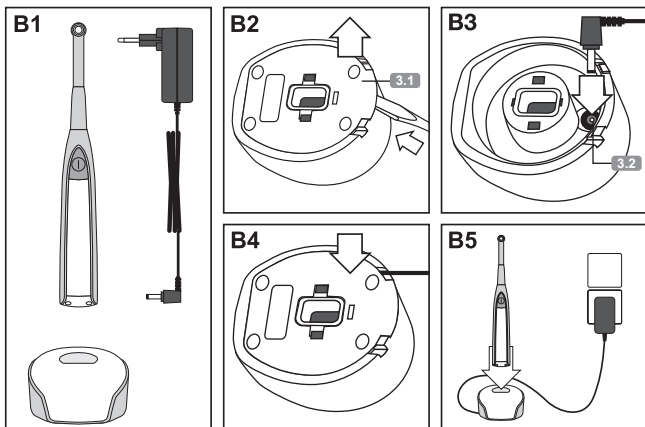
3.1 Floor panel


3.2 Charging base socket

SmartLite® Focus® curing light – operation at a glance

Procedure C

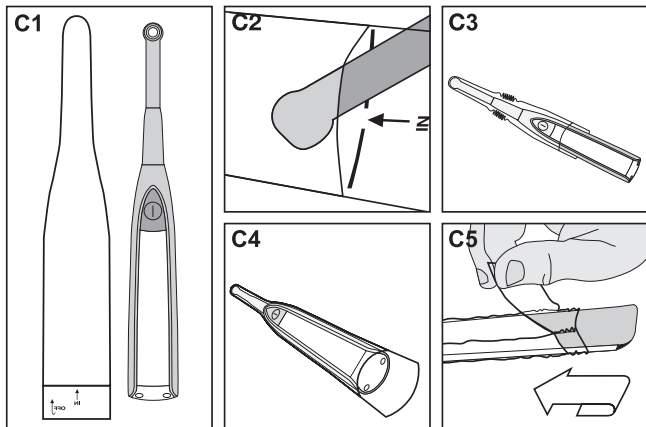Procedure D
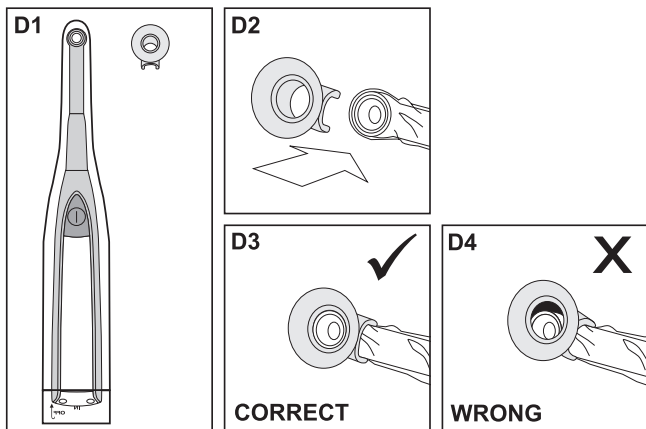
| | |
|--------------------------|----|
| Instructions for Use | 2 |
| Gebrauchsanweisung | 13 |
| Mode d'emploi | 24 |
| Istruzioni per l'uso | 35 |
| Instrucciones de uso | 46 |
| Инструкция по применению | 57 |

SmartLite® Focus®

Pen-Style LED Curing Light

CAUTION: For dental use only.

USA: Rx only.

| Content | Page |
|--|-----------|
| 1 Product description | 2 |
| 2 Safety notes | 4 |
| 3 Step-by-step instructions | 6 |
| 4 Hygiene | 7 |
| 5 Maintenance | 9 |
| 6 Reorder information, technical data, warranty terms | 10 |

1 Product description

The **SmartLite® Focus®** curing light is a cordless pen-style, LED light polymerization device for use by trained dental professionals in dental offices or dental laboratories.

SmartLite® Focus® curing light is characterized by:

- Small size and lightweight ergonomic design.
- Compact cordless design with convenient handling features and exchangeable battery pack.
- Smart battery charging mechanism with virtually permanent light operating availability.
- Individually adjustable LED tip, rotatable by 330°.
- LED tip design providing excellent intra-oral access.
- Polymerization area (optic effective cross-sectional area) of minimum 7.5 mm in diameter.
- Up to 20 seconds curing time per activation with audible signal every 10 seconds.
- Advanced heat management system limiting LED tip temperature.

1.1 SmartLite® Focus® curing light **1** – operation at a glance

1.1.1 ON/OFF key **1.1**

- Starts or disrupts the 20-second curing cycle.

1.1.2 Battery light

- **Permanently green **1.2**** (only indicated when SmartLite® Focus® curing light handpiece is inserted into the charging base **3**): SmartLite® Focus® curing light is fully recharged.
- **Flashing green **1.2**** (only indicated when SmartLite® Focus® curing light handpiece is inserted into the charging base **3**): SmartLite® Focus® curing light is in Standard Charging Mode.
- **Fast flashing green **1.2**** (only indicated when SmartLite® Focus® curing light handpiece is inserted into the charging base **3**): SmartLite® Focus® curing light is in Automatic Quick Charging Mode.
- **Flashing yellow **1.3****: SmartLite® Focus® is almost discharged and needs recharging. At first occurrence approximately 1 minute curing time left.
- **Permanently yellow **1.3****: SmartLite® Focus® curing light is fully discharged (no curing time left) and needs recharging.

1.1.3 Audible signals

- **One blip**: At start of curing cycle, after 10 seconds curing time together with flashing blue LED, and when SmartLite® Focus® curing light is fully recharged.
- **Two blips**: After 20 seconds consecutive curing time. Curing light automatically extinguishes after 20 seconds.
- **Repetitive short blip every 7 seconds together with flashing yellow battery light**: Only occurs when device is turned off. Approximately 1 minute operation time left at first occurrence. Device needs recharging.
- **One long blip together with permanent yellow battery light**: Only occurs when device is turned off. Device is fully discharged and needs recharging.

1.2 Indications

- Light-curing of dental materials which cure at a wavelength of 460-490 nm.

1.3 Contraindications

- Use in patients prone to photobiological reactions (including patients with solar urticaria or erythropoietic protoporphyria) or those currently undergoing treatment with photosensitizing pharmaceuticals.

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

Never modify SmartLite® Focus® or any of its equipment. Any modification may compromise safety and effectiveness.

2.1.1 SmartLite® Focus® handpiece 1

- Always make sure SmartLite® Focus® Shields 6 are securely fixed to SmartLite® Focus® to avoid accidental aspiration (press SmartLite® Focus® Shield firmly into correct position; see Procedure D for details).
- Always make sure light aperture is not covered by SmartLite® Focus® Shields (see Figure D3 and Figure D4).
- Do not try to remove LED tip from handpiece as they are permanently connected.
- The LED tip is rotatable by 330°. Do not attempt to fully rotate the LED tip.
- Do not use the device as a tissue retractor, as this may damage the connection between LED tip and handpiece.
- Do not use a damaged device, e.g. if among others the glass cover of the LED tip is scratched, broken or missing.
- The SmartLite® Focus® Shields will wear over time. Exchange defect shield against spare shields included in the package or available as refill (see also [6.1 Accessories]).
- Only authorized technicians should repair the handpiece or battery pack.

2.1.2 Charging base 3

- Low voltages are present inside the charging base (12 V DC). Do not open charging base when connected to power source. Use only under dry conditions. Do not use if the charging base is wet. Avoid short circuit between contact pins inside the charging base. Only authorized technicians should repair the charging base.
- Do not use for voltages different from the range indicated on the charging base.
- Note: Unplug power connector 4 from power source before disconnecting power connector from the charging base.
- Always make sure the charging base is placed offside the dental unit and only touched with clean/disinfected gloves to prevent from exposure to spatter or spray of body fluids.
- Always make sure handpiece is completely reprocessed before inserting it into the charging base.

2.1.3 Battery pack **2**

Prevent battery pack from short circuit during use and storage.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Anyone with a history of retinopathy should consult their eye specialist before operating this unit. Use the SmartLite® Focus® curing light extremely carefully and comply with all the necessary safety precautions (including wearing suitable, light filtering safety glasses).
- Anyone who has had a cataract operation may be especially sensitive to light and should be advised against undergoing treatment with a SmartLite® Focus® curing light unless adequate safety precautions are taken such as wearing suitable, light filtering safety glasses.
- Do not use a SmartLite® Focus® curing light which has not been properly reprocessed. Protect SmartLite® Focus® curing light from gross contamination by applying the single use SmartLite® Sleeves. SmartLite® Sleeves are intended for single use only. Discard after use. Do not reuse sleeves in other patients in order to prevent cross-contamination.
- For electrical medical devices specific safety precautions apply which have to be observed¹. Do not use mobile phones or other portable communication tools, as they could interfere with the function of the SmartLite® Focus® curing light.
- Never aim the light directly at unprotected soft tissues, as this may cause injury or irritation. Do not aim the light at eyes. Light reflected from the tooth surface may also injure eyes. Use the SmartLite® Focus® Shields supplied with the unit or suitable, light filtering safety glasses².
- Limit the action of the light to the area being treated.
- All dental curing lights cause a certain degree of heat development. Extended operation in areas near the pulp and soft tissues may result in severe damage. Do not cure the same tooth surface for more than 20 seconds at a time. If required, polymerize at intermitting intervals of 20 seconds each.
- Use of any accessories other than specified in this Instructions for Use may result in increased emissions or decreased immunity of the SmartLite® Focus® and its components.
- Do not use adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, closely watch SmartLite® Focus® and its components to verify normal operation in this configuration.

¹ For technical details (on electromagnetic emissions and electromagnetic immunity) see: www.dentsply.eu – chapter: IFU/MSDS.

² SmartLite® Focus® is classified as a Risk Group 2 device according to EN 62471.

3 Step-by-step instructions

3.1 Installation and charging (Procedure A and Procedure B)

1. Remove floor panel 3.1 of charging base by lifting floor panel at its back end (see Figure B2).
2. Connect jack 4.1 to charging base socket 3.2 (see Figure B3).
3. Close floor panel (see Figure B4) and plug in power connector 4 .
4. Make sure power socket used for power connector is accessible at all times in case of necessary emergency disconnection.
5. Insert battery pack 2 into SmartLite® Focus® handpiece 1 (see Figure A2 and Figure A3).
SmartLite® Focus® battery pack is pre-charged and ready to use upon purchase.
6. To recharge the battery, insert the handpiece 1 into the charging base 3 (see Figure B5). The battery light will be flashing green, indicating battery is recharging.

The SmartLite® Focus® Automatic Quick Charging Mode allows a minimum of 5 minutes operating time after 10 minutes recharging.

3.2 Operation

1. Protect LED tip from gross contamination by applying the single use SmartLite® Sleeves (see Procedure C).
2. Use the SmartLite® Focus® Shields supplied with the unit in combination with mounted SmartLite® Sleeves or use suitable, light filtering safety glasses (see Procedure D). Always make sure SmartLite® Focus® Shields are securely fixed to SmartLite® Focus® to avoid accidental aspiration (press SmartLite® Focus® Shield firmly into correct position). Always make sure SmartLite® Focus® Shields are properly mounted on SmartLite® Focus® without covering the light aperture (see Figure D3 and Figure D4).
3. Adjust the LED tip: The LED tip is rotatable by 330°, thus the position of light emission may be individually adjusted. The LED tip must be adjusted to an angle that allows optimal curing of the restoration. The LED tip should be positioned **as closely as possible to the restoration**.
4. Polymerization: Briefly press the ON/OFF key [1.1.1 ON/OFF key]. Audible operation signals are described in [1.1.3 Audible signals].
5. To stop curing light before the end of the 20 second cycle, press ON/OFF key at any time. To cure a restorative material that requires a curing time longer than 20 seconds, repeat activation by pressing the ON/OFF key after the end of each 20-second curing cycle (see also [2.2 Precautions]).
6. Curing times for a range of Dentsply Sirona materials are outlined in the Curing Guide provided. When using other products, please refer to respective product's Instructions for Use.

4 Hygiene

4.1 SmartLite® Sleeves **5**

The use of the SmartLite® Sleeves (see **Procedure C**) is a mandatory precautionary measure against gross contamination, but does not substitute reprocessing of the SmartLite® Focus® curing light.

1. Remove SmartLite® Sleeves immediately after use (see **Figure C5**).

Cross-contamination.

Infection.

1. SmartLite® Sleeves are intended for single use only. Do not reuse in order to prevent cross-contamination.
2. Discard in accordance with local regulations.

4.2 SmartLite® Focus® handpiece **1**

Electrical short-circuit or dangerous malfunction.

Injury.

1. Safeguard handpiece against liquid penetration during cleaning and disinfection.
2. Do not try to remove LED tip from handpiece as they are permanently connected.

Cross-contamination.

Infection.

1. Always make sure the handpiece is completely reprocessed before inserting it into the charging base.

NOTICE

Wrong cleaning or disinfection method.

Damage to SmartLite® Focus® curing light.

1. Do not submerge into liquids.
2. Do not clean or disinfect using solutions based on orange oil, acetone or hydrogen peroxide. Do not use the following products: Optim 33 TB, CaviCide1.
3. Removing of the battery pack is not recommended for regular cleaning and disinfection.
4. Start reprocessing within 1 hour after use.

4.2.1 Manual Cleaning

1. Right after use carefully remove sleeve without contaminating device and discard sleeve according to local regulations.
2. Discard examination gloves according to local regulations and disinfect hands with an appropriate bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations and use according to disinfectant solution manufacturer's Instructions for Use.
3. Use a new clean pair of examination gloves.

- Clean device with an appropriate pH-neutral, phosphate-free cleaning solution (e.g.: Dr. Schumacher Instru Plus [3%]³) according to local regulations and use according to cleaning solution manufacturer's Instructions for Use.
- Remove cleaning solution residue with a damp cloth. Use tap water to dampen cloth.
- Dry device with a lint-free single-use cloth.
- Pay special attention to device seams and insertions.

4.2.2 Manual Disinfection

- Thoroughly wipe cleaned device [4.2.1 Manual Cleaning] using a single use cloth dampened with an aldehyde- and alkylamine-free, alcohol-based, bactericidal, virucidal, fungicidal disinfectant solution (e.g.: BBraun Meliseptol[®] Rapid³) approved according to local regulations and use according to disinfectant manufacturer's Instructions for Use. Please note [4.2] for wrong cleaning or disinfection method.
- Pay special attention to device seams and insertions.
- After 1 minute remove disinfectant solution residue with a damp cloth. Use deionized water to dampen cloth.
- Dry device with a lint-free single-use cloth.
- Pay special attention to device seams and insertions.

Device can be disinfected for up to 6,000 times.

4.3 SmartLite[®] Focus[®] Shields **6**

NOTICE

Wrong cleaning or disinfection method.

Damage to SmartLite[®] Focus[®] Shields.

- Do not clean or disinfect with highly aggressive agents (e.g. solutions based on orange oil or acetone).
- Start reprocessing within 1 hour after use.

4.3.1 Manual Cleaning

- Discard examination gloves according to local regulations and disinfect hands with an appropriate bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations and use according to disinfectant solution manufacturer's Instructions for Use.
- Use a new clean pair of examination gloves.
- Clean device with an appropriate pH-neutral, phosphate-free cleaning solution (e.g.: Dr. Schumacher Instru Plus [3%]³) according to local regulations and use according to cleaning solution manufacturer's Instructions for Use.
- Pay special attention to device seams and insertions.
- Remove cleaning solution residue with a cloth soaked with tap water.
- Dry device with a lint-free single-use cloth.

4.3.2 Manual Disinfection

- Thoroughly wipe cleaned device [4.3.1 Manual Cleaning] using a single use cloth soaked with an aldehyde- and alkylamine-free, alcohol-based, bactericidal, virucidal, fungicidal disinfectant solution (e.g.: BBraun Meliseptol[®] Rapid³) approved according to local regulations and use according to disinfectant manufacturer's Instructions for Use.
- Pay special attention to device seams and insertions.
- After 1 minute remove disinfectant solution residue with a cloth soaked with deionized water.

³ Not a registered trademark of Dentsply Sirona, Inc.

4. Dry device with a lint-free single-use cloth.

SmartLite® Focus® Shields can be disinfected for up to 100 times.

4.3.3 Autoclaving (optional)

The SmartLite® Focus® Shields can also be autoclaved.

1. Steam autoclave SmartLite® Focus® Shields at 134 °C/2 bar = 273 °F/29 psi for 3 minutes 30 seconds.

SmartLite® Focus® Shields can be autoclaved for up to 60 times.

5 Maintenance

5.1 Light output monitoring

1. Make sure that the LED aperture is clean and scratch-free; otherwise light output will be reduced and may be insufficient for proper curing of the material.
2. The light intensity of SmartLite® Focus® should be checked frequently to ensure appropriate curing by using a commercially available radiometer. As radiometers differ greatly, the following procedure is recommended.
3. Upon receipt of SmartLite® Focus® check the light intensity with the radiometer of your choice and record the value. This initial reading is for reference only, and may not represent the actual light intensity.
4. For subsequent monitoring retest the light intensity with the same radiometer that was used for the initial reference and compare it to the reference intensity.
5. If radiometer value drops below reference intensity, light efficiency can be verified using the i•Cure. Place i•Cure on a sheet of paper on a flat surface. Choose i•Cure segment according to step height required (please note that the step height should be twice the curing depth to be confirmed). Fill with composite. Hold SmartLite® Focus® in close proximity to the upper aperture and cure. If the material on the lower aperture has been cured (i.e. cannot be scraped off with a plastic spatula), the curing depth according to ISO 4049:2009 equals half the chosen step height. If i•Cure test is successful, SmartLite® Focus® is ready to use.
6. Do not continue to use SmartLite® Focus® if both light output is below reference intensity and i•Cure test has failed.

5.2 Battery 2

- Battery is equipped with low self discharge technology resulting in a long operating life.
- Battery is pre-charged and ready to use upon purchase. Charging before first use may improve initial operating time but is not mandatory.
- When the battery light flashes **green** 1.2, the battery is charging. Upon complete recharging, the battery light remains permanently green. The battery needs approximately 3 hours to be fully recharged. SmartLite® Focus® has a Smart Recharging Technology. Depending on battery status, SmartLite® Focus® will automatically choose between Automatic Quick Charging Mode (fast flashing green light) and Standard Charging Mode (flashing green light).
- The SmartLite® Focus® Automatic Quick Charging Mode allows a minimum of 5 minutes operating time after 10 minutes recharging.
- When the battery light flashes **yellow (fast)** 1.3, the battery needs to be recharged. At first occurrence approximately 1 minute curing time remain for completion of the treatment. Light output is not reduced during this period. After this period, the device switches off automatically and the battery light remains permanently **yellow** indicating the need to recharge.

- If the battery pack **2** needs to be replaced, release battery pack by gently inserting a screwdriver into the hole at the lower back end of the handpiece. The screwdriver is just needed to unlock the battery pack, please use fingers to completely remove battery pack as shown in **Figure A4** and **Figure A5**. Do not use the screwdriver as a lever, in order to avoid damage to the SmartLite® Focus® handpiece.

6 Reorder information, technical data, warranty terms

6.1 Accessories

Accessory

| | Reorder no. |
|---|--------------------|
| SmartLite® Focus® battery pack 2 | 644.50.021 |
| SmartLite® Sleeves 5 | 644.50.030 |
| SmartLite® Focus® Shields 6 | 644.50.040 |
| SmartLite® Focus® power connector incl. adapters 4 | 644.50.050 |
| SmartLite® Focus® refill (curing light and battery pack) 1 | 644.50.060 |
| SmartLite® Focus® charging base 3 | 644.50.070 |
| i•Cure | 644.40.054 |

6.2 Serial number

Handpiece and battery pack have different serial numbers. Handpiece serial number can be found inside the handpiece after battery pack removal. Battery pack serial number is located on battery pack.

The serial number **[SN]** should be quoted in all correspondence, which requires identification of the product.

| | | |
|--|-------|-------|
| Serial number format SmartLite® charging base: | SLXX | xxxxx |
| Serial number format SmartLite® Focus® handpiece: | SFXX | xxxxx |
| Serial number format SmartLite® Focus® battery pack: | SFBXX | xxxxx |

6.3 Specifications and equipment class

Environmental Note

Device and battery disposal:

This device is provided with a nickel metal hydride battery. Device and battery must not be disposed in normal domestic waste. For environmental reasons, please dispose device and battery according to local environmental guidelines or regulations.

AC supply connection: 100 V...240 V / ~ 50 Hz...60 Hz

Power input: Maximum 14 W

Equipment class: Class II equipment

Follow instructions for use:

Protection from ingress of liquids into handpiece:
IPX4

| | |
|--|--|
| Operation: | Ambient temperature: Between 0 °C and +30 °C (32 °F and 86 °F) Relative humidity: Between 20% and 90% |
| Storage: | Ambient temperature: Between -10 °C and +35 °C (14 °F and 95 °F) |
| Transport: | Ambient temperature: Between -10 °C and +50 °C (14 °F and 122 °F) |
| Battery performance: | <ul style="list-style-type: none"> • Low self-discharge technology results in a long operating life. • Battery is pre-charged and ready to use upon purchase. • Time for battery recharge: Approximately 3 hours. • Automatic Quick Charging Mode: allows a minimum of 5 minutes operating time after 10 minutes recharging. |
| Battery over current/temperature protection: | Resettable fuse |
| Light emitting diode: | 3 W LED |
| Average light intensity: | Approximately 1000 mW/cm ² (CureRite™) |
| Output peak wavelength range: | Between 460 nm and 490 nm (intensity maximum around 475 nm) |
| Unit handpiece dimensions: | Length: 240 mm Width: 31 mm |
| Unit weight: | Handpiece with battery pack: 90 grams Charging base with power connector: 200 grams |

Complies with the following Directives/Standards:

| | |
|---------------|---|
| 93/42/EEC | Council Directive 93/42/EEC of 14 June 1993 concerning medical devices amended by directive 2007/47/EC, annex 1 |
| 2002/95/EC | Restriction of the use of hazardous substances in electrical and electronic equipment |
| IEC 60601-1 | 2005 – Medical Electrical Equipment (General requirements for basic safety and essential performance) |
| IEC 60601-1-2 | 2005 – Medical electrical equipment – Part 1-2: General requirements for safety and essential performance – Collateral standard: Electromagnetic compatibility – Requirements and tests |
| ISO 10650 | 2015 – Dentistry – Powered polymerization activators |
| EN 62471 | 2008 – Photobiological Safety of Lamps and Lamp Systems |
| IEC 62471 | 2006 – Photobiological Safety of Lamps and Lamp Systems |

The SmartLite® Focus® curing light complies with:

6.4 Warranty terms

Dentsply Sirona grants a 2-year warranty on all components of the SmartLite® Focus® except the battery. The battery is covered by a 1-year warranty. The warranty commences on the date of purchase. Within the warranty period, Dentsply Sirona will eliminate free of charge any defects in the appliance resulting from faults in material or workmanship either by repairing or exchanging parts or exchanging the whole device at Dentsply Sirona's discretion.

Not covered by this warranty: Damage arising from improper use (operation with incorrect current/voltage, unsuitable power point, breakage, cleaning by other than the recommended methods), normal wear and defects which have a negligible effect on the value or operation of the appliance.

This warranty becomes void if repairs are undertaken by unauthorized persons.

This warranty extends to every country where this device is supplied by Dentsply Sirona or its appointed distributor and where no import restrictions or legal regulations hinder or prevent service being given under warranty.

Service under this warranty does not affect the expiration date of the warranty. The warranty on parts or entire devices which are exchanged ends when the warranty on this device expires.

In the event of a claim of this device, return the complete device (charging unit and the LED curing light) together with the invoice to your dealer or send it to your nearest Dentsply Sirona Service Center.

All other claims including those for damages resulting from this warranty are excluded unless our liability is legally mandatory.

© Dentsply Sirona 2017-02-03

[These Instructions for Use are based on Master Version 11]

SmartLite® Focus®

“Pen-Style” LED Polymerisationslampe

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Dentsply DeTrey GmbH
Distributed in U.S.A by