# XP Bond<sup>®</sup>

## Universal Total Etch Adhesive

Adhésif universel monocomposant

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## Universal Total Etch Adhesive

CAUTION: For dental use only.

Content	Page
<b>1 Product description</b> .....	<b>2</b>
<b>2 Safety notes</b> .....	<b>3</b>
<b>3 Step-by-step instructions</b> .....	<b>5</b>
<b>4 Hygiene</b> .....	<b>8</b>
<b>5 Lot number and expiration date</b> .....	<b>8</b>

### 1 Product description

**XP Bond®** adhesive is a universal etch-and-rinse one-bottle dental adhesive designed to bond resin-based materials to enamel and dentin as well as to metals and ceramic.

**XP Bond®** adhesive stands for eXtra Performance due to high bond strength to enamel and dentin, easy and comfortable application and a high degree of technique robustness.

**XP Bond®** adhesive, when used on its own, is designed to be used with light cured resin-based restorative materials and cements.

**XP Bond®** adhesive, when mixed with Self Cure Activator (available separately), is designed to be used with dual cure/self cure resin materials. For the use in conjunction with Self Cure Activator please refer to the Instructions for Use "Self Cure Activator".

#### 1.1 Delivery forms

- Easy-squeeze bottle

#### 1.2 Composition

- Carboxylic acid modified dimethacrylate (TCB resin)
- Phosphoric acid modified acrylate resin (PENTA)
- Urethane dimethacrylate (UDMA)
- Triethyleneglycol dimethacrylate (TEGDMA)
- 2-Hydroxyethylmethacrylate (HEMA)
- Butylated benzenediol (stabilizer)
- Ethyl-4(dimethylamino)benzoate
- Camphorquinone
- Functionalised amorphous silica
- Tertiary butanol

### 1.3 Indications

Self-priming adhesive for:

- Direct, light cured composite and compomer restoratives.
- Indirect restorations; light cured, resin cemented inlays, onlays, crown and bridge retainers.
- Composite/compomer, ceramic and amalgam repairs.
- Cavity varnish for use with fresh amalgam.

For the use of XP Bond® for the following indications please refer to the Instructions for Use “Self Cure Activator”:

- Direct, dual cure or self cure composite restorations and core build-ups.
- Indirect restorations; dual cured and self cured resin cemented inlays, onlays, crown and bridge retainers.
- Dual cured and self cured resin cemented endodontic post cementation.

For the use of XP Bond® for adhesive bonding of direct amalgam restorations please refer to the Instructions for Use “Amalgam Bonding Accessory Kit”<sup>1</sup>.

### 1.4 Contraindications

- Use with patients who have a history of severe allergic reactions to acrylate or methacrylate resins or any of the other components.
- Direct application to dental pulp (direct pulp capping).

### 1.5 Compatible tooth conditioners

XP Bond® adhesive is used following phosphoric acid conditioning with conventional dentin/enamel conditioners including Dentsply Sirona 34% and 36% conditioners (see Instructions for Use of selected conditioner).

### 1.6 Compatible restorative materials

XP Bond® adhesive is used in conjunction with suitable restorative or cement material and is chemically compatible with conventional methacrylate-based composite restorative materials including Dentsply Sirona visible light cured composite restorative or cement materials (see Instructions for Use of selected restorative or cement material).

## 2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

#### Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

### 2.1 Warnings

The material contains polymerizable acrylate and methacrylate monomers which may be irritating to skin, eyes and oral mucosa and may cause sensitization by skin contact and allergic contact dermatitis in susceptible persons.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes rinse with plenty of water and seek medical attention.

<sup>1</sup> Not available in all countries.

- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

## 2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.
- Tightly close bottle immediately after use.
- Use in a well-ventilated area. Avoid inhaling vapor.
- Flammable: Keep away from sources of ignition. Take precautionary measures against static discharges.
- Avoid the product saturating the gingival retraction cord. If it soaks into the cord, it may set hard and bond the cord to the underlying tooth surface making removal difficult.
- Applicator tips are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
- Interactions:
  - Do not use eugenol- and hydrogen peroxide-containing materials in conjunction with this product since they may interfere with hardening of the product.
  - If mineral-impregnated (e.g. ferric compounds) retraction cords and/or hemostatic solutions are used in conjunction with adhesive procedures, marginal seal may be adversely affected, allowing microleakage, subsurface staining and/or restoration failure. If gingival retraction is necessary, use of plain, non-impregnated cord is recommended.
  - XP Bond® adhesive is a light cured material. Protect from ambient light.
  - Insufficient data exist to support the use of desensitizing agents and/or cavity cleansing agents with XP Bond® adhesive. Recommended pre-treatment is pumice or prophylaxis paste with a rubber cup. (See Step-by-Step Instructions)
  - If H<sub>2</sub>O<sub>2</sub> has been used to clean the cavity, proper rinsing is essential. Higher concentration H<sub>2</sub>O<sub>2</sub> may interfere with the setting of polymerizable material and should not be used prior to the application of XP Bond® adhesive.

## 2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Sensitization. Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

## 2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Keep the product out of direct sunlight and store in a well ventilated place.
- Store the product at temperatures between 10 °C and 28 °C (50 °F and 82 °F). Use the product at room temperature.
- Protect from moisture.
- Do not use after expiration date.

### 3 Step-by-step instructions

#### 3.1 Direct restoration (light cured composite resin and compomers)

##### 3.1.1 Cleaning

1. Clean uninstrumented enamel and dentin with a rubber cup and pumice or a non-fluoride cleaning paste such as Nupro® prophylaxis paste.
2. Wash thoroughly with water spray and air dry.
3. Clean freshly instrumented enamel and dentin with water spray, then air dry.

##### 3.1.2 Pulp protection

1. In deep cavities cover the dentin close to the pulp (less than 1 mm) with a hard-setting calcium hydroxide liner (Dycal® calcium hydroxide liner, see Instructions for Use) leaving the rest of the cavity surface free for bonding.

##### 3.1.3 Tooth conditioning/dentin pre-treatment

When used as a bonding agent for direct restorative materials, it is recommended to follow the total etch technique described in this chapter. Prior to use, please see the Instructions for Use of the respective conditioner.

##### Application of 34%-36% phosphoric acid conditioner

1. After application of rubber dam or other suitable isolation technique, apply the respective conditioner.
2. Apply the respective conditioner to the cavity surfaces starting at the enamel margins.
3. For best results, condition enamel for at least 15 seconds and dentin for 15 seconds.

##### Rinsing and blot drying

1. Remove conditioner with aspirator tube and/or vigorous water spray and rinse conditioned areas thoroughly for at least 15 seconds.
2. Remove rinsing water completely by blowing gently with an air syringe or by blot drying with a cotton pellet. Do not desiccate dentin.
3. Proceed immediately to application of XP Bond® adhesive.

##### Contamination.

Restoration failure.

1. Once the surfaces have been properly treated, they must be kept uncontaminated.
2. If salivary contamination occurs, thoroughly clean with vigorous water spray, dry, and repeat conditioning procedure for 5 seconds only. Rinse and dry as described above.

##### 3.1.4 Dosage

1. To prevent the bottle from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the bottle is handled offside the dental unit with clean/disinfected gloves.
2. Dispense XP Bond® adhesive directly onto a clean applicator tip, making sure that the bottle does not come in direct contact with the applicator tip, or place one to two drops of XP Bond® adhesive into a clean CliXdish™ mixing well or standard dappen dish.
3. Replace cap promptly.

Technique Tip: Material in a closed CliXdish™ mixing well will remain usable for up to 15 minutes. Material dispensed to a standard mixing well must be used immediately.

### 3.1.5 Application and curing

1. Using the applicator tip, apply the product wetting all cavity surfaces uniformly. Avoid pooling. This surface should remain undisturbed for 20 seconds.
2. Evaporate solvent by thoroughly drying with clean, dry air from a dental syringe for at least 5 seconds. Use a vacuum aspirator to prevent spatter of the adhesive onto the mucosa. Surface should have a uniform glossy appearance. If not, repeat application and air dry.
3. Cure XP Bond® adhesive for 10 seconds<sup>2</sup> using a curing light.

#### **Insufficient curing.**

Inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing cycle.
3. Check curing output.

4. Place restorative material over the cured XP Bond® adhesive as per restorative material manufacturer's Instructions for Use.

### 3.2 Light cured composite, ceramic and amalgam repairs

#### 3.2.1 Preparation

1. Roughen and clean fractured restoration surface with a fine diamond bur and create mechanical retention, if possible.

#### 3.2.2 Tooth conditioning/dentin pre-treatment

1. Etch fractured tooth surface with the respective conditioner as outlined in 3.1.3 Tooth conditioning/dentin pre-treatment.

#### 3.2.3 Treatment of restoration

1. Etch fractured restoration surface with the respective conditioner for at least 15 seconds as outlined in 3.1.3 Tooth conditioning/dentin pre-treatment.

#### 3.2.4 Application and curing

1. Apply adhesive to the fractured restoration surfaces and light cure<sup>2</sup> the adhesive [3.1.5 Application and curing].

#### 3.2.5 Completion

1. Complete repair with placement and curing of desired shade(s) of light cured composite restorative following manufacturer's Instructions for Use.

### 3.3 Cementation of light transmitting indirect restoration with a light cured resin cement

#### 3.3.1 Treatment of restoration

1. Treat internal bonding surface of restoration according to manufacturer's or dental laboratory's Instructions for Use, i.e. etching, mechanical roughening and/or Silane Coupling Agent application.
2. Apply a single coat of XP Bond® adhesive to the internal bonding surface of the restoration. Immediately air dry for 5 seconds.

Light curing of applied XP Bond® adhesive is not necessary.

<sup>2</sup> Check curing light for minimum light output of at least 800 mW/cm<sup>2</sup>. Cure for at least 20 seconds if light output is between 500 and 800 mW/cm<sup>2</sup>.

### **3.3.2 Treatment of tooth preparation**

1. Clean and condition tooth preparation, apply and light cure the material as described in 3.1 Direct restoration (light cured composite resin and compomers).

### **3.3.3 Cementation**

1. Prepare and apply light cured resin cement according to manufacturer's Instructions for Use.

### **3.4 Cavity varnish for use with fresh amalgam**

When used as a cavity varnish, XP Bond® adhesive is not an amalgam adhesive.

#### **3.4.1 Preparation**

1. Finish preparation by removing existing restorations and/or caries.

#### **3.4.2 Cleaning**

Clean preparation and place a hard-setting calcium hydroxide liner (Dycal® calcium hydroxide liner, see Instructions for Use) if needed, according to:

1. Cleaning [3.1.1 Cleaning].
2. Pulp protection [3.1.2 Pulp protection].

#### **3.4.3 Tooth conditioning/dentin pre-treatment**

1. Rinse and carefully air dry cavity preparation, but do not desiccate exposed dentin.
2. Condition enamel and dentin following section 3.1.3 Tooth conditioning/dentin pre-treatment.

#### **3.4.4 Application**

1. Apply and light cure XP Bond® adhesive as described for direct restorations, see section 3.1.3 Tooth conditioning/dentin pre-treatment.

#### **3.4.5 Completion**

1. Place and condense amalgam (e.g. Dispersalloy® Dispersed Phase Alloy, see Instructions for Use) as per manufacturer's directions.

## 4 Hygiene

### **Cross-contamination.**

Infection.

1. Do not reuse single use products. Dispose in accordance with local regulations.
2. Reprocess reusable products as described below.

### 4.1 CliXdish™ mixing well

For reprocessing instructions please refer to the Instructions for Use of the CliXdish™, which is available on our webpage at [www.dentsply.eu/IFU](http://www.dentsply.eu/IFU). If requested, we will send you a free printed copy of the Instructions for Use in the language you require within 7 days. You can use the order form provided on our webpage for this purpose.

## 5 Lot number ( ) and expiration date ( )

1. Do not use after expiration date. ISO standard is used: "YYYY-MM"
2. The following numbers should be quoted in all correspondence:
  - Reorder number
  - Lot number
  - Expiration date

© Dentsply Sirona 2016-12-02

[These Instructions for Use are based on Master Version 07]

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Dentsply DeTrey GmbH