# Xeno<sup>®</sup> III

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# Xeno<sup>®</sup> III

## Single Step Self Etching Dental Adhesive

CAUTION: For dental use only.

USA: Rx only.

Content	Page
<b>1 Product description</b> .....	<b>2</b>
<b>2 Safety notes</b> .....	<b>3</b>
<b>3 Step-by-step instructions</b> .....	<b>5</b>
<b>4 Hygiene</b> .....	<b>6</b>
<b>5 Lot number and expiration date</b> .....	<b>7</b>

### 1 Product description

**Xeno<sup>®</sup> III<sup>1</sup>** adhesive is a single step self etching dental adhesive designed to bond direct light cured restoratives to enamel and dentin.

**Xeno<sup>®</sup> III** adhesive is available in bottles or as Unit Dose delivery.

**Xeno<sup>®</sup> III** adhesive has the following features:

- This single step bonding system acid etches, primes and bonds both enamel and dentin simultaneously.
- No water rinsing after application.

#### 1.1 Delivery forms

- Easy squeeze bottle
- Unit Dose delivery

#### 1.2 Composition

##### Liquid A

- 2-hydroxyethylmethacrylate (HEMA)
- Purified water
- Ethanol
- Butylated hydroxy toluene (BHT)
- Highly dispersed silicon dioxide

##### Liquid B

- Phosphoric acid modified methacrylate (Pyro-EMA)
- Mono fluoro phosphazene modified methacrylate resin (PEM-F)
- Urethane dimethacrylate (UDMA)
- Butylated hydroxy toluene (BHT)

<sup>1</sup> Xeno<sup>®</sup> III adhesive is the third generation of a self etching dental adhesive developed by Dentsply Sankin and marketed in Japan under the brand Xeno CF II Bond.

- Camphorquinone
- Ethyl-4(dimethylamino)benzoate

### 1.3 Indications

- Direct restoration of all cavity classes in combination with light curing resin-based restorative materials.

### 1.4 Contraindications

- Use with patients who have a history of severe allergic reaction to methacrylate resins or any of the components.
- Direct application to dental pulp (direct pulp capping).
- Use with chemical or dual cure materials.

### 1.5 Compatible Restorative Materials

Xeno<sup>®</sup> III adhesive is used in conjunction with suitable restorative material and is chemically compatible with conventional methacrylate-based composite restorative materials including Dentsply Sirona visible light cured composite restorative materials (see Instructions for Use of selected restorative material).

## 2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

#### Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

### 2.1 Warnings

The material contains polymerizable methacrylate monomers which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons. The material is acidic and may cause burns.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

## 2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.
- To prevent the bottle from exposure to spatter or spray of body fluids or contaminated hands it is recommended that the bottle is handled offside the dental unit with clean/disinfected gloves.
- Tightly close bottle immediately after use.
- Use in a well-ventilated area. Avoid inhaling vapor.
- Flammable: Keep away from sources of ignition. Take precautionary measures against static discharges.
- Avoid the product saturating the gingival retraction cord. If it soaks into the cord, it may set hard and bond the cord to the underlying tooth surface making removal difficult.
- Unit Dose delivery and applicator tips are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
- Interactions:
  - Do not use eugenol- or hydrogen peroxide-containing materials in conjunction with this product since they may interfere with hardening and cause softening of the product.
  - If mineral-impregnated (e.g. ferric compounds) retraction cords and/or hemostatic solutions are used in conjunction with adhesive procedures, marginal seal may be adversely affected, allowing microleakage, subsurface staining and/or restoration failure. If gingival retraction is necessary, use of plain, non-impregnated cord is recommended.
  - Xeno® III adhesive is a light cured material. Protect from ambient light.
  - If refrigerated, allow material to reach room temperature prior to use.

## 2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

## 2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Keep the product out of direct sunlight and store in a well-ventilated place.
- Store at temperatures between 2 °C and 24 °C (35 °F and 75 °F). Use the product at room temperature.
- Protect from moisture.
- Do not freeze.
- Do not use after expiration date.

### 3 Step-by-step instructions

#### 3.1 Cavity preparation

1. Prepare the cavity so that no residual restorative material is left.
2. In cases where no preparation or instrumentation has been carried out, roughen the surface to be bonded with a finishing bur.
3. For restoration of cervical lesions, finish enamel and dentin margins. Enamel margins should be beveled to mask the restoration/tooth interface. This may require instrumentation.
4. Rinse surface with water spray.
5. Remove rinsing water completely by blowing gently with an air syringe or by blot-drying with a (cotton) pellet. Do not desiccate the tooth substance.
6. Use a rubber dam or cotton rolls to isolate the cavity from contamination. If contamination occurs, thoroughly clean with vigorous water spray and repeat steps 4 and 5.

#### 3.2 Pulp protection

1. In deep cavities cover the dentin close to the pulp (less than 1 mm) with a hard-setting calcium hydroxide liner (Dycal<sup>®</sup> calcium hydroxide liner, see Instructions for Use) leaving the rest of the cavity surface free for bonding.

#### 3.3 Dosage

##### Easy squeeze bottle

Dispense an equal amount of Liquid A and Liquid B into a CliXdish™ mixing well or standard dappen dish or mixing well.

For optimum dosage, proceed as follows:

##### Liquid A

1. Shake bottle two to three times prior to use.
2. Incline bottle. If liquid does not drop, squeeze.
3. Clean the dropper of the bottle after each application using a soft paper tissue.
4. Replace **green** cap promptly.

##### Liquid B

5. Incline bottle; wait for approximately two seconds. Squeeze until liquid drops.
6. Replace **black** cap promptly.
7. Mix liquid thoroughly for approximately five seconds with the applicator tip supplied.
8. Technique Tip: Material in a closed CliXdish™ mixing well will remain usable for **up to 60 minutes**. Material dispensed into a standard mixing well must be used immediately.

##### Unit Dose delivery

1. It is not necessary to open the larger round cap of the applicators' container. Open the triangular tab. Rotate the container so that the open tab is at the bottom; incline and shake container until one applicator emerges.
2. Detach a Unit Dose by folding along the perforation and pull the printed side upwards. Hold the Unit Dose blister between thumb and forefinger. Firmly press green area to force liquid into orange mixing and dispensing well.
3. Perforate foil on top of mixing and dispensing well by placing the applicator at right-angles to the centre of the orange circle. Enlarge the opening with circular movements to maximum width. Create a homogeneous mixture by stirring for at least 5 seconds. To ease removal of the adhesive the Unit Dose may be inclined.

### 3.4 Application and curing

1. Using the applicator tip, apply the product sufficiently (larger cavities may require 2 or more applications) wetting all cavity surfaces uniformly. Avoid pooling.
2. Leave undisturbed for at least 20 seconds.
3. Evaporate solvent by thoroughly drying with clean, dry air from a dental syringe until there is **no more movement** of the adhesive, but for **at least 2 seconds**. Use a vacuum aspirator to prevent spatter of the adhesive onto the mucosa.
4. Cure Xeno® III adhesive **for 10 seconds** using a curing light. Ensure uniform exposure of all surfaces.

#### Insufficient curing.

Inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing cycle.
3. Check curing output.

5. Immediately place restorative material over the cured Xeno® III adhesive as per restorative material manufacturer's Instructions for Use.

#### Multiple cavities.

When restoring multiple cavities it is advisable to fully restore one cavity (adhesive and restorative) before proceeding to the next cavity.

## 4 Hygiene

#### Cross-contamination.

Infection.

1. Do not reuse single use products. Dispose in accordance with local regulations.
2. Reprocess reusable products as described below.

### 4.1 CliXdish™ mixing well

For reprocessing instructions please refer to the Instructions for Use of the CliXdish™, which is available on our webpage at [www.dentsply.eu/IFU](http://www.dentsply.eu/IFU). If requested, we will send you a free printed copy of the Instructions for Use in the language you require within 7 days. You can use the order form provided on our webpage for this purpose.

## **5 Lot number ( ) and expiration date ( )**

1. Do not use after expiration date. ISO standard is used: “YYYY-MM”
2. The following numbers should be quoted in all correspondence:
  - Reorder number
  - Lot number
  - Expiration date

**© Dentsply Sirona 2016-12-06**

**[These Instructions for Use are based on Master Version 05]**

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Dentsply DeTrey GmbH